# SZABO SCANDIC

Part of Europa Biosite

## Produktinformation

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[mail@szabo-scandic.com](mailto:mail@szabo-scandic.com)

[www.szabo-scandic.com](http://www.szabo-scandic.com)

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

**DENGUE Serotyping**  
**Detection and discrimination of the Dengue Virus with Real Time PCR**

REV ST.RT-03-ENITA.1

**REF** RT-03 - 48 tests

## Instructions For Use

### **INTRODUCTION AND PURPOSE OF USE**

The Dengue Serotyping system is a quantitative test that allows the RNA amplification, by means of Real Time PCR, of anchored capsid protein region of Dengue Virus (DENV), discriminating the 4 serotypes: DENV-1, DENV-2, DENV-3 and DENV-4.

The Procedure allows the detection of the RNA target by means a One step retro-amplification reaction.

The analysis of the results is made using a Real Time PCR analyzer (thermal cycler integrated with a system for fluorescence detection and a dedicated software).

### **CONTENT**

The kit contains reagents enough to perform 48 amplification tests:

	<b>Quantity</b>	<b>Description</b>
R1	3 x 20 µl	<b>Enzyme mMix</b> Reverse Transcriptase e Taq Polymerase enzymes (Pink Cap)
R2	3 x 270 µl	<b>DENV1-2 probe Mix</b> Upstream primer, downstream primer, Target probe DENV-1 (FAM), Target probe DENV-2 (VIC), Internal Control (IC) probe (CY5), Nuclease-free water, ROX, dNTPs, Tris-HCl, KCl, MgCl <sub>2</sub> (White Cap)
R3	3 x 270 µl	<b>DENV3-4 probe Mix</b> Upstream primer, downstream primer, Target probe DENV-3 (FAM), Target probe DENV-4 (VIC), Internal Control (IC) probe (CY5), Nuclease-free water, ROX, dNTPs, Tris-HCl, KCl, MgCl <sub>2</sub> (Blu Cap)
R4	3 x 35 µl	<b>Denv1-2 Positive Control</b> synthetic RNA corresponding to anchored capsid protein region (Green Cap)
R5	3 x 35 µl	<b>Denv3-4 Positive Control</b> synthetic RNA corresponding to anchored capsid protein region (Yellow Cap)
R6	3 x 160 µl	<b>Internal control</b> phage MS2 (Red Cap)
R7	1 x 100 µl	<b>Negative Control</b>

### **MATERIALS AND STRUMENTATION REQUIRED BUT NOT SUPPLIED**

Disposable latex powder-free gloves or similar material;

Bench microcentrifuge (12,000 - 14,000 rpm);

Micropipettes and Sterile tips with aerosol filter;

Vortex;

Plastic materials (microplate and optical adhesive cover);

Heat block (only for extraction)

EZ1 Advanced XL DSP Virus Card. - Ref. 9018703 - QIAGEN.

ATL Buffer - Ref. 939016 - QIAGEN.

2 ml Microtube PP Skirted - Ref. 72.694 - QIAGEN

QIASymphony plastic materials (Filter Tips, sample prep Cartridges, 8 Rod covers, etc...)

### **Reagents**

The **Dengue Serotyping kit** was developed and validated to be used with the following extraction method:

Manual Extraction

Ref. 52906. QIAmp Viral RNA Mini Kit

The kit allows the manual RNA extraction from tested samples. The kit contains reagents for 250 samples. (QIAGEN)

Automatic extraction

Ref. 62724. EZ1 XL DSP Virus Kit.

The kit allows the automatic viral RNA from Human samples.

The kit contains reagents for 48 samples. (QIAGEN)

Ref.937055. QIASymphony® DSP Virus/Pathogen midi kit.

The kit allows the automatic viral RNA from Human samples. The kit contains reagents for 96 samples. (QIAGEN)

## **Instruments**

The Dengue Serotyping kit was evaluated to be used with the following instruments:

### **Extraction System**

Ref. 9001492. EZ1 Advanced XL.

Robotic Workstation for the automatic purification of the nucleic acids until 14 samples simultaneously (QIAGEN)

Ref. 9001297 QIASymphony SP

Robotic Workstation for the automatic purification of the nucleic acids (QIAGEN).

### **Real Time PCR**

- *7500 Fast from Lifetechnologies*
- *Rotor-Gene Q MDx from QIAGEN*

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturer's instruction and recommendations.

### **SAMPLES**

The Dengue Serotyping kit must be used with extracted RNA from biological samples: Serum and Plasma. Collected samples must be shipped and stored at +2 - +8°C and used within 3 days from the collected data.

Store the sample at -20°C if it is used after 3 days.

### **PRECAUTIONS FOR USE**

- This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.
- Carefully read this Instruction For Use before using the kit.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: RNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- Periodically wipe the working area with 0,5% hypochlorite.
- Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.
- Use powder-free gloves. Do not leave fingerprints on optical caps. Do not write on caps as this may cause an interference with fluorescent detection.
- Avoid any contact between hands and eyes or nose during specimen collection and testing.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood. Waste should be discarded according to local law.
- Never pipette solutions by mouth.
- Use calibrated and regularly checked pipettes and instrumentation only.
- Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.
- Wash hands carefully after handling samples and reagents.
- Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
- Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

### **LIMITS OF THE METHOD**

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and/or controls. Therefore, you should:

- physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products
- use tips with filters to prevent cross-contamination between samples
- use disposable gloves and change them frequently
- carefully open test tubes to prevent aerosol formation
- close every test tube before opening another one

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

Patient Drug treatment may interfere with the final result of the molecular biology analysis.

The proper functioning of the amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

### **WARNINGS**

- Read carefully the instructions for use before using this test.
  - Use only extracted RNA from Serum and/or Plasma
  - Do not mix reagents from different lots.
  - Thaw and carefully mix the reagents of the kit before use.
  - The PCR mix has to be freshly prepared every time.
- After reconstitution, the amplification master mix must be used in one time (16 reactions). Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.
- Do not use beyond the expiration date which appears on the package label.

- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- In case of damaged package, contact the technical support before using the kit.
- In case of any serious incident that has occurred in relation to the device, a notice shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### STORAGE AND STABILITY

Store the product Dengue Serotyping at -20°C.

The Dengue Serotyping kit is shipped on dry ice. The kit components should be frozen.

If one or more components are not frozen upon receipt or if the tubes have been compromised during transport, contact Clonit srl for assistance.

An intact and well stored product has a stability of 6 months from the date of production. Do not use beyond the expiration date which appears on the package label.

Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently

### ANALYTICAL PROCEDURE

#### Manual Extraction

Ref. 52906. *QIAmp Viral RNA Mini Kit*

#### Procedure to Serum and Plasma

Follow the instructions inside the kit *QIAmp Viral RNA Mini Kit*.

After the incubation period for 10 min at room temperature, add 5 µl of Internal Control

Follow the instructions inside the kit

Elute the sample in 50 µl of buffer AVE.

Samples are now ready for amplification or storage at -80°C

#### Automatic extraction

Ref. 62724. *EZ1 XL DSP Virus Kit*

#### Procedure to Serum and Plasma

Follow the instructions inside the kit *EZ1 XL DSP Virus Kit*.

Volume of sample to be used:

Serum [µl]	Plasma [µl]
200	200

#### Preparation of the Carrier and the Internal Control

Solve completely the lyophilized RNA carrier in elution buffer (AVE), from 310 µl, split in aliquots and store to -20 ± 5°C. Not thawing and freezing the aliquots more than 2 times.

For each analyzed sample, dilute 3,6 µl of a original solution include the RNA Carrier and 10 µl of *Internal Control* in total volume of 60 µl using elution buffer (AVE)

Select the protocol starting from 200 µl of samples with the elution of 60 µl.

Samples are now ready for amplification or storage at -80°C

#### Automatic Extraction

Ref.937055. *QIASymphony® DSP Virus/Pathogen midi kit*.

#### Procedure to Serum and Plasma

Follow the instructions inside the kit *QIASymphony DSP Virus/Pathogen Midi*. Use this volume of sample with 2 ml Microtube PP Skirted:

Serum [µl] (including excess volume)	Serum [µl] (including excess volume)
1200	1200

Select the protocol **Cellfree1000\_V7\_DSP\_default\_IC**.

#### Preselected elution volume

Preselected elution volume (µl)	Initial elution volume (µl)
60	90

#### Preparation of the Carrier

To prepare a carrier RNA (CARRIER) stock solution, add 1350 µl Buffer AVE (AVE) (provided in 2µl vials) to the tube containing 1350 µg lyophilized carrier RNA (CARRIER) to obtain a solution of 1 µg/µl. Dissolve the carrier RNA (CARRIER) divide it into conveniently sized aliquots, and store at 2-8°C for up to 2 weeks.

### Addition of the internal control to the samples

For each Serum or Plasma samples, 10µl internal controls must be added with carrier RNA (CARRIER)–Buffer AVE (AVE) mixture, and the total volume of the internal control–carrier RNA (CARRIER)–Buffer AVE (AVE) mixture remains 120 µl.

elution volume (µl)	RNA CARRIER (µl)	internal control (µl)	Buffer AVE (µl)	Final volume per sample (µl)
60	4.5	10	105,5	120

The internal control will be added to the sample automatically by the instrument.

### SOFTWARE SETTING

#### Lifetechnologies 7500 fast

Turn the instrument and the computer on and open the control software. Click on "**Advance Setup**": by default the software will show the page "**experiment properties**". Write in the "**experiment name**" the file name, choose the type of instrument (**7500 o 7500fast**), the type of reaction (**quantitation standard curve**), the type of used reagent (**Taqman®Reagents**) and the reaction time of analysis (**Standard ≈ 2 hours to complete a run**).

Open the page named "**page setup**" (sheet **Define Target and Samples**).

In the window **Define Targets** set:

Target mix DENV 1-2	Reporter	Quencher
<b>DENV-1</b>	FAM	MGBNFQ
<b>DENV-2</b>	VIC	MGBNFQ
<b>Internal Control (IC)</b>	CY5	None

Target mix DENV 3-4	Reporter	Quencher
<b>DENV-3</b>	FAM	MGBNFQ
<b>DENV-4</b>	VIC	MGBNFQ
<b>Internal Control (IC)</b>	CY5	None

Set the samples' name in the window "**Define Samples**".

In the same page "**plate setup**" select the sheet "**Assign Target and Samples**". On the screen you will see the microplate draft.

Select an area of the plate where the DENV1-2 and DENV3-4 controls will be placed: select wells of the plate and set targets (DENV-1, DENV-2 and IC for the first mix – DENV-3, DENV-4 and IC for the second mix). Select "**Assign target to selected wells**" in the blank, the "**Positive control**" for the targets.

Choose an area in the plate where negative control will be placed for each mix: select "**Assign target to selected wells**" in the blank, the "**task Negative (N)**" for the targets.

Select an area of the plate where samples will be placed in each mix: select the wells and set targets (DENV-1, DENV-2 and IC for the first mix – DENV-3, DENV-4 and IC for the second mix). Link every well to a sample, through the window "**Assign samples to selected wells**".

For each sample, select in the blank "**Assign targets to selected wells**" the "**task UnKnown (U)**" for the targets.

Set ROX as passive reference, using it as normalizer of detecting fluorescence.

Open "**Run Method**" (sheet **Graphic View**) and set the right thermal cycling with the "collect data" in annealing phase.

<u>cycles</u>	<u>denaturation</u>	<u>annealing</u>	<u>extension</u>
1	50° C 40 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

In the window "**Reaction volume plate per well**" set a volume of 30 µl.

After preparing the plate, and correctly inserting it in the instrument, press the button "**Start Run**".

#### Rotor Gene Q MDx

The experiments can be **set using the Quick Start Wizard or the Advanced Wizard**, which appears when the software is started.

Select the wizard "**Advanced**". As a first step, select the model "**Two Step Reaction**" with a double click in the "**New Run**".

In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "**Locking Ring Attached**", check the checkbox and then click "**Next**".

Enter the name of the operator and the reaction volume of 30 µl, and then click "**Next**".

In the next window click on "**edit profile**". Set the following thermal cycle:

<u>cycles</u>	<u>denaturation</u>	<u>annealing</u>	<u>extension</u>
1	50° C 40 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

Select the annealing from the thermal profile and click on "**Acquiring A to cycling**".

In the next window, select yellow and Red from the available channels and add it to acquiring channel along with the green channel and click "OK". In the next window click on "OK" and then click "Next".

Click on "Edit Gain" button and set the following values for each channel:

Target	Reporter	Gain
DENV-1 e DENV-3	Green	5
DENV-2 e DENV-4	Yellow	5
Internal Control (IC)	Red	5

To begin the course, click on the button "Start Run". You can save the model before you begin your run by clicking on "Save Template".

After clicking on the button "Start Run" window appears "Save As". The stroke can be saved in the desired position by the user.

Once the run started, the window "Edit Samples" allows you to set the name of samples and controls in the positions in which they were loaded on the instrument.

Select the locations where they were positioned the controls DENV1-2 for the first mix and DENV3-4 for the second mix designate them as DENV1-2 Positive and DENV3-4 Positive.

Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Positive Control".

Select the location where you placed the Negative Control for each mix and name it as Negative Control. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Negative Controls".

Select the location of each sample for each mix and enter the name or code of the patient. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Unknown".

At the end of the operation click "OK" in the "edit samples" and wait until the end of the race for the analysis (see "Interpretation of Results").

#### PREPARATION OF THE REACTIONS:

Thaw a tube of **Enzyme mMix**;

Thaw a tube of **DENV1-2 probes Mix**;

Thaw a tube of **DENV3-4 probes Mix**;

For a total number of tests other than 16, prepare a mixture for n + 1 tests, following the volumes shown in the table:

Reagent	Description	Volume for single test	Reagent	Description	Volume for single test
R1 -	<b>Enzyme mMix</b>	0,5	R1 -	<b>Enzyme mMix</b>	0,5
R2	<b>DENV1-2 probes</b>	14,5	R2	<b>DENV3-4 probes</b>	14,5

Mix carefully by vortex **9µl** of **Enzyme mMix** and **260µl** of **DENV1-2 probes Mix** (the mix as produced is enough to prepare **16 reactions** of amplification: **1 positive controls, 1 negative control and 14 samples**).

Mix carefully by vortex **9µl** of **Enzyme mMix** and **260µl** of **DENV3-4 probes Mix** (the mix as produced is enough to prepare **16 reactions** of amplification: **1 positive controls, 1 negative control and 14 samples**).

Distribute, in the amplification plate, **15µl of just reconstituted mix** in chosen positions as already set on the instrument software for each mix.

Distribute, in the negative control position, **15 µl** of solution taken by the negative control vial for each mix.

Distribute, in chosen position for each sample, **15 µl** of corresponding sample for each mix.

Distribute, in chosen positions for the positive controls, **15µl of DENV1-2 positive control and DENV3-4 positive control** in the respective mix.

Seal up accurately the plate using an optical adhesive film and verify that there aren't air bubbles in the mix, to avoid the amplification interferences.

For the Rotor-Gene Q MDx, seal each tube with the appropriate cap. The air bubbles presence is not influent; the rotor centrifugal force will allow automatic deletion.

Transfer the plate in the instrument and push the button "Start Run".

#### QUANTITATIVE ANALYSIS

##### Lifetechnologies 7500 Fast

At the end of the PCR run, the software automatically opens the "Analysis" window in the "Amplification plot" sheet on the menu on the left.

Select the wells corresponding to the positive control, negative control and samples for analysis.

Select in the "Option" window inside the "Target" pop-up menu the **DENV-1 target**. Check the correct setting of the threshold. Select in the "Option" window inside the "Target" pop-up menu the **DENV-2 target**. Check the correct setting of the threshold. Select in the "Option" window inside the "Target" pop-up menu the **DENV-3 target**. Check the correct setting of the threshold. Select in the "Option" window inside the "Target" pop-up menu the **DENV-4 target**. Check the correct setting of the threshold. Select in the "Option" window inside the "Target" pop-up menu the **IC Control target**. Check the correct setting of the threshold. The analysis of the results is made selecting from the menu in the left the page "Analysis". From the page "Amplification Plot" verify the amplification plot for every single sample. Opening the sheet "view well table" in the right side of the software it is possible to verify the data obtained from experiments: Threshold Cycles and emitted fluorescence. Clicking from the menu file and selecting the box export, the window "export properties" will open. Indicate the file name, select the position to save it (**Browse**) and click on button "Start export". In this way the software will permit to save a excel file with all the data corresponding to selected experiment.

## **Rotor Gene-Q MDx**

At the end of the PCR run open the "Analysis" window. Select the "Quantification" sheet and click on "cycling A (green)". Select from the menu "Dynamic Tube" and subsequently "Slope correct".

Eliminate the first 10 amplification cycles from the analysis by selecting the "Ignore First" button and setting the value 10. Check the correct setting of the threshold in the space provided "CT calculation – Threshold" and delete the first 10 cycles from the interpolation of the Threshold by entering the value 10 in the "Eliminate Cycles Before" box.

Open the "Analysis" window. Select the "Quantification" sheet and click on "cycling A (yellow)".

Select from the menu "Dynamic Tube" and subsequently "Slope correct". Eliminate the first 10 amplification cycles from the analysis by selecting the "Ignore First" button and setting the value 10. Check the correct setting of the threshold in the space provided "CT calculation – Threshold" and delete the first 10 cycles from the interpolation of the Threshold by entering the value 10 in the "Eliminate Cycles Before" box. Check the presence of a value of to CT in the table "Quant.Results". Open the "Analysis" window. Select the "Quantification" sheet and click on "cycling A (Red)". Select from the menu "Dynamic Tube" and subsequently "Slope correct". Eliminate the first 10 amplification cycles from the analysis by selecting the "Ignore First" button and setting the value 10. Check the correct setting of the threshold in the space provided "CT calculation – Threshold" and delete the first 10 cycles from the interpolation of the Threshold by entering the value 10 in the "Eliminate Cycles Before" box. Also in this case, you can print a report of the analysis by clicking on the "Report" window and selecting the file in the first **Quantification cycling A (green)** and then the file **cycling A (yellow)**.

## **INTERPRETATION OF RESULTS**

In the amplification reactions of each sample, the Ct values of each specific probe for Denv1, Denv2, Denv3 and Denv4 are used to detect the presence of the Target in analysis.

The instrument software is able to analyze the fluorescence emitted by specific probes and the fluorescence increase of the specific probe for Denv1 (FAM) or Denv2 (VIC) and for Denv3 (FAM) or Denv4 (VIC) indicate a positive sample for the target in question. Correct functioning of the amplification mixture can be verified by analyzing the following parameters:

Parameter	Reference
Denv1-2 positive Control	Ct ≤ 25
Denv3-4 positive Control	Ct ≤ 25

If the results of the amplification reaction of the positive controls produce a Ct > 25 or undetermined, the session cannot be considered valid and must be repeated.

As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data and the other laboratory tests done on the patient. As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

The use of positive and negative controls in each amplification session allow to verify the correct functioning of the amplification mix and the absence of any contamination. In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from extraction process until detection step. In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from reverse transcription process until detection step. Be sure that emitted fluorescence from internal control amplification has not a Ct > 35 or undetermined. If a sample shows an undetermined DENV RNA and an internal control Ct > 35, this means that there have been problems in the extraction stage or in the retro-amplification stage; therefore the sample could be a false negative.

### **Repeat the sample.**

It can be considered valid the samples with a Ct > 35 for the internal control, and a high concentration of DENV RNA. In this case, the competitive nature of PCR reaction can hide or disadvantage the internal control amplification.

Detector FAM or VIC	Detector VIC	PCR Run	Sample
Ct undetermined	Ct > 35 or undetermined	Not Valid	repeat
Ct undetermined	Ct < 35	Valid	Negative
High Ct	Ct < 35	Valid	Positive
Low Ct	Ct > 35 or undetermined	Valid	HighPositive

## **PERFORMANCES**

### **Analytical sensitivity:**

It is considered as analytical sensitivity the highest dilution (title) to which a positive sample can be diluted without the system losing the ability to detect with a positivity rate of ≥ 95%. The analytical sensitivity of the system was assessed by analyzing synthetic RNA, quantified by spectrophotometric analysis, containing the regions of interest (anchored capsid protein) of the virus in serial dilutions.

Probit Analysis	Copies/μl	Manual Extraction cps/ml	EZ1 Automatic Extraction cps/ml	QIASymphony Automatic Extraction cps/ml
Denv-1	2,28 cps/μl	813 cps/ml	1026 cps/ml	205 cps/ml
Denv-2	0,63 cps/μl	225 cps/ml	283 cps/ml	57 cps/ml
Denv-3	0,76 cps/μl	271 cps/ml	342 cps/ml	68 cps/ml
Denv-4	1,05 cps/μl	374 cps/ml	472 cps/ml	94 cps/ml

### **Clinical sensitivity:**

It is considered as clinical sensitivity the ability to detect true positive samples in the totality of the samples screened as positive. The analysis was made on DENV positive samples and the test was performed following the method recommendations. Positive samples were confirmed with another CE approved method available on the market.

Obtained results show a clinical sensitivity of 100%.

### Linearity/Proportionality

The system linearity was valued analyzing synthetic RNA (Denv1-2 RNA and Denv3-4 RNA), quantified by spectrophotometric analysis, containing the regions of interest anchored capsid protein of the virus in serial dilutions (1:10) from 1.500.000.000 copies/reactions to 15 copies/reaction of RNA in 15µl of extracted material added in the retro-amplification reaction.

The evaluation was performed analyzing 10 calibration curves, that showed these parameters:

Positive Control Denv-1	Ct ≤ 25	Average Ct 24,24
Correlation Coefficient	$0,990 \leq r^2 \leq 1$	Average $r^2$ 0,999
Slope	$-3,6 \leq \text{Slope} \leq -3,2$	Average slope -3,51
PCR efficiency	$90 \leq \text{Efficiency} \leq 100$	Average Eff 93%

Positive Control Denv-2	Ct ≤ 25	Average Ct 22,94
Correlation Coefficient	$0,990 \leq r^2 \leq 1$	Average $r^2$ 0,999
Slope	$-3,6 \leq \text{Slope} \leq -3,2$	Average slope -3,3
PCR efficiency	$90 \leq \text{Efficiency} \leq 100$	Average Eff 101%

Positive Control Denv-3	Ct ≤ 25	Average Ct 21,41
Correlation Coefficient	$0,990 \leq r^2 \leq 1$	Average $r^2$ 0,999
Slope	$-3,6 \leq \text{Slope} \leq -3,2$	Average slope -3,34
PCR efficiency	$90 \leq \text{Efficiency} \leq 100$	Average Eff 99,15%

Positive Control Denv-4	Ct ≤ 25	Average Ct 22,35
Correlation Coefficient	$0,990 \leq r^2 \leq 1$	Average $r^2$ 1
Slope	$-3,6 \leq \text{Slope} \leq -3,2$	Average slope -3,29
PCR efficiency	$90 \leq \text{Efficiency} \leq 100$	Average Eff 101%

### Reproducibility and Repeatability

The reproducibility and repeatability of the system were valued analyzing 3 dilutions of synthetic RNA containing the anchored capsid protein region of interest for DENV and quantified by spectrophotometric analysis, plus a negative control (negative RNA).

For each session, 5 replicates were made in 3 different sessions, performed by different technicians, using 3 different lots of product.

Hypothetical value	Lot	N° repetitions	Average Reveal. Conc.	Inaccuracy %
100.000 Denv-1	L1-L2-L3	15	97908	2%
100.000 Denv-2	L1-L2-L3	15	94077	6%
100.000 Denv-3	L1-L2-L3	15	118626	19%
100.000 Denv-4	L1-L2-L3	15	107401	7%
1.000 Denv-1	L1-L2-L3	15	1058	6%
1.000 Denv-2	L1-L2-L3	15	843	16%
1.000 Denv-3	L1-L2-L3	15	1034	3%
1.000 Denv-4	L1-L2-L3	15	918	8%
10 Denv-1	L1-L2-L3	15	7,9	21%
10 Denv-2	L1-L2-L3	15	6,8	22%
10 Denv-3	L1-L2-L3	15	10,4	4%
10 Denv-4	L1-L2-L3	15	10,5	5%

The average Inaccuracy percentage is 10%.

### Diagnostic Specificity:

It is considered as diagnostic specificity the ability of the method to detect trues negative samples. The diagnostic specificity of the system was evaluated analyzing human samples tested and confirmed as DENV negative with an other CE marked method available on the market.

Diagnostic specificity is 100%.

### Analytical Specificity:

Test's specificity was guaranteed by the use of specific primers for Dengue Virus.

The alignment of the choose regions for specific primers' hybridization for DENV with available sequences of the anchored capsid protein region present in database, demonstrated: their conservation, the absence of significative mutations for the analysed target.

The analysis "in Silico " showed how the alignment of primers and probes allows the correct identification of the 4 serotype.

**Cross-Reactivity:**

The exam of alignment of the region chosen for hybridization of primers specific for Dengue Virus with the sequences available in the bank data of the anchored capsid protein region showed the their conservation, the absence of significant mutations and the complete specificity for the targets analyzed.

To check the cross-reactivity of the assay, samples tested as positive for other viruses were analysed following the method instructions.

Sample	Positive sample	Obtained Results cps/ml
1	ZIKA	< 100
2	WNV	< 100
3	Yellow Fever Virus	< 100
4	Chikungunya Virus	< 100
5	Usutu Virus	< 100

**INTERFERENCES:**

Verify that in the RNA extracted from the sample there is no contamination from mucoproteins and haemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorbtion of Nucleic Acids) and 280 nm (maximum absorbtion of Proteins). A pure RNA should have a ratio of approximately 2.

**QUALITY CONTROL**

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

In accordance with the Clonit srl ISO EN 13485 Certified quality Management System, each lot of **Dengue Serotyping** is tested against predetermined specification to ensure consistent product quality.

**BIBLIOGRAPHY**

*Single-Reaction Multiplex Reverse Transcription PCR for Dection of Zika, Chikungunya, and Dengue Viruses.*

Jesse J. Waggoner, Lionel Gresh, Alisha Mohamed-Hadley, Gabriela Ballesteros, Maria Jose Vargas Davila, Yolanda Tellez, Malaya K. Sahoo, Angel Balmaseda, Eva Harris, Benjamin A. Pinsky  
Emerging Infectious Diseases [www.cdc.gov/eid](http://www.cdc.gov/eid) Vol. 22, No. 7, July 2016

*Development of an Internally Controlled Real-Time Reverse Transcriptase PCR Assay for Pan-Dengue Virus Detection and Comparison of Four Molecular Dengue Virus Detection Assays*

Jesse J. Waggoner, Janaki Abeynayake, Malaya K. Sahoo, Lionel Gresh, Yolanda Tellez, Karla Gonzalez, Gabriela Ballesteros, Angel Balmaseda, Kumudu Karunaratne, Eva Harris, Benjamin A. Pinsky *Journal of Clinical Microbiology* p. 2172–2181 July 2013 Volume 51 Number 7

**TECHNICAL ASSISTANCE**

For any question and support please contact our Technical support:

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	Quantità	Descrizione
R1	3 x 20 µl	<b>Enzyme mMix</b> Reverse Transcriptase e Taq Polymerase enzymes (Tappo Rosa)
R2	3 x 270 µl	<b>DENV1-2 probe Mix</b> Upstream primer, downstream primer, Target probe DENV-1 (FAM), Target probe DENV-2 (VIC), Internal Control (IC) probe (CY5), Nuclease-free water, ROX, dNTPs, Tris-HCl, KCl, MgCl <sub>2</sub> (Tappo Bianco)
R3	3 x 270 µl	<b>DENV3-4 probe Mix</b> Upstream primer, downstream primer, Target probe DENV-3 (FAM), Target probe DENV-4 (VIC), Internal Control (IC) probe (CY5), Nuclease-free water, ROX, dNTPs, Tris-HCl, KCl, MgCl <sub>2</sub> (Tappo Blu)
R4	3 x 35 µl	<b>Denv1-2 Positive Control</b> RNA sintetico corrispondente alla regione anchored capsid protein (Tappo Verde)
R5	3 x 35 µl	<b>Denv3-4 Positive Control</b> RNA sintetico corrispondente alla regione anchored capsid protein (Tappo Giallo)
R6	3 x 160 µl	<b>Internal control</b> Fago MS2 (Tappo Rosso)
R7	1 x 100 µl	<b>Controllo negativo</b>

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### **BIBLIOGRAFIA**

- *Single-Reaction Multiplex Reverse Transcription PCR for Detection of Zika, Chikungunya, and Dengue Viruses.* Jesse J. Waggoner, Lionel Gresh, Alisha Mohamed-Hadley, Gabriela Ballesteros, Maria Jose Vargas Davila, Yolanda Tellez, Malaya K. Sahoo, Angel Balmaseda, Eva Harris, Benjamin A. Pinsky *Emerging Infectious Diseases* www.cdc.gov/eid Vol. 22, No. 7, July 2016
- *Development of an Internally Controlled Real-Time Reverse Transcriptase PCR Assay for Pan-Dengue Virus Detection and Comparison of Four Molecular Dengue Virus Detection Assays* Jesse J. Waggoner, Janaki Abeynayake, Malaya K. Sahoo, Lionel Gresh, Yolanda Tellez, Karla Gonzalez, Gabriela Ballesteros, Angel Balmaseda, Kumudu Karunaratne, Eva Harris, Benjamin A. Pinsky *Journal of Clinical Microbiology* p. 2172-2181 July 2013 Volume 51 Number 7

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**CLONIT S.r.l.**