SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

Oncohematology Panel

A new line of one-step
qRT-PCR kits, by Clonit

In patients with Acute Myeloid Leukemia (AML), qRT-PCR shows significant sensitivity in the detection of the translocations or chromosomal abnormalities. It represents an important tool in the screening of acute leukemia providing useful information for the diagnosis and prognosis and allowing patients to be assigned to the appropriate risk group for management and therapy.

PML-RARA t(15;17)(q22;q21)

In vitro diagnostic test for the identification and quantification of the PML-RARA fusion genes resulting from the t(15;17)(q22;q21), by multiplex one-step Real-Time PCR.

One-tube mix for PML-RARA BCR1, PML-RARA BCR2, PML-RARA BCR3 fusion transcripts and the ABL gene IC.

AML1-ETO t(8;21)(q22;q22)

In vitro diagnostic test for the identification and quantification of the AML1-ETO fusion gene transcript resulting from t(8;21)(q22;q22), by one-step Real-Time PCR.

One-tube mix for AML1-ETO fusion transcript and the ABL gene IC.

Inv(16)(p13q22) - CBFB-MYH11

In vitro diagnostic test for the detection and quantification of the CBFB/MYH11 fusion gene resulting from the Inv(16)(p13q22), by multiplex one-step Real-Time PCR.

Mix 1: Inv(16) type A, type D, type E variants and the ABL gene IC.

Mix 2 (rare variants): Inv(16) type B, C, F, G, H I, J variants and the ABL gene IC.

Thanks to the Standard set, it is possible the calculation of the ratio of specific fusion transcript signal to endogenous control gene signal in each sample. This allows the identification of the **Minimal Residual Disease (MRD)**.

Clonit's Oncohematology panel has been designed with striking and innovative features to ensure a fast, safe and effective diagnosis in patients with Acute Myeloid Leukemia:

- The PML-RARA t(15;17)(q22;q21) allows the identification of all BCR1, BCR2, BCR3 transcripts by a one-mix reaction
- The Inv(16)(p13q22) - CBFB-MYH11 can detect all the 10 variants of the CBFB-MYH11 transcript described in the literature
- The Standard set for quantification is always included into the kits

Oncohematology Panel

The CE IVD kits are compliant to the
"Europe Against Cancer (EAC) Guidelines -
Leukemia 2003"

General information:

- **Protocol:** one-step Real-Time RT-PCR
- **Time to result:** 1h and 30 minutes
- **Quantification Standards:** synthetic sequences corresponding to target fusion region and ABL
4 points (10^2 cps/ μ L – 10^5 cps/ μ L). The standard sets are included into the kits.
- **Starting samples:** peripheral blood or bone marrow
- **Validated PCR Real time Systems:**
 - Applied Biosystems 7500 Fast (ThermoFisher SCIENTIFIC)
 - Rotor-Gene Q MDx (RG-Q MDx - QIAGEN)
 - CFX96 Real-Time PCR Detection System (Bio-Rad)
- **RNA Manual extraction:** RNeasy Mini Kit (QIAGEN)
- **RNA Automatic extraction:** CloNext 12 or CloNext 24
- **Package:** 24 tests
- **Storage:** -20°C

Information for orders:

Description	Ref.	Regulatory	Tests
PML-RARA t(15;17)(q22;q21)	RT-100	CE-IVD	24 tests
AML1-ETO t(8;21)(q22;q22)	RT-101	CE-IVD	24 tests
Inv(16)(p13q22) - CBFβ-MYH11	RT-102	CE-IVD	24 tests

Contatti:

Clonit srl Via Lombardia, 6 – 27010 Sizzano (PV)
Tel. +39 02 56814413| Fax +39 02 56814515
www.clonit.it | customercare@clonit.it

Brochure_Oncohematology_20220713

Inv(16)(p13q22) - CBFB/MYH11

Detection of the fusion gene transcripts Inv(16)(p13q22) - CBFB/MYH11 by Real Time PCR

REV ST.RT-102-ENITA.0

REF RT-102 - 24 tests

Instructions For Use

INTRODUCTION AND PURPOSE OF USE

The **Inv(16)(p13q22) - CBFB/MYH11** is a quantitative assay based on nucleic acids amplification for the identification and the quantification of the CBFB/MYH11 gene fusion results from the Inv(16)(p13q22) in samples of RNA extracted from Mononuclear cells isolated from whole blood collected in EDTA or Bone Marrow collected in EDTA.

The Inv(16)(p13q22) - CBFB/MYH11 allow also the detection and quantification of ABL as a endogenous control gene.

Standard curves of known amounts of both the endogenous control and the gene fusion cDNA allow the calculation of the ratio of specific fusion transcript signal to endogenous control gene signal in each sample, allowing the identification of the minimal residual disease (MRD). The test is not intended to assess the suitability of blood, blood components, cells, tissues or organs for transfusion, transplantation or cell administration.

The device is intended for detecting the presence of gene fusion results from the t **Inv(16)(p13q22)**, a recurrent genetic abnormality especially in adult Acute Myeloid Leukemia (10% of all de novo AML). This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.

CONTENT

The kit contains sufficient reagents to perform 24 amplification tests.

Quantity	Description
R1 2 x 75 µl	Amplification mMix dNTPs, Tris-HCl, KCl, MgCl ₂ , thermostable MMLV reverse transcriptase, Taq Polymerase, AmpErase Uracil N-Glycosylase (UNG) Nuclease-free water, ROX (Pink Cap)
R2 2 x 70 µl	ADE probes Mix Upstream primers, downstream primers, Target probe Inv(16) type A, D and E (FAM), Target probe Control Gene (ABL) (TexasRed), Nuclease-free water (White Cap)
R3 2 x 70 µl	RARE probes Mix Upstream primers, downstream primers, Target probe Inv(16) type B, C, F, G, H I and J (FAM), Target probe Control Gene (ABL) (TexasRed), Nuclease-free water (Yellow Cap)
R4 2 x 50 µl	Inv(A)/ABL (100.000 cps/µl) Control standard dilution 1 Synthetic Nucleic Acids corresponding to gene fusion region derived from Inv(16) type A and ABL as control gene at the concentration of 10 ⁵ copies/µl (Blu Cap)
R5 2 x 50 µl	Inv(A)/ABL (10.000 cps/µl) Control standard dilution 2 Synthetic Nucleic Acids corresponding to gene fusion region derived from Inv(16) type A and ABL as control gene at the concentration of 10 ⁴ copies/µl (Green Cap)
R6 2 x 50 µl	Inv(A)/ABL (1.000 cps/µl) Control standard dilution 3 Synthetic Nucleic Acids corresponding to gene fusion region derived from Inv(16) type A and ABL as control gene at the concentration of 10 ³ copies/µl (Yellow Cap)
R7 2 x 50 µl	Inv(A)/ABL (100 cps/µl) Control standard dilution 4 Synthetic Nucleic Acids corresponding to gene fusion region derived from Inv(16) type A and ABL as control gene at the concentration of 10 ² copies/µl (White Cap)
R8 2 x 50 µl	RARE Positive Control Synthetic Nucleic Acids corresponding to gene fusion region derived from Inv(16) type F and ABL as control gene at the concentration of 10 ³ copies/µl (Red Cap)
R9 2 x 50 µl	Negative Control (Natural Cap)

MATERIALS AND INSTRUMENTATION REQUIRED BUT NOT SUPPLIED

- Disposable latex powder-free gloves or similar material;
- Bench microcentrifuge (12,000 - 14,000 rpm);
- Micropipettes and Sterile tips with aerosol filter;
- Vortex;
- Plastic materials (microplate and optical adhesive cover);
- β-mercaptoethanol (β-ME)
- Red Blood Cell Lysis Buffer (Ex.: Ref 11 814 389 001- Roche)

Reagents

The **Inv(16)(p13q22) - CBFB/MYH11** kit was developed and validated to be used with the following extraction methods:

Manual Extraction

Ref. 74104

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The kit allows the RNA extraction from tested samples. The kit contains reagents for 50 (QIAGEN).

Automatic Extraction

Ref. OP02015. CloNext Total RNA Extraction Kit

the kit allows the automatic extraction of Total RNA from Human samples. The kit contains reagents for 48 samples. (CLONIT)

Alternative nucleic acid extraction systems and kits might also be appropriate following the manufacturers instructions. The suitability of the nucleic acid extraction procedure for use with Inv(16)(p13q22) - CBFB/MYH11 has to be validated by the user.

Instruments

The Inv(16)(p13q22) - CFBF/MYH11 kit was developed and validated to be used with the following instruments:

Extraction System

- CloNext 12 or CloNext 24 (ref. OP01018/OP01019). Robotic Workstation for the automatic purification of the nucleic acids.

Real Time PCR

- 7500 Fast from Lifetechnologies
- Rotor-Gene Q MDx from QIAGEN
- CFX96 Real Time PCR System from Bio-Rad

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturers' instructions and recommendations.

SAMPLES

The **Inv(16)(p13q22) - CFBF/MYH11** must be used with extracted RNA from the following biological samples: **Mononuclear cells isolated from whole blood collected in EDTA or Bone Marrow collected in EDTA**. Collected samples must be transported and stored at + 2- + 8 ° C and used within 3 days from the date of collection. Do not freeze peripheral blood to avoid RNA degradation. The extracted RNA can be stored at -80 ° C for up to six months.

PRECAUTIONS FOR USE

- This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.
- Carefully read this Instruction For Use before using the kit.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: RNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- Periodically wipe the working area with 0,5% hypochlorite.
- Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.
- Use powder-free gloves. Do not leave fingerprints on optical caps. Do not write on caps as this may cause an interference with fluorescent detection.
- Avoid any contact between hands and eyes or nose during specimen collection and testing.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood. Waste should be discarded according to local law.
- Never pipette solutions by mouth.
- Use calibrated and regularly checked pipettes and instrumentation only.
- Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.
- Wash hands carefully after handling samples and reagents.
- Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
- Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

LIMITS OF THE METHOD

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and/or controls. Therefore, you should:

- physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products
- use tips with filters to prevent cross-contamination between samples
- use disposable gloves and change them frequently
- carefully open test tubes to prevent aerosol formation
- close every test tube before opening another one

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

Patient Drug treatment may interfere with the final result of the molecular biology analysis.

The proper functioning of the amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

A negative result obtained with this product suggests that the RNA of Inv(16)(p13q22) - CFBF/MYH11 was not detected in RNA extracted from the sample, but it may also contain Inv(16)(p13q22) - CFBF/MYH11 RNA at a lower titre than the detection limit for the product (detection limit for the product, see paragraph on Performance Characteristics); in this case the result would be a false negative.

WARNINGS

- Read carefully the instructions for use before using this test.
- Use only extracted RNA from Mononuclear cells isolated from whole blood collected in EDTA or Bone Marrow collected in EDTA. The RNA extracted from the starting sample must not contain heparin, hemoglobin, Ficoll®, ethanol or propan-2-ol, to avoid PCR inhibition phenomena and invalid results. Amounts of extracted RNA greater than 1.5 µg per reaction can inhibit the real time RT-PCR. Amounts of human genomic DNA exceeding 100 ng per reaction present in the RNA extracted from sample can interfere or inhibit the the real time RT-PCR
- Do not mix reagents from different lots.
- Thaw and carefully mix the reagents of the kit before use.
- The PCR mix has to be freshly prepared every time.
- After reconstitution, the amplification master mix must be used in one time (12 reactions). Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.
- Do not use beyond the expiration date which appears on the package label. Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- In case of damaged package, contact the technical support before using the kit. In case of any serious incident that has occurred in relation to the device, a notice shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

STORAGE AND STABILITY

Store the product **Inv(16)(p13q22) - CFBF/MYH11** at -20°C.

The **Inv(16)(p13q22) - CFBF/MYH11** kit is shipped on dry ice. The kit components should be frozen.

If one or more components are not frozen upon receipt or if the tubes have been compromised during transport, contact Clonit for assistance. An intact and well stored product has a stability of 6 months from the date of production. Do not use beyond the expiration date which appears on the package label.

Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently

ANALYTICAL PROCEDURE

Leukocytes isolation

It is advisable to perform the leukocytes isolation according to the laboratory current protocol or with procedure described in the extraction systems reported below.

Manual Extraction

Ref. 74104 - RNeasy Mini Kit (QIAGEN).

Follow the instructions inside the RNeasy Mini Kit - Procedure for Total RNA from Animal Cells

Elute the sample in 100µl.

Samples are now ready for amplification or storage at -80°C.

Automatic Extraction

Ref. OP02015. CloNext Total RNA Extraction Kit

Follow the instructions provided with the kit.

Elute the sample in a final volume of 50µl.

Samples are now ready for amplification or storage at -80°C.

SOFTWARE SETTING

Lifetechnologies 7500 fast

Turn the instrument and the computer on and open the control software. Click on "**Advance Setup**": by default the software will show the page "**experiment properties**". Write in the "**experiment name**" the file name, choose the type of instrument (**7500** or **7500fast**), the type of reaction (**quantitation standard curve**), the type of used reagent (**Taqman®Reagents**) and the reaction time of analysis (**Standard** ≈ 2 hours to complete a run).

Open the page named "**page setup**" (sheet **Define Target and Samples**).

In the window, **Define Targets** set:

Target Mix ADE	Reporter	Quencher
Inv(16) ADE	FAM	NONE
ABL	TexasRed	NONE

Target mix RARE	Reporter	Quencher
Inv(16) RARE	FAM	NONE
ABL	TexasRed	NONE

Mix ADE

Set the samples name in the window "**Define Samples**".

In the same page "**plate setup**" select the sheet "**Assign Target and Samples**". On the screen you will see the microplate map.

Select an area of the plate where the controls will be placed: select the wells of the plate and set both targets (Inv(16) ADE and ABL). In the "**Assign target to selected wells**" field select the "**task Standard (S)**" for Inv(16) ADE and ABL target and set the controls concentration.

Choose an area in the plate where negative control will be placed: select "**Assign target to selected wells**" in the blank, the "**task Negative (N)**" for the Inv(16) ADE and ABL target.

Select an area of the plate where samples will be placed: select the wells and set both targets (Inv(16) ADE and ABL). Link every well to a sample, through the window "**Assign samples to selected wells**".

For each sample, select in the field "**Assign targets to selected wells**" the "**task UnKnown (U)**" for the Inv(16) ADE and ABL target.

Set NONE as passive reference.

Mix RARE

Set the samples name in the window "**Define Samples**".

In the same page "**plate setup**" select the sheet "**Assign Target and Samples**". On the screen you will see the microplate map.

Select an area of the plate where the controls will be placed: select the wells of the plate and set both targets (Inv(16) RARE and ABL). In the "**Assign target to selected wells**" field select the "**task Standard (S)**" for Inv(16) RARE and ABL target.

Choose an area in the plate where negative control will be placed: select "**Assign target to selected wells**" in the blank, the "**task Negative (N)**" for the Inv(16) RARE and ABL target.

Select an area of the plate where samples will be placed: select the wells and set both targets (Inv(16) RARE and ABL). Link every well to a sample, through the window "**Assign samples to selected wells**".

For each sample, select in the field "**Assign targets to selected wells**" the "**task UnKnown (U)**" for the Inv(16) RARE and ABL target.

Set NONE as passive reference.

Open "**Run Method**" (sheet **Graphic View**) and set the correct thermal profile, with the "collect data" in annealing/extension phase:

cycles	UNG	Reverse transcription	Denaturation	Annealing
1	25°C 2 min	50° C 15 min		
1			95°C 2 min	
50			95° C 3 sec	63° C 45 sec

In the window "**Reaction volume plate per well**" set a volume of 20 µl.

After preparing the plate, and correctly inserting it in the instrument, press the button "**Start Run**".

Rotor Gene Q MDx

The experiments can be set using the **Quick Start Wizard** or the **Advanced Wizard**, which shows up when the software is started.

Select the wizard "**Advanced**". As a first step, select the model "**Two Step Reaction**" with a double click in the "**New Run**".

In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "**Locking Ring Attached**", check the checkbox and then click "**Next**".

Enter the name of the operator and the reaction volume of 20 µl, and then click "Next".

In the next window click on "**edit profile**". Set the following thermal cycle:

cycles	UNG	Reverse transcription	Denaturation	Annealing
1	25°C 2 min	50° C 15 min		
1			95°C 2 min	
50			95° C 3 sec	63° C 45 sec

Select the annealing/extension from the thermal profile and click on "**Acquiring A to cycling**".

In the next window, select Orange from the available channels and add it to **acquiring channel** along with the Green channel and click "**OK**".

Click on "Edit Gain" button and set the following values for each channel:

Reporter	Gain
Green	5
Orange	5

In the next window click on "OK" and then click "Next".

To start the run, click on the button "Start Run". You can save the model before you begin your run by clicking on "Save Template".

After clicking on the button "Start Run" the window "Save As" appears. The run file can be saved in the desired position by the user. Once the run is started, the window "Edit Samples" allows you to set the name of samples and controls in the positions in which they were loaded on the instrument.

Mix ADE

Select the locations where the controls of known concentration were positioned and designate them as Inv(A)/ABL Standard.

Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Standards". Enter the concentrations of the controls following the instructions indicated in the paragraph **Analysis of the results**.

Select the location where you placed the Negative Control and name it as Negative Control. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Negative Control".

Select the location of each sample and enter the name or code of the patient. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Unknown"

At the end of the operation click "OK" in the "edit samples" and wait until the end of the run for the analysis (see "Interpretation of Results")

Mix RARE

Select the locations where the Positive control is positioned and designate them as RARE Positive Control

Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Positive Control".

Select the location where you placed the Negative Control and name it as Negative Control. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Negative Control".

Select the location of each sample and enter the name or code of the patient. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Unknown"

At the end of the operation click "OK" in the "edit samples" and wait until the end of the run for the analysis (see "Interpretation of Results")

CFX96 Real Time PCR

Turn the instrument and the computer on and start the control software. In the principal screen will appear the window "Startup wizard": select "CFX96" and press "ok". In the next window push "create new" and set the thermal protocol, with the "collect data" in annealing/extension phase, and set the reaction volume (20µl):

cycles	UNG	Reverse trascription	Denaturation	Annealing
1	25°C 2 min	50° C 15 min		
1			95°C 2 min	
50			95° C 3 sec	63° C 45 sec

Mix ADE

Select the locations where the controls of known concentration were positioned and choose the "Sample Type" Standards.

Click "Load" check boxes to load fluorophores and Type or select Target Name. In the box "load concentration" set the concentrations of the 4 calibrators, following the instructions indicated in the paragraph **Analysis of the results**.

Select the location where you placed the Negative Control. Choose the "Sample Type" NTC. Click "Load" check boxes to load fluorophores and Type or select Target Name

Select the location of each sample and enter the name or code of the patient. Choose the "Sample Type" Unknown. Click "Load" check boxes to load fluorophores and Type or select Target Name

Mix RARE

Select the locations where the RARE Positive Control is positioned and choose the "Sample Type" Positive Control.

Click "Load" check boxes to load fluorophores and Type or select Target Name.

Select the location where you placed the Negative Control. Choose the "Sample Type" NTC. Click "Load" check boxes to load fluorophores and Type or select Target Name

Select the location of each sample and enter the name or code of the patient. Choose the "Sample Type" Unknown. Click "Load" check boxes to load fluorophores and Type or select Target Name

Save the plate clicking the next button and start the experiment.

PREPARATION OF THE REACTIONS:

Thaw a tube of **Amplification mMix**;

Thaw a tube of **ADE probes Mix**;

Thaw a tube of **RARE probes Mix**;

Add **70µl of Amplification Mmix** directly into the tube of ADE probes Mix containing 70µl. Mix carefully by vortexing.

Add **70µl of Amplification Mmix** directly into the tube of RARE probes Mix containing 70µl. Mix carefully by vortexing.

The mixture thus produced is sufficient to prepare 12 reactions: 4 positive control, 1 negative control and 7 samples.

For a total number of tests other than 12, prepare a mixture for n + 1 tests, following the volumes shown in the table:

Mix ADE		
Reagent	Description	Volume for single test (µl)
R1	Amplification mMix	5,0
R2	ADE probes Mix	5,0

Mix RARE		
Reagent	Description	Volume for single test (µl)
R1	Amplification mMix	5,0
R3	RARE probes Mix	5,0

Distribute, in the amplification plate, **10 μ l** of the reconstituted mix in chosen positions as already set on the instrument software for each mixes.

Distribute, in the negative control position for each mixes, **10 μ l** of solution taken by the negative control vial.

Distribute, in the chosen position for each sample for each mixes, **10 μ l** of corresponding sample.

Distribute, in the chosen positions for the positive controls of the mix ADE, **10 μ l** of **10² copies/ μ l** solution, **10³ copies/ μ l** solution, **10⁴ copies/ μ l** solution and **10⁵ copies/ μ l** solution.

Distribute, in the chosen positions for the positive controls of the mix RARE, **10 μ l** of RARE Positive control solution.

Seal up accurately the plate using an optical adhesive film and verify that there are not air bubbles in the mix, to avoid interferences in the detection.

For the Rotor-Gene Q MDx, seal each tube with the appropriate cap. The air bubbles presence is not relevant; due to the centrifugal force applied by the rotor.

Transfer the plate or tubes in the instrument and push the button "Start Run".

QUANTITATIVE ANALYSIS

Lifetechnologies 7500 Fast

At the end of the PCR run, the software automatically opens the "**Analysis**" window in the "**Amplification plot**" sheet on the menu on the left.

Select the wells corresponding to the positive controls, negative control and samples for analysis.

Select in the "**Option**" window inside the "**Target**" pop-up menu the **Inv(16) ADE target**. Check the correct setting of the threshold.

Select in the "**Option**" window inside the "**Target**" pop-up menu the **Inv(16) RARE target**. Check the correct setting of the threshold.

Select in the "**Option**" window inside the "**Target**" pop-up menu the **ABL Control target**. Check the correct setting of the threshold.

Mix ADE

The analysis of the results is made selecting from the menu in the left the page "**Analysis**". From the page "**Standard Curve**", maintaining open the sheet "**view well plate**" in the right side of the software select the wells containing the quantity standards and verify the parameters described in the paragraph "**Interpretation of the Results**" (coefficient of correlation, slope etc...) for Inv(16) ADE and ABL targets.

From the page "**Amplification Plot**" verify the amplification plot for every single sample.

Opening the sheet "**view well table**" in the right side of the software it is possible to verify the data obtained from the run: Threshold Cycles, emitted fluorescence and the target quantification expressed in copies/reaction or copies/ml, depending on the settings of the calibration curve.

Clicking on the **file** menu and selecting the box **export**, the window "**export properties**" will open. Indicate the file name, select the position to save it (**Browse**) and click on button "**Start export**". In this way the software will permit to save an excel file with all the data corresponding to the selected experiment.

Mix RARE

The analysis of the results is made selecting from the menu in the left the page "**Analysis**".

Select the wells corresponding to the positive control, negative control and samples for analysis. From the page "**Amplification Plot**" verify the amplification plot for every single sample.

Opening the sheet "**view well table**" in the right side of the software it is possible to verify the data obtained from the run: Threshold Cycles, emitted fluorescence.

Clicking on the **file** menu and selecting the box **export**, the window "**export properties**" will open. Indicate the file name, select the position to save it (**Browse**) and click on button "**Start export**". In this way the software will permit to save an excel file with all the data corresponding to the selected experiment.

Rotor Gene Q MDx

Mix ADE

At the end of the PCR run open the "Analysis" window. Select the "**Quantification**" sheet and click on "**cycling A.green**".

Select from the menu "**Dynamic Tube**" and subsequently "**Slope correct**". If necessary, reduce the background noise of non-specific signals by selecting the "**Outlier Removal**" option and entering a value between 5% and 30%.

Check the correct setting of the threshold in the space provided "**CT calculation – Threshold**".

Repeat the steps above for the fluorescence channels **cycling A (orange)**.

You can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification **cycling A.green**" and then the file cycling **cycling A.orange**".

Mix RARE

At the end of the PCR run open the "Analysis" window. Select the "**Quantification**" sheet and click on "**cycling A.green**".

Select from the menu "**Dynamic Tube**" and subsequently "**Slope correct**". If necessary, reduce the background noise of non-specific signals by selecting the "**Outlier Removal**" option and entering a value between 5% and 30%.

Check the correct setting of the threshold in the space provided "**CT calculation – Threshold**".

Repeat the steps above for the fluorescence channels **cycling A (orange)**.

You can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification **cycling A.green**" and then the file cycling **cycling A.orange**".

CFX96 Real Time PCR System

At the end of the PCR, select the "**quantitation**" sheet. On the top of the screen, select "**settings**" from the menu and choose "**Baseline Threshold...**".

You can export the report pushing the paper block figure on the top of the screen

INTERPRETATION OF RESULTS

Mix ADE

By means of the Real Time PCR reaction it is possible to obtain the RNA quantification of CFBF/MYH11 gene fusion results from the Inv(16)(p13q22) and ABL control gene in the original sample, by setting the values of the positive controls of the calibration curve. To calculate these values, all the dilution steps to which the sample has been subjected during the extraction and amplification setup stages must be considered.

The measuring range of the assay is between the first and last calibrators of the standard curve.

The Ct values obtained from the amplification of 4 controls of known titre are used by the software for the calculation of the calibration curve from which the unknown samples are interpolated.

A proper functioning of the amplification mix can be verified by analyzing these parameters:

Parameter	Reference Inv(16)	Reference ABL
RTS conc. 10 ⁵ copies/μl	Ct ≤ 25	Ct ≤ 25
Correlation coefficient	0.990 ≤ r ² ≤ 1	0.990 ≤ r ² ≤ 1
Slope	-3,6 ≤ Slope ≤ -3,1	-3,6 ≤ Slope ≤ -3,1
PCR Efficiency	90 ≤ Efficiency ≤ 110	90 ≤ Efficiency ≤ 110

If the RTS amplification reaction at a concentration of 10⁵ copies produces a Ct > 25 or undetermined the analytical session cannot be considered valid and must be repeated.

Verify that the correlation coefficient value (r²), the slope or the reaction efficiency fit to the limits indicated in the above table or do not deviate much from them, which represent the ideal range for a proper PCR reaction.

In the instrument software, following the instructions in the software settings paragraph, set for each standard the concentration value expressed in copies/reactions as indicated in the following table:

		Copies/reaction
	RTS 1	1.000.000
	RTS 2	100.000
	RTS 3	10.000
	RTS 4	1.000

The concentration of the sample, expressed in the same units, will be automatically obtained from the instrument software after the analysis. To convert this value in copies/ml, use the following formula:

$$copie / ml = \frac{1000}{Ve} \times \frac{Ev}{Ea} \times C_{reaz}$$

where:

- **Ve**: Starting Volume of Whole Blood or Bone marrow suspension used to isolate Mononuclear cells, expressed in μl
- **Ev**: eluted sample Volume during extraction, expressed in μl
- **Ea**: extracted sample volume used for amplification, expressed in μl
- **Creaz**: the result obtained by interpolation of the calibration curve, expressed in copies/reaction

Any other concentration, dilution or splitting up of the sample must be considering in the formula.

Mix RARE

A proper functioning of the amplification mix can be verified analysing these parameters:

Parameters	Ref.
Rare Positive Control	
Inv(16) RARE	Ct ≤ 32
ABL	Ct ≤ 32

If the amplification reaction of each controls produces a Ct > 32, the session cannot be considered valid and so it must be repeated

Analysis of the samples

In the amplification reaction of each sample, the Ct values specific probe for Inv(16) and the specific probe for the control gene ABL are used to detect the presence of the Target being analyzed.

An increase of the fluorescence of the specific probes for Inv(16) (FAM) and ABL (TexasRed) indicates the presence of the corresponding gene fusion and control gene in the sample.

Due to the nature of the different type of Inv(16)(p13q22) it is possible to have an amplification signal in both mixes (mix ADE and Mix RARE)

FAM – Inv(16)		TexRed - ABL		Interpretation
Mix 1 Inv(16) mix ADE	Mix 2 Inv(16) mix RARE	Mix 1 Inv(16) mix ADE	Mix 2 Inv(16) mix RARE	
Ct ≤ 50	Ct not relevant	Ct not relevant		Inv(16) Detected
Ct not relevant	Ct ≤ 50	Ct not relevant		Inv(16) Detected
Ct Absent	Ct Absent	Ct ≤ Ct of Inv(A)/ABL 1000cps/μl	Ct ≤ Ct of RARE Positive Control	Inv(16) Not Detected
Ct Absent	Ct Absent	Ct > Ct of Inv(A)/ABL 1000cps/μl	Ct of RARE Positive Control Not Relevant	Invalid Result <i>Extraction and/or collection of sample not performed correctly. Proceed by repeating the test from the extraction phase. If the data is confirmed inconclusive, proceed to the collection of a new sample.</i>
Ct Absent	Ct Absent	Ct > Ct of Inv(A)/ABL 1000cps/μl Not Relevant	Ct > Ct of RARE Positive Control	

As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data and the other laboratory tests available for the patient.

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

The use of positive and negative controls in each amplification session allows to verify the correct functioning of the amplification mix and the absence of any contamination

Normalized copy number (NCN)

In the amplification reaction of each sample, Inv(16) Ct values are used to detect and quantify the presence of the mRNA target, while Ct ABL values are used to detect and quantify the presence of the control gene mRNA (validation of the extraction and normalization of the target). The values of Ct Inv(16) and Ct ABL in the amplification reaction of each sample and of the Standards Curves are used to calculate the amount (**copy number – CN**) of Inv(16) and ABL mRNA.

To compensate variations of RNA integrity and differences in the RT step the CBFB/MYH11 gene fusion results from the Inv(16)(p13q22) was normalized to a control gene.

The CN of Inv(16) and ABL mRNA are then used to calculate the percentage of CN of Inv(16) mRNA normalized with respect to CN of ABL mRNA (% Inv(16)).

The ratio of these CN values gives the normalized copy number (NCN):

$$\text{NCN} = (\text{Fusione Gene CN}) / (\text{Control Gene CN})$$

Increase or decrease > 1 log during therapy is deemed to be confirmed. Owing to the variance inherent in the method, an increase or decrease up to 0.5 log will be assessed as the same level of MRD, and the same applies to an increase or decrease below 10 copies Inv(16) per 10000 copies ABL, because of a greater methodological variation limit in this range.

An increase or decrease in the range of 0.5–1 log takes an intermediate position and will be defined as an unconfirmed increase or decrease

PERFORMANCE EVALUATION

Analytical sensitivity:

The analytical sensitivity (Limit of Detection) of this assay is considered the concentration of target nucleic acid corresponding to 95% probability of detection. The analytical sensitivity of the system was assessed by analyzing serial dilutions of synthetic Nucleic Acids, quantified by spectrophotometric analysis, containing the regions of interest of the CBFB/MYH11 gene fusion results from the Inv(16)(p13q22)

	Inv(16)(p13q22) - CBFB/MYH11
Limit of Detection	5,0 cps/μl

Clinical sensitivity:

It is considered as clinical sensitivity the ability to detect true positive samples in the totality of the samples screened as positive. The analysis was made on positive samples tested and confirmed as Inv(16) positive. The test was performed following the method recommendations.

Obtained results show a clinical sensitivity of 100%.

Linearity/Proportionality:

The system linearity was valued analyzing synthetic Nucleic Acids, quantified by spectrophotometric analysis, containing the genomic regions of interest (Inv(16)(p13q22) - CBFB/MYH11) in serial dilutions (1:10) from 100.000 copie/μl to 100 copie/μl in 10μl of extracted material added in the amplification reaction.

The evaluation was performed analyzing calibration curves, that showed these parameters:

		7500 FAST	ROTORGENE-Q	CFX96
RTS conc. 10 ⁵ copies (FAM)	Ct ≤ 25	Average Ct 22,36	Average Ct 19,7	Average Ct 21,47
Correlation Coefficient	0.990 ≤ r ² ≤ 1	Average r ² 0,996	Average r ² 0,999	Average r ² 0,998
Slope	-3,6 ≤ Slope ≤ -3.1	Average slope. – 3,48	Average slope. – 3,39	Average slope. – 3,45
PCR efficiency	90 ≤ Efficiency ≤ 110	Average eff. 93%	Average eff. 97%	Average eff. 94%

Diagnostic Specificity:

It is considered as diagnostic specificity the ability of the method to detect true negative samples. The diagnostic specificity of the system was evaluated by analyzing human genomic samples tested and confirmed as Inv(16) negative.

The results indicate that Inv(16)(p13q22) - CBFB/MYH11 diagnostic specificity is 100%.

Analytical Specificity:

The specificity of the test is guaranteed by the use of specific primers for CBFB/MYH11 fusion gene.

The alignment of the selected regions for specific primers hybridization for CBFB/MYH11 fusion gene with available sequences present in database, demonstrated: their conservation, the absence of significant mutations and the complete specificity for the analysed target.

Cross-Reactivity:

To check the cross-reactivity of the assay, samples tested as positive for other fusion gene involved in Acute Myeloid Leukemia disease were analysed following the method instructions, obtaining the following results:

Positive sample	7500 Fast	Rotorg-Q	CFX96
AML1-ETO	Negative	Negative	Negative
PML-RARA BCR1	Negative	Negative	Negative
PML-RARA BCR2	Negative	Negative	Negative
PML-RARA BCR3	Negative	Negative	Negative

Interferences:

Verify that in the RNA extracted from the sample there is no contamination from mucoproteins and haemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorption of Nucleic Acids) and 280 nm (maximum absorption of Proteins). A pure RNA should have a ratio of approximately 2.

QUALITY CONTROL

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

In accordance with the Clonit srl ISO EN 13485 Certified quality Management System, each lot of Inv(16)(p13q22) - CBFB/MYH11 is tested against predetermined specification to ensure consistent product quality.

BIBLIOGRAPHY

- J Gabert, E Beillard, VHJ van der Velden, W Bi, D Grimwade, N Pallisgaard, G Barbany, G Cazzaniga, JM Cayuela, H Cave ´, F Pane, JLE Aerts, D De Micheli, X Thirion, V Pradel, M Gonza ´lez, S Viehmann, M Malec, G Saglio and JJM van Dongen. "**Standardization and quality control studies of 'real-time' quantitative reverse transcriptase polymerase chain reaction of fusion gene transcripts for residual disease detection in leukemia – A Europe Against Cancer Program**". References: Leukemia (2003) 17, 2318–2357.
- JJM van Dongen, EA Macintyre, JA Gabert, E Delabesse, V Rossi, G Saglio, E Gottardi, A Rambaldi, G Dotti, F Griesinger, A Parreira, P Gameiro, M Gonza ´lez Dia ´z, M Malec, AW Langerak, JF San Miguel and A Biondi. "**Report of the BIOMED-1 Concerted Action: Investigation of minimal residual disease in acute leukemia**". References: Leukemia (1999) 13, 1901–1928. Viehmann, A Teigler-Schlegel, J Bruch, C Langebrake, D Reinhardt and J Harbott. "**Monitoring of minimal residual disease (MRD) by real-time quantitative reverse transcription PCR (RQ-RT-PCR) in childhood acute myeloid leukemia with AML1/ETO rearrangement**". Leukemia (2003) 17, 1130–1136.

TECHNICAL ASSISTANCE

For any question and support please contact our Technical support:

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BIBLIOGRAFIA

- J Gabert, E Beillard, VHJ van der Velden, W Bi, D Grimwade, N Pallisgaard, G Barbany, G Cazzaniga, JM Cayuela, H Cave´, F Pane, JLE Aerts, D De Micheli, X Thirion, V Pradel, M Gonza´lez, S Viehmann, M Malec, G Saglio and JJM van Dongen. "Standardization and quality control studies of 'real-time' quantitative reverse transcriptase polymerase chain reaction of fusion gene transcripts for residual disease detection in leukemia - A Europe Against Cancer Program". References: Leukemia (2003) 17, 2318-2357.
- JJM van Dongen, EA Macintyre, JA Gabert, E Delabesse, V Rossi, G Saglio, E Gottardi, A Rambaldi, G Dotti, F Griesinger, A Parreira, P Gameiro, M Gonza´lez Dia´z, M Malec, AW Langerak, JF San Miguel and A Biondi. "Report of the BIOMED-1 Concerted Action: Investigation of minimal residual disease in acute leukemia". References: Leukemia (1999) 13, 1901-1928.
- Viehmann, A Teigler-Schlegel, J Bruch, C Langebrake, D Reinhardt and J Harbott. "Monitoring of minimal residual disease (MRD) by real-time quantitative reverse transcription PCR (RQ-RT-PCR) in childhood acute myeloid leukemia with AML1/ETO rearrangement. Leukemia (2003) 17, 1130-1136.

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CLONIT S.r.l.