SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

quany Zika (NS5 region)
Detection and quantification of the Zika Virus with Real Time PCR

REV ST.RT-40-ENITA.4

REF RT-40 - 48 tests

Instructions For Use

INTRODUCTION AND PURPOSE OF USE

The **quany Zika** system is a quantitative test that allows the RNA amplification and quantification, by means of *Real Time PCR*, of NS5 region of Zika Virus RNA.

The Procedure allows the detection of the RNA target by means a retro-amplification reaction.

The analysis of the results is made using a Real Time PCR analyzer (thermal cycler integrated with a system for fluorescence detection and a dedicated software).

CONTENT

The kit contains reagents enough to perform 48 amplification tests:

	Quantity	Description
R1	3 x 10 µl	Enzyme mMix Reverse Transcriptase e Taq Polymerase enzymes (Pink cap)
R2	3 x 270 µl	Zika probe Mix Upstream primer, downstream primer, Target probe (FAM), Internal Control (IC) probe (VIC), Nuclease-free water, ROX dNTPs, Tris-HCl, KCl, MgCl ₂ (White Cap)
R3	3 x 35 µl	synthetic RNA corresponding to NS5 region 100.000 cps/µl (Blue cap)
R4	3 x 35 µl	synthetic RNA corresponding to NS5 region 10.000 cps/µl (Green cap)
R5	3 x 35 µl	synthetic RNA corresponding to NS5 region 1.000 cps/µl (Yellow Cap)
R6	3 x 35 µl	synthetic RNA corresponding to NS5 region 100 cps/µl (White cap)
R7	3 x 160 µl	Internal control phage MS2 (Red Cap)
R8	1 x 50 µl	Negative Control

MATERIALS AND STRUMENTATION REQUIRED BUT NOT SUPPLIED

Disposable latex powder-free gloves or similar material;
 Bench microcentrifuge (12,000 - 14,000 rpm);
 Micropipettes and Sterile tips with aerosol filter;
 Vortex;
 Plastic materials (microplate and optical adhesive cover);
 Heat block (only for extraction)

Reagents

The **quany Zika** kit was developed and validated to be used with the following extraction method:

Manual Extraction

Ref. 52906. *QIAmp Viral RNA Mini Kit*

The kit allows the manual RNA extraction from tested samples. The kit contains reagents for 250 samples. (QIAGEN)

Instruments

The **quany Zika** kit was evaluated to be used with the following instruments:

Real Time PCR

The **quany Zika** kit was evaluated to be used with the following real time PCR instruments:

- *Rotor-Gene Q MDx* from QIAGEN
- *CFX96 Real Time PCR System* from Bio-Rad

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturer's instruction and recommendations.

SAMPLES

The quanty Zika kit must be used with extracted RNA from biological samples: **Serum, Saliva and Urine**. Collected samples must be shipped and stored at +2 - +8°C and used within 3 days from the collected data.

PRECAUTIONS FOR USE

- This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.
- Carefully read this Instruction For Use before using the kit.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: RNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- Periodically wipe the working area with 0,5% hypochlorite.
- Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.
- Use powder-free gloves. Do not leave fingerprints on optical caps. Do not write on caps as this may cause an interference with fluorescent detection.
- Avoid any contact between hands and eyes or nose during specimen collection and testing.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood. Waste should be discarded according to local law.
- Never pipette solutions by mouth.
- Use calibrated and regularly checked pipettes and instrumentation only.
- Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.
- Wash hands carefully after handling samples and reagents.
- Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
- Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

LIMITS OF THE METHOD

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and/or controls. Therefore, you should:

- physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products
- use tips with filters to prevent cross-contamination between samples
- use disposable gloves and change them frequently
- carefully open test tubes to prevent aerosol formation
- close every test tube before opening another one

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

Patient Drug treatment may interfere with the final result of the molecular biology analysis.

The proper functioning of the amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

WARNINGS

- Read carefully the instructions for use before using this test.
- Use only extracted RNA from Siero and/or Plasma and/or Urine
- Do not mix reagents from different lots.
- Thaw and carefully mix the reagents of the kit before use.
- The PCR mix has to be freshly prepared every time.

After reconstitution, the amplification master mix must be used in one time (16 reactions). Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.

- Do not use beyond the expiration date which appears on the package label.
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- In case of damaged package, contact the technical support before using the kit.
- In case of any serious incident that has occurred in relation to the device, a notice shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

STORAGE AND STABILITY

Store the product quanty Zika at –20°C.

The quanty Zika kit is shipped on dry ice. The kit components should be frozen.

If one or more components are not frozen upon receipt or if the tubes have been compromised during transport, contact Clonit srl for assistance. An intact and well stored product has a stability of 12 months from the date of production. Do not use beyond the expiration date which appears on the package label. Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently

ANALYTICAL PROCEDURE

Manual Extraction

Ref. 52906. QIAmp Viral RNA Mini Kit

Procedure to Serum, Saliva and Urine

Follow the instructions inside the kit QIAmp Viral RNA Mini Kit.

After the incubation period for 10 min at room temperature, add 5 µl of Internal Control

Follow the instructions inside the kit

Elute the sample in 50 µl of buffer AVE.

Samples are now ready for amplification or storage at -80°C

SOFTWARE SETTING

Rotor Gene Q MDx

The experiments can be **set using the Quick Start Wizard or the Advanced Wizard**, which appears when the software is started. Select the wizard "**Advanced**". As a first step, select the model "**Two Step Reaction**" with a double click in the "**New Run**". In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "**Locking Ring Attached**", check the checkbox and then click "**Next**".

Enter the name of the operator and the reaction volume of 30 µl, and then click "**Next**".

In the next window click on "**edit profile**". Set the following thermal cycle:

cycles	denaturation	annealing	extension
1	50° C 20 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

Select the annealing phase from the thermal profile and click on "**Acquiring A to cycling**".

In the next window, select yellow from the available channels and add it to acquiring channel along with the green channel and click "**OK**". In the next window click on "**OK**" and then click "**Next**".

Click on "Edit Gain" button and set the following values for each channel:

Reporter	Gain
Green	5
Yellow	5
Orange	5

To begin the run, click on the button "**Start Run**". You can save the model before you begin your run by clicking on "**Save Template**". After clicking on the button "**Start Run**" window appears "**Save As**". The stroke can be saved in the desired position by the user.

Once the run started, the window "**Edit Samples**" allows you to set the name of samples and controls in the positions in which they were loaded on the instrument. Select the locations where were positioned the controls of known concentration and identify them as Zika Virus Standard. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Standards**". Enter the concentrations of the controls. Select the position where you placed the Negative Control and name it as Negative Control. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Negative Controls**".

Select the position of each sample and enter the name or code of the patient. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Unknown**". At the end of the operation click "**OK**" in the "**edit samples**" and wait until the end of the race for the analysis (see "**Interpretation of Results**").

CFX96 Real Time PCR

Turn the instrument and the computer on and start the control software. In the principal screen will appear the window "**Startup wizard**": select "**CFX96**" and press "ok". In the next window push "**create new**" and set the thermal protocol, with the "collect data" in annealing phase, and set the reaction volume (30µl):

cycles	denaturation	annealing	extension
1	50° C 20 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

Save the protocol and click the next button. The software will open by default the sheet "**plate**". Click "**create new**" and select the "**Fluorophores**" button to choose fluorophores (FAM and VIC). Select the locations where they were positioned the controls of known concentration and choose the "Sample Type" **Standards**. Click "Load" check boxes to load fluorophores and Type or select Target Name. In the box "load concentration" set the concentrations of the 4 calibrators, following the instructions indicated in the paragraph Analysis of the results. Select the location where you placed the Negative Control. Choose the "**Sample Type**" **NTC**. Click "**Load**" check boxes to load fluorophores and Type or select Target Name. Select the location of each sample and enter the name or code of the patient. Choose the "**Sample Type**" **Unknown**. Click "**Load**" check boxes to load fluorophores and Type or select Target Name. Save the plate clicking the next button and start the experiment.

PREPARATION OF THE REACTIONS:

Thaw a tube of **Enzyme mMix**;

Thaw a tube of **Zika probes Mix**;

Mix carefully by vortex **9 μ l** of **Enzyme mMix** and **260 μ l** of **Zika probes Mix** (the mix as produced is enough to prepare **16 reactions** of amplification: **4 positive controls, 1 negative control and 11 samples**).

For a total number of tests other than 16, prepare a mixture for $n + 1$ tests, following the volumes shown in the table:

Reagent	Description	Volume for single test (μ l)
R1 -	Enzyme mMix	0,5
R2	Zika probes Mix	14,5

Distribute, in the amplification plate, **15 μ l of just reconstituted mix** in chosen positions as already set on the instrument software.

Distribute, in the negative control position, **15 μ l** of solution taken by the negative control vial.

Distribute, in chosen position for each sample, **15 μ l** of corresponding sample.

Distribute, in chosen positions for the positive controls, **15 μ l of 10² copies/ μ l, 10³ copies/ μ l, 10⁴ copies/ μ l and 10⁵ copies/ μ l.**

Seal up accurately the plate using an optical adhesive film and verify that there aren't air bubbles in the mix, to avoid the amplification interferences. For the Rotor-Gene Q MDx, seal each tube with the appropriate cap. The air bubbles presence is not influent; the rotor centrifugal force will allow automatic deletion. Transfer the plate in the instrument and push the button "**Start Run**".

QUANTITATIVE ANALYSIS

Rotor Gene-Q MDx

At the end of the PCR run click "**Option**" button under the amplification plot on green channel. Select "**normalize to Cycling A. orange**".

Open the "**Analysis**" window. Select the "**Quantification**" sheet and click on "**cycling A.green/cycling A.orange**". Select from the menu "**Dynamic Tube**" and subsequently "**Slope correct**". If necessary, reduce the background noise of non-specific signals by selecting the "**Outlier Removal**" option and entering a value between 5% and 30%.

Check the correct setting of the threshold in the space provided "**CT calculation – Threshold**".

Repeat the steps above for each of the fluorescence channels "**cycling A (yellow)**"

You can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification **cycling A.green/cycling A.orange** and then the file **cycling A.yellow/cycling A.orange**

CFX96 Real Time PCR System

At the end of the PCR, select the "**quantitation**" sheet. On the top of the screen, select "**settings**" from the menu and choose "**Baseline Threshold...**".

You can export the report pushing the paper block figure on the top of the screen.

INTERPRETATION OF RESULTS

Through the Real Time PCR reaction it is possible to obtain the RNA quantification of Zika Virus RNA, by setting the values of the positive controls of the calibration curve. To calculate these values, all the dilutions steps that the sample has undergone during the extraction and amplification stages must be considered.

The system can detect until 15 copies of RNA per reaction. The Ct values obtained from the amplification of 4 controls of known titer are used by the software for the calculation of the calibration curve from which the unknown samples are interpolated. A proper functioning of the amplification mix can be verified analyzing these parameters:

Parameters	Reference
RTS conc. 10 ⁵ copie/ μ l (FAM)	Ct \leq 25
Correlation Coefficients	0.990 \leq r ² \leq 1
Slope	-3,6 \leq Slope \leq -3.1
PCR Efficiency	90 \leq Efficiency \leq 110

If the RTS amplification reaction at a concentration of 10⁵ copies produces a Ct > 25 or undetermined the session can't be considered valid and must be repeated. The correlation coefficient value (r²), the slope and the reaction efficiency should fit to the limits indicated in the above table. By correctly setting the standards concentration as a function of the extraction system you can get the quantization of the sample directly in copies / ml:

	Manual Extr.	Alternative extract.
	QIAmp Viral RNA mini kit	
	copie/ml	Copie/reaz
RTS 1	35.700.000	1.500.000
RTS 2	3.570.000	150.000
RTS 3	357.000	15.000
RTS 4	35.700	1.500

When alternative extraction systems are used sample concentration expressed in copies/ml will be obtained using the formula:

$$\text{copie/ml} = \frac{1000}{V_e} \times \frac{E_v}{E_a} \times C_{\text{reaz}}$$

where:

- **Ve**: extracted sample Volume expressed in μl
- **Ev**: eluted sample Volume during extraction stage expressed in μl
- **Ea**: extracted sample volume used for amplification expressed in μl
- **C_{reaz}**: copies provided by the instrument.

As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data and the other laboratory tests done on the patient. As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results. The use of positive and negative controls in each amplification session allow to verify the correct functioning of the amplification mix and the absence of any contamination. In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from extraction process until detection step. In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from reverse transcription process until detection step. Be sure that fluorescence emitted from internal control amplification has not a Ct > 35 or undetermined. If a sample shows an undetermined Zika Virus RNA and an internal control Ct >35, this means that there have been problems in the extraction stage or in the retro-amplification stage; therefore the sample could be a false negative. **Repeat the sample.**

It can be considered valid the samples with a Ct > 35 for the internal control, and a high concentration of Zika Virus RNA. In this case, the competitive nature of PCR reaction can hide or disadvantage the internal control amplification.

Detector FAM	Detector VIC/Hex	PCR Run	Sample
Ct undetermined	Ct > 35 or undetermined	Not Valid	repeat
Ct undetermined	Ct < 35	Valid	Negative
High Ct	Ct < 35	Valid	Positive
Low Ct	Ct > 35 or undetermined	Valid	HighPositive

PERFORMANCES

Analytical sensitivity:

It is considered as analytical sensitivity the highest dilution (title) to which a positive sample can be diluted without the system losing the ability to detect it, with a positivity rate of $\geq 95\%$. The analytical sensitivity of the system was assessed by analyzing synthetic RNA, quantified by spectrophotometric analysis, containing the regions of interest (NS5) of the virus in serial dilutions.

Analytical sensitivity of **quany Zika** determined by Probit analysis:

		Average cps/ μl	Manual Extraction
Probit Analysis	Copies/ μl	1,15 cps/ μl	410,71 cps/ml
	95% Confidence interval	Lower 0,332 cps/ μl	118,57 cps/ml
		Upper. 12,14 cps/ μl	4335,71 cps/ml

Linearity/Proportionality

The system linearity was evaluated analyzing syntetic RNA (pGEM-3Z-ZIKA_2), quantified by spectrophotometric analysis, containing the regions of interest NS5 of the virus in serial dilutions (1:10) from 150.000.000 copie/reactions to 150 copies/reaction of RNA in 15 μl of extracted material added in the retro-amplification reaction.

The evaluation was performed analyzing 10 calibration curves, that showed these parameters:

RTS conc. 10 ⁵ copies (FAM)	Ct ≤ 25	Medium Ct 20,68
Correlation Coefficient	0.990 $\leq r^2 \leq 1$	Medium r ² 0,999
Slope	-3,6 \leq Slope \leq -3.1	Medium slope -3,380
PCR efficiency	90 \leq Efficiency \leq 110	Medium Eff 98%

Reproducibility and Repeatability

The reproducibility and repeatability of the system were valued analyzing 3 dilutions of synthetic RNA containing the NS5 region of interest for Zika Virus and quantified by spectrophotometric analysis, plus a negative control (negative RNA).

For each session, 5 replicates were made in 3 different sessions, performed by different technicians, using 3 different lots of product

Hypothetical value	Lot	N° repetitions	Med. Reveal. Conc.	Inaccuracy %
1.500.000 copie	L1	15	1515541	1%
1.500.000 copie	L2	15	1631745	9%
1.500.000 copie	L3	15	1574864	5%
15.000 copie	L1	15	15095	1%
15.000 copie	L2	15	17088	14%
15.000 copie	L3	15	13985	7%
150 copie	L1	15	170	13%
150 copie	L2	15	168	12%
150 copie	L3	15	111	26%

The average Inaccuracy percentage of Zika Virus method is 10%.

Analytical Specificity:

Test's specificity was guaranteed by the use of specific primers for Zika Virus.

The alignment of the chosen regions for specific primers' hybridization for Zika Virus with available sequences of the NS5 region present in database, demonstrated: their conservation, the absence of significant mutations and the complete specificity for the analysed target.

INTERFERENCES:

Verify that in the RNA extracted from the sample there is no contamination from mucoproteins and haemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorption of Nucleic Acids) and 280 nm (maximum absorption of Proteins). A pure RNA should have a ratio of approximately 2.

QUALITY CONTROL

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

In accordance with the Clonit srl ISO EN 13485 Certified quality Management System, each lot of **quany Zika** is tested against predetermined specifications to ensure consistent product quality.

BIBLIOGRAPHY

O.Faye, O.Faye, D.Diallo, M.Diallo, M.Weidmann and A.Alpha Sall

Quantitative real-time PCR detection of Zika Virus and evaluation with field-caught Mosquitoes

Virology Journal 2013, 10 :311

Ann.Claire Gourinant et al.

Detection of Zika Virus in Urine

Emerging Infectious diseases – www.cdc.gov/eid - volume 21, No.1, January 2015

R.S.Lanciotti et al

Genetic and Serologic Properties of Zika Virus Associated with an epidemic, Yap State, Micronesia, 2007

Emerging Infectious diseases – www.cdc.gov/eid - volume 14, No.8, August 2008

K.Shinohara et al

Zika fever imported from Thailand to Japan and diagnosed by PCR in the Urine

Journal of Travel medicine 2016, 1-3 doi :10.1093/jtm/tav011

TECHNICAL ASSISTANCE

For any question and support please contact our Technical support:

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CLONIT S.r.l.