SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

Leishmaniae spp
Detection and amplification of the Leishmaniae genome with Real Time PCR

REV ST.RT-63-ENITA.0

REF RT-63 - 48 tests

Instructions For Use

INTRODUCTION AND PURPOSE OF USE

The Leishmaniae spp system is a quantitative test that allows the DNA amplification, by means of *Real Time PCR*, of *Leishmaniae* genome extracted from biological samples.

The procedure allows the detection of the DNA target (*18s Ribosomal RNA gene*) through a genomic amplification reaction.

The analysis of the results is made using a Real Time PCR analyzer (thermal cycler integrated with a system for fluorescence detection and a dedicated software).

CONTENT

The kit contains reagents enough to perform 48 amplification tests:

	Quantity	Description
R1	3 x 220 µl	Amplification mMix dNTPs, Tris-HCl, KCl, MgCl ₂ , Taq Polymerase, <i>AmpErase</i> Uracil N-Glycosylase (<i>UNG</i>), Nuclease-free water, ROX (Pink Cap)
R2	3 x 130 µl	Leishmaniae probes Mix Upstream primer, downstream primer, Target probe (FAM), Internal control (β-globin) Probe (VIC) Nuclease-free water (White cap)
R3	3 x 35 µl	Leishmaniae High concentration
R4	3 x 35 µl	Leishmaniae Low concentration
R5	1 x 30 µl	Negative Control

MATERIALS AND STRUMENTATION REQUIRED BUT NOT SUPPLIED

Disposable latex powder-free gloves or similar material;
 Bench microcentrifuge (12,000 - 14,000 rpm);
 Micropipettes and Sterile tips with aerosol filter;
 Vortex;
 Plastic materials (microplate and optical adhesive cover);
 Dry block shaker for 1.5ml conical tubes
 Magnetic rack for 1.5ml conical tubes
 Heat block (only for extraction)
 EZ1 Advanced XL DSP Virus Card. - Ref. 9018703 - QIAGEN.
 ATL Buffer - Ref. 939016 - QIAGEN.

Reagents

The **Leishmaniae spp** kit was developed and validated to be used with the following extraction method:

Manual Extraction

Ref. 51304/51306 *QIAmp DNA mini kit*. The kit allows the DNA extraction from tested samples. The kit contains reagents for 50/250 samples.(QIAGEN).

Automatic Extraction

Ref. 62724. *EZ1 XL DSP Virus Kit* The kit allows the automatic viral DNA from Human samples. The kit contains reagents for 48 samples. (QIAGEN)

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10629800 - *VERSANT® Sample Preparation 1.2 Reagents kit box 1*. The kit allows the manual DNA extraction from Human samples. The kit contains reagents enough to perform the DNA extraction for 96 samples.

10629801 - *VERSANT® Sample Preparation 1.2 Reagents kit box 2*. The kit allows the manual DNA extraction from Human samples. The kit contains reagents enough to perform the DNA extraction for 96 samples.

Instruments

The **Leishmaniae spp** kit was evaluated to be used with the following instruments:

Extraction System

Ref. 9001492. *EZ1 Advanced XL*.

Robotic Workstation for the automatic purification of the nucleic acids until 14 samples simultaneously (QIAGEN)

Real Time PCR

The **Leishmaniae spp** kit was evaluated to be used with the following real time PCR instruments:

- 7500 Fast from Lifetechnologies
- StepOne Plus from Lifetechnologies
- Versant kPCR AD from Siemens o Stratagene MX3005P/MX3000P
- Rotor-Gene Q MDx from QIAGEN
- CFX96 Real Time PCR System from BioRad
- LightCycler 480 from Roche

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturer's instruction and recommendations

SAMPLES

The **Leishmaniae spp** system must be used with extracted DNA from the following biological samples: **Whole blood EDTA**. Collected material must be shipped and stored at +2 - +8°C. Store the samples at -20°C if not used within 3 days..

PRECAUTIONS FOR USE

- This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.
- Carefully read this Instruction For Use before using the kit.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: RNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- Periodically wipe the working area with 0,5% hypochlorite
- Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.
- Use powder-free gloves. Do not leave fingerprints on optical caps. Do not write on caps as this may cause an interference with fluorescent detection.
- Avoid any contact between hands and eyes or nose during specimen collection and testing.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood. Waste should be discarded according to local law.
- Never pipette solutions by mouth.
- Use calibrated and regularly checked pipettes and instrumentation only.
- Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.
- Wash hands carefully after handling samples and reagents.
- Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
- Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

LIMITS OF THE METHOD

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and/or controls. Therefore, you should:

- physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products
- use tips with filters to prevent cross-contamination between samples
- use disposable gloves and change them frequently
- carefully open test tubes to prevent aerosol formation
- close every test tube before opening another one

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results. Patient Drug treatment may interfere with the final result of the molecular biology analysis. The proper functioning of the amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

WARNINGS

- Read carefully the instructions for use before using this test.
- Use only extracted DNA from Whole blood EDTA.
- Do not mix reagents from different lots.
- Thaw and carefully mix the reagents of the kit before use.
- The PCR mix has to be freshly prepared every time.

After reconstitution, the amplification master mix must be used in one time (16 reactions). Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.

- Do not use beyond the expiration date which appears on the package label.
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- In case of damaged package, contact the technical support before using the kit.
- In case of any serious incident that has occurred in relation to the device, a notice shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

STORAGE AND STABILITY

Store the product **Leishmaniae spp** at -20°C..

The **Leishmaniae spp** kit is shipped on dry ice. The kit components should be frozen. If one or more components are not frozen upon receipt or if the tubes have been compromised during transport, contact Clonit srl for assistance. An intact and well stored product has a stability of 12 months from the date of production. Do not use beyond the expiration date which appears on the package label. Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The reagents should be frozen in aliquots, if they are to be used intermittently.

ANALYTICAL PROCEDURE

Manual Extraction

Ref. 51304/51306 - QIAmp DNA mini kit. (QIAGEN).

Procedure to Whole blood

Follow the instructions inside the kit QIAmp DNA mini kit.

Elute the sample in 50 µl of buffer AE Samples are now ready for amplification or storage at -20°C

Automatic Extraction

Ref. 62724 - EZ1 XL DSP Virus Kit on EZ1 Advanced XL.

Procedure to Whole Blood

Follow the instructions inside the kit EZ1 XL DSP Virus Kit.

Volume of sample to be used:

Whole blood [µl]	ATL [µl]	Final volume Samples [µl]
200	200	400

Preparation of the Carrier

Solve completely the lyophilize RNA carrier in elution buffer (AVE), from 310 µl, split in aliquots and store to $-20 \pm 5^\circ\text{C}$. Not thawing and freezing the aliquots more than 2 times. For each analyzed sample, dilute 3,6 µl of a original solution include the RNA Carrier in total volume of 60 µl using elution buffer (AVE) Follow the instructions inside the kit EZ1 XL DSP Virus Kit. Select the protocol starting from 400 µl of samples with the elution of 60 µl.

Manual extraction (SIEMENS)

Ref. 10629800 - VERSANT® Sample Preparation 1.2 Reagents kit box 1.

Ref. 10629801 - VERSANT® Sample Preparation 1.2 Reagents kit box 2.

Follow the instructions supplied by Siemens and elute it in 70 µl of Elution buffer. Transfer 55 µl of eluted sample to an appropriately size tube. Samples are now ready for amplification or storage at -20°C.

SOFTWARE SETTING

Lifetechnologies 7500 fast/StepOne plus

Turn the instrument and the computer on and open the control software. Click on "Advance Setup": by default the software will shows the page "experiment properties". Write in the "experiment name" the file name, choose the type of instrument (**7500 or 7500fast / StepOne or StepOne Plus**), the type of reaction (**quantitation standard curve**), the type of reagents used (**Taqman®Reagents**) and the analysis reaction time (**Standard ≈ 2 hours to complete a run**).

Open the page named "page setup" (sheet **Define Target and Samples**). In the window **Define Targets** set:

Target	Reporter	Quencer
Leishmaniae probe:	FAM	TAMRA
IC (β-globin) probe:	VIC	TAMRA

Set the samples' name in the window "Define Samples".

In the same page "plate setup" select the sheet "Assign Target and Samples". On the screen you will see the microplate draft. Select an area of the plate where the controls will be placed: select wells of the plate and set both targets (Leishmaniae and b-globin). Select "Assign target to selected wells" in the blank, the "task Standard (S)" for Leishmaniae target.

Choose an area in the plate where negative control will be placed: select "Assign target to selected wells" in the blank, the "task Negative (N)" for the Leishmaniae target.

Select an area of the plate where samples will be placed: select the wells and set both targets (Leishmaniae and β-globin). Link every well to a sample, through the window "Assign samples to selected wells".

For each sample, select in the blank "Assign targets to selected wells" the "task UnKnown (U)" for the Leishmaniae target.

Set ROX as passive reference, using it as normalizer of detected fluorescence.

Open "Run Method" (sheet **Graphic View**) and set the thermal cycling as follows, with the "collect data" in annealing/extension phase:

cycles	denaturation	annealing/extension
1	50° C 2 min	
1	95°C 10 min	
45	95° C 15 sec	60° C 1 min

In the window "Reaction volume plate per well" set a volume of 25 µl.

After having prepared the plate, and correctly inserted in the instrument, press the button "Start Run".

Rotor-Gene Q MDx

The experiments can be set using the **Quick Start Wizard** or the **Advanced Wizard**, which appears when the software is started.

Select the wizard "**Advanced**". As a first step, select the model "Two Step Reaction" with a double click in the "New Run".

In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "Locking Ring Attached", check the checkbox and then click "Next". Enter the name of the operator and the reaction volume of 25 µl, and then click "Next".

In the next window click on "edit profile". Set the following thermal cycle:

cycles	denaturation	annealing/extension
1	50° C 2 min	
1	95°C 10 min	
45	95° C 15 sec	60° C 1 min

Select the annealing / extension from the thermal profile and click on "Acquiring A to cycling."

In the next window, select **yellow** from the **available channels** and add it to **acquiring channel** along with the green channel and click **"OK"**. In the next window click on **"OK"** and then click **"Next"**. Click on "Edit Gain" button and set the following values for each channel:

Reporter	Gain
Green	5
Yellow	5

To begin the course, click on the button **"Start Run"**. You can save the model before you begin your run by clicking on **"Save Template"**. After clicking on the button **"Start Run"** window appears **"Save As"**.

The stroke can be saved in the desired position by the user. Once the run started, the window **"Edit Samples"** allows you to set the name of samples and controls in the positions in which they were loaded on the instrument. Select the locations where you placed the controls. Clicking on the box next to **"Type"** correspondent, in the dropdown menu **"Samples"** you can select the type of sample being analyzed. Select **"Positive control"**.

Select the location where you placed the Negative Control and name it as Negative Control. Clicking on the box next to **"Type"** correspondent, in the dropdown menu **"Samples"** you can select the type of sample being analyzed. Select **"Negative Controls"**. Select the location of each sample and enter the name or code of the patient. Clicking on the box next to **"Type"** correspondent, in the dropdown menu **"Samples"** you can select the type of sample being analyzed. Select **"Unknown"**. At the end of the operation, click **"OK"** in the **"edit samples"** and wait until the end of the race for the analysis (see **"Interpretation of Results"**).

Versant kPCR AD or Stratagene MX3005P/MX3000P

Turn the instrument on and wait until both green lamps have fixed light, turn on the computer and start the control software. In the principal screen will appear the window **"New Experiment Options"**: select **"Experiment type": quantitative PCR (Multiple Standard)**. Turn the lamp on 20 minutes before doing a new experiment. To turn the lamp on, click on the icon of the lamp in the tool bar or select **"Lamp On"** from the menu **"Instruments"**.

Verify the right setting of the gain of the fluorescent reporters: in the menu of settings, choose: **"Instrument"** and then **"Filter set gain setting"**

Reporter	Gain
FAM	8
JOE/HEX	2
ROX	1

Click on button **"setup"** in the toolbar and choose **"Plate Setup"**.

Sign the wells correspondent to positive controls. Define the high and low control positions in the menu by setting:

Well type:	Collect Fluorescent Data:	Reference Dye:	Replicate Symbol:
Pos. Control FAM	FAM/HEX/ROX	ROX	None

Clicking on every single well it will appear the window **"well information"**. Choose the name of the positive control (*Leishmania High Control and Leishmania Low Control*).

Identify the wells correspondent to Negative control. Define the Negative control positions in the menu by setting:

Well type:	Collect Fluorescent Data:	Reference Dye:	Replicate Symbol:
NTC	FAM/HEX/ROX	ROX	None

Clicking on every single well will appear the window **"well information"**, and you can set NTC as the name.

Set near the name of the fluorescent reporters, the name of the analyzed targets:

FAM	HEX
Leishmania	β -Globin

Sign the wells correspondent to samples. Define the sample positions in the right menu, setting:

Well type:	Collect Fluorescent Data:	Reference Dye:	Replicate Symbol:
Unknown	FAM/HEX/ROX	ROX	None

Clicking on every single well it will appear the window **"well information"**; it is possible to write in the name or the code of the sample. It's possible to set the name of the dye near the name of the analyzed target:

FAM	HEX
Leishmania	β -Globin

In the tool bar choose the sheet **"Thermal Profile Setup"** and set the correct thermal cycle and reading the fluorescence in the annealing/extension step

<u>cycles</u>	<u>denaturation</u>	<u>annealing/extension</u>
1	50° C 2 min	
1	95°C 10 min	
45	95° C 15 sec	60° C 1 min

After preparing the plate and inserting it in the instrument, press the button **"Run"**, selecting the sheet Thermal profile status and check the accuracy of the thermal profile. Select the box **Turn Lamp Off** at the end of execution. Push the button Start: the software will ask to indicate the name of saved file and will begin the analysis.

CFX 96 Real Time PCR

Turn the instrument and the computer on and start the control software. In the principal screen will appear the window "**Startup wizard**": select "**CFX96**" and press "**ok**". In the next window push "**create new**" and set the thermal protocol and the reaction volume (25µl):

cycles	denaturation	annealing/extension
1	50° C 2 min	
1	95°C 10 min	
45	95° C 15 sec	60° C 1 min

Save the protocol and click the next button. The software will open in default the sheet "**plate**". Click "**create new**", select "**Fluorophores button**" to choose fluorophores (FAM and VIC). Select the locations where they were positioned the controls of known concentration and choose the "**Sample Type**" **Standards**. Click "**Load**" check boxes to load fluorophores and Type or select Target Name. Select the location where you placed the Negative Control. Choose the "**Sample Type**" **NTC**. Click "**Load**" check boxes to load fluorophores and Type or select Target Name. Select the location of each sample and enter the name or code of the patient. Choose the "**Sample Type**" **Unknown**. Click "**Load**" check boxes to load fluorophores and Type or select Target Name. Save the plate clicking the next button and start the experiment

LightCycler 480

Turn the instrument and the computer on and start the control software. In the principal screen, on "**Experiment Creation**" select the "**Plate type**" and push the "**New experiment**" button. The window "experiment" appears. On the "Run protocol" sheet set: Thermal protocol, Reaction volume (25 µl) and Detection format (**dual colour hydrolysis probe**), with the "collect data" in annealing/extension phase:

cycles	denaturation	annealing/extension
1	50° C 2 min	
1	95°C 10 min	
45	95° C 15 sec	60° C 1 min

Push the "**Subset editor**" button and in this window, select an area of the plate where controls and samples will be placed. Push the "**Sample editor**" button. Select the correct workflow (Step1: **Abs quantification**), choose the samples Subset created in the step before and insert the name for each well. Choose the correct type for each well: Positive CTR, Negative CTR or Unknown sample. Push again the "**experiment button**", insert the plate in the instrument and push "**start run**"

PREPARATION OF THE REACTIONS:

Thaw a tube of **Amplification mMix**;

Thaw a tube of **Leishmaniae probes Mix**;

Mix carefully by vortex **210µl** of **Amplification mMix** and **126µl** of **Leishmaniae probes Mix** (the prepared master mix will allow to perform **16 reactions** of amplification: **2 positive controls, 1 negative control and 13 samples**).

For a total number of tests other than 16, prepare a mixture for n + 1 tests, following the volumes shown in the table:

Reagent	Description	Volume for single test (µl)
R1 -	Amplification mMix	12,5
R2	Leishmaniae probes Mix	7,5

Distribute, in the amplification plate, **20 µl of just reconstituted mix** in chosen positions, as already setted on the instrument software.

Distribute, in the negative control position, **5 µl** of the negative control.

Distribute, in chosen position for each sample, **5 µl** of corresponding sample.

Distribute, in chosen positions for the positive controls, **5 µl** of **low control** and **high control**.

Seal up accurately the plate using an optical adhesive film and verify that there aren't air bubbles in the mix, to avoid the amplification interferences.

For the Rotor-Gene Q MDx, seal each tube with the appropriate cap. The air bubbles presence is not influent; the rotor centrifugal force will allow automatic deletion.

Transfer the plate in the instrument and push the button "**Start Run**".

QUALITATIVE ANALYSIS

Lifetechnologies 7500 Fast, StepOne Plus.

At the end of the PCR run, the software automatically opens the "**Analysis**" window in the "**Amplification plot**" sheet on the menu on the left.

Select the wells corresponding to the positive control, negative control and samples for analysis.

Select in the "**Option**" window inside the "**Target**" pop-up menu the **Leishmania target**.

Check the correct setting of the threshold.

Select in the "**Option**" window inside the "**Target**" pop-up menu the **IC Control target**.

Check the correct setting of the threshold.

The analysis of the results is made selecting from the menu in the left the page "**Analysis**".

From the page "**Amplification Plot**" verify the amplification plot for every single sample.

Opening the sheet "**view well table**" in the right side of the software it is possible to verify the data obtained from experiments: Threshold Cycles, emitted fluorescences etc...

Clicking from the menu file and selecting the box export, the window "**export properties**" will open. Indicate the file name, select the position to save it (**Browse**) and click on button "**Start export**". In this way the software will permit to save a excel file with all the data corresponding to selected experiment.

Rotor-Gene Q MDx

At the end of the PCR run open the "Analysis" window.

Select the "Quantification" sheet and click on "cycling A (green)".

Select from the menu "Dynamic Tube" and subsequently "Slope correct".

Check the correct setting of the threshold in the space provided "CT calculation – Threshold". Open the "Analysis" window.

Select the "Quantification" sheet and click on "cycling A (yellow)".

Select from the menu "Dynamic Tube" and subsequently "Slope correct".

Check the correct setting of the threshold in the space provided "CT calculation – Threshold".

Also in this case, you can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification cycling A (green) and then the file cycling A (yellow).

Versant Kpcr AD O Stratagene MX3005P/MX3000P

Click on button "Analysis" in the toolbar. The software will open in default the sheet "Analysis Term Setting".

Activate the buttons FAM and HEX in the lower part of the screen and select testing samples.

Click on sheet "results"; the software will open in default the page "Amplification plot".

Check the correct setting of the threshold in the specific window "Threshold fluorescence", in the menu on the right of the screen.

Selecting the box Text report" from menu "Area to Analyze" in the right side of the screen it's possible to verify the data obtained from the experiments: Threshold Cycles, emitted Fluorescences etc..

From the window Text Report it's possible to export the results obtained clicking file, export on main menu.

CFX96 Real Time PCR System

At the end of the PCR, select the "quantitation" sheet. On the top of the screen, select "settings" from the menu and choose "Baseline Threshold..." .

You can export the report pushing the paper block figure on the top of the screen.

LightCycler 480

At the end of the run, select and choose the correct analysis type that you want to analyse: "Abs Quant/Fit Points".

Choose the "subset samples" that you want to analyze. Select the sheet "NoiseBand", below the graph you can choose "NoiseBand (Fluorescand move the line of NoisBand directly from the chart.

Repeat this action for each fluorophore with the "Fuel Filter".

Clicking on the sheet "Analysis" you can set the value of the

Threshold choosing "Threshold (manual)".

After setting the parameters, press the "Calculate".

Repeat this action for each fluorophore.

INTERPRETATION OF RESULTS

In the Real Time PCR reaction the Ct values of specific probe for **Leishmania** are used for detect the presence of the Target in analysis. Fluorescence increase of the specific probe for **Leishmania** (FAM) indicates the positivity of the sample for the target in exam.

The samples that were present Ct values > 40 should be subjected to further verification as close to the limit of sensitivity of the system. As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data and other laboratory tests done on the patient.

The use of positive and negative control in each amplification session allows to verify the correct functioning of the amplification mix and the absence of any contamination. The instrument software is able to analyze the fluorescences that are emitted by the specific probe for **Leishmania** (FAM) and by the specific probe for the positive internal control (β -globin VIC).

A proper functioning of the amplification mix can be verified analyzing these parameters:

Parameters	Ref.
Conc. High control (FAM)	Ct \leq 27
Conc. Low control (FAM)	Ct \leq 33

If the amplification reaction of each controls produces a Ct > 27, the session can't be considered valid and so it must be cancelled.

Be sure that there isn't any specific fluorescence increasing for examining target in negative control (FAM).

In the amplification reaction of each sample, the Ct values for the internal control (β -globin) specific probe are used for validating the analysis session beginning from extraction process until detection stage.

A good extraction performances presents internal control (β -globin) threshold cycle between 22 and 25.

Be sure that emitted fluorescence from internal control amplification has not a Ct > 28 or undetermined. If a sample presents an undetermined Leishmania DNA and internal control Ct >28 means that some problems happened in the extraction stage or in the amplification stage; therefore the sample could be a false negative. **Repeat the sample.**

Detector FAM	Detector VIC/JOE	Assay	Sample
Ct undetermined	Ct > 28 o undetermined	Not valid	Repeat
Ct undetermined	Ct < 28	Valid	Negative
Ct positivo	Ct < 28	Valid	Positive
Ct basso	Ct > 28 undetermined	Valid	High Positive

PERFORMANCES

Analytical sensitivity:

It is considered as analytical sensitivity the highest dilution (title) to which a positive sample can be diluted without the system losing the ability to detect with a positivity rate of \geq 95%. The analytical sensitivity of the system was assessed by analyzing plasmid DNA (pTZ-L), quantified by spectrophotometric analysis, containing the genomic region of interest (18s Ribosomal RNA gene) of the protozoa in serial dilutions

The analytical sensitivity of **Leishmaniae spp** is determined by Probit analysis.

Probit Analysis	Copies/ul	Average cps/ul	Manual Extraction	Automatic Extraction
	95% confidence interval	1,03 cps/ul Lower 0,47 cps/ul Upper. 6,39 cps/ul	257 cps/ml 117 cps/ml 1597 cps/ml	309 cps/ml 141 cps/ml 1917 cps/ml

Clinical sensitivity:

It is considered as clinical sensitivity the ability to detect true positive samples in the totality of the samples screened as positive. The analysis is made on Leishmaniae positive samples and the test is performed following the method advices. Positive samples are confirmed with another disposable method.

Obtained results show a clinical sensitivity of 100%.

Diagnostic Specificity:

It is considered as diagnostic specificity the ability of the method to detect trues negative samples. The diagnostic specificity of the system is valued analyzing human genomic samples tested and confirmed as negative with another disposable system.

Diagnostic specificity is 100% for material extracted from EDTA blood.

Analytical Specificity:

Test's specificity is guaranteed by the use of specific primers for determining Leishmaniae.

The alignment of the choose regions for specific primers' hybridization for Leishmaniae demonstrated: their conservation, the absence of significative mutations and the complete specificity for the analyzed target.

INTERFERENCES:

Verify that in the DNA extracted from the sample there is no contamination from mucoproteins and haemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorbtion of Nucleic Acids) and 280 nm (maximum absorbtion of Proteins). A pure DNA should have a ratio of approximately 1.8.

QUALITY CONTROL

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

In accordance with the Clonit srl ISO EN 13485 Certified quality Management System, each lot of Leishmaniae spp kit is tested against predetermined specification to ensure consistent product quality

BIBLIOGRAFIA

Omran F. Osman, L. Oskam, ED E. Zijlstra, Nel C. M. Kroon, G. J. Schoone, E. A. G. Khalil, A. M. El-Hassan, and P. A. Kager. **"Evaluation of PCR for Diagnosis of Visceral Leishmaniasis"**. Journal of Clinical Microbiology, Oct. 1997, p. 2454–2457

L. Lachaud, E. Chabbert, P. Dubessay, J. Reynes, J. Lamothe, and P. Bastien. **"Comparison of Various Sample Preparation Methods for PCR Diagnosis of Visceral Leishmaniasis Using Peripheral Blood"**. Journal of Clinical Microbiology, Feb. 2001, p. 613–617.

A.C.R. Medeiros, S.S. Rodrigues and A.M.F. Rosolino. **"Comparison of the specificity of PCR and the histopathological detection of leishmania for the diagnosis of American cutaneous leishmaniasis"**. Braz J Med Biol Res. 2002 Apr;35(4):421-4.

Gregory Spanakos, Eleni Patsoula, Tzeni Kremastinou, George Saroglou and Nicholas Vakalis. **"Development of a PCR-based method for diagnosis of Leishmania in blood samples"**. Molecular and Cellular Probes (2002) 16, 415-420.

TECHNICAL ASSISTANCE

For any question and support please contact our Technical support:

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CLONIT S.r.l.