SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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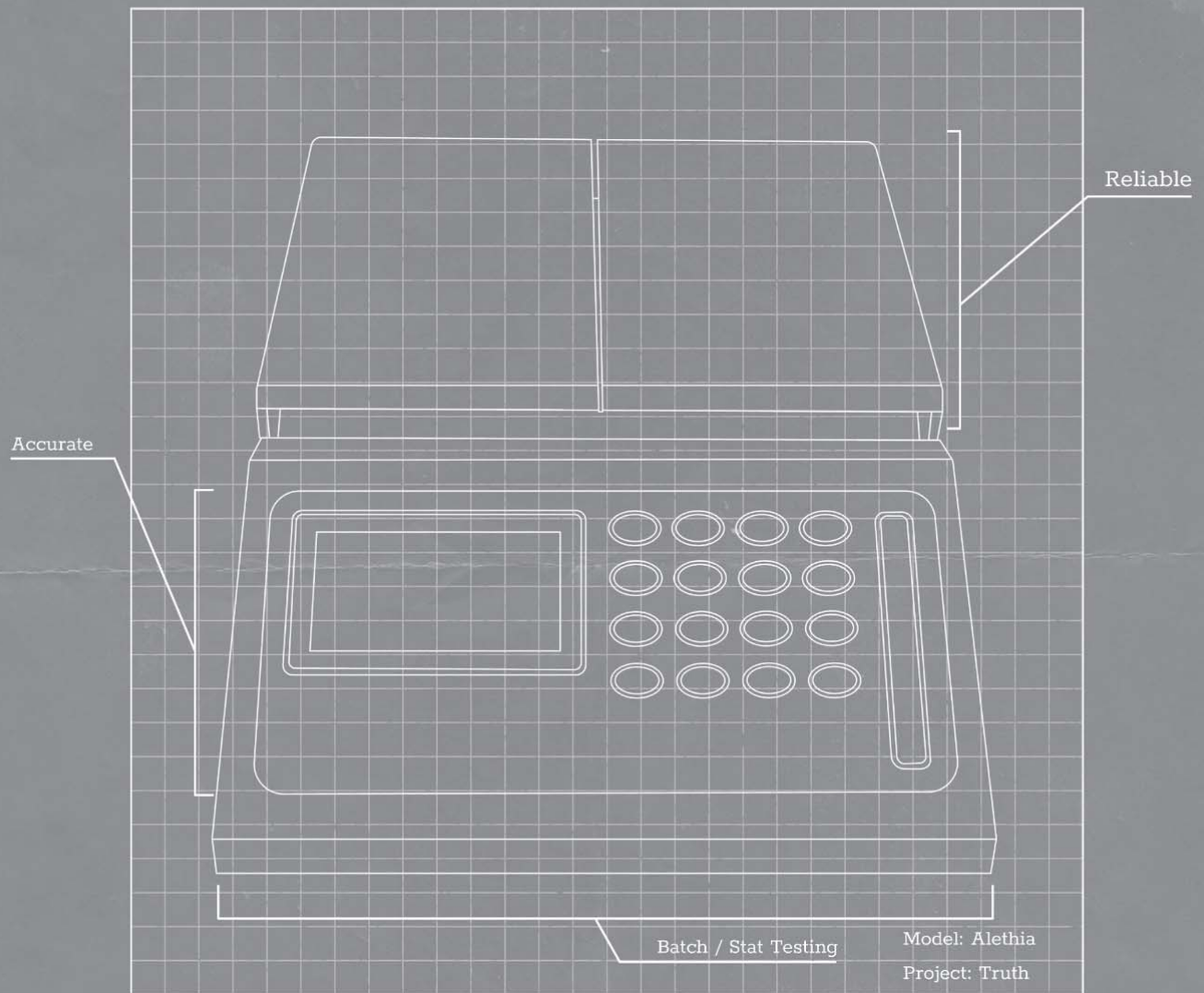
mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

alethia™

by **meridian** BIOSCIENCE

Operator's Manual

SN11042 REV. 07/18

Table of Contents

<i>Instrument Intended Use and Function</i>	3
<i>Installation Procedures and Requirements</i>	3
<i>Principles of Operation</i>	5
<i>Performance Characteristics and Specifications</i>	5
<i>Operating Instructions</i>	7
<i>Calibration Procedures</i>	13
<i>Operational Precautions and Limitations</i>	15
<i>Service and Maintenance</i>	16
<i>Alethia Set-Up and Installation Checklist</i>	Appendix I
<i>Alethia Decontamination Record</i>	Appendix II
<i>Alethia Error Codes and Troubleshooting</i>	Appendix III
<i>Alethia Onboard Barcode Scanner Information</i>	Appendix IV
<i>Alethia Handheld Barcode Scanner Information</i>	Appendix V
<i>Alethia Compliance Testing Summary</i>	Appendix VI

Instrument Intended Use and Function

Alethia is an automated isothermal amplification and detection system for use with Meridian Bioscience, Inc.'s Alethia Loop-Mediated Amplification assays.

Alethia is intended to be used by trained laboratory professional users in laboratory settings.

Installation Procedures and Requirements

Alethia and its accessories are securely packaged to prevent damage during shipment to the end user. The Alethia shipping container and packaging should be inspected for damage prior to installation. Damaged instrumentation should not be installed as this could create a hazard to the end user. Report any damage to Meridian's Technical Support Services at 1-800-343-3858.

Alethia Package Contents

- ① Alethia Automated Isothermal Amplification and Detection System (*Catalog 610189*)
- ② Alethia Power Supply and Power Cord
- ③ Alethia Verification Standards (*Catalog 610170*)
- ④ Alethia USB Cable

External Thermal Printer Package Contents (*Catalog 610196; shipped separately*):

- ① Printer
- ② Printer Cable
- ③ Printer Power Supply (12VDC Linear Adaptor)
- ④ Thermal Roll Paper (1 Roll)

Alethia Optional Accessories

- ① External Keyboard (*Catalog 610174, shipped separately*)
- ② Handheld Barcode Scanner with stand (*Catalog 610188, shipped separately*)

Alethia Operator's Manual (*SN11042; shipped separately*)

Alethia Installation

Installation of Alethia and its accessories can be performed after contents have been inspected and the requirements described in this Operator's Manual have been reviewed. Appendix I includes a general Checklist for Set-up and Installation of Alethia. It is recommended that the Alethia Instrument is used and stored in a physically secure location to prevent unauthorized access to the device.

Component	Symbol	Installation Instruction
Printer		<p>Connect the external printer to Alethia using the supplied cabling. The printer connection is located at the back of the instrument and is identified by the symbol shown.</p> <p>Secure the printer connection and connect the power supply cable provided with the printer packaging to the printer.</p>
Power Supply	12V ---- @4.5A	<p>Connect the power supply cable provided with the Alethia packaging to the unit. The power supply connection is located at the back of the instrument. The connection port for the power supply is identified by the symbol shown. Connect the power supply cable to the power supply box.</p> <p>Plug the pronged ends of the Alethia and printer power cords to an appropriate power receptacle.</p>
External Keyboard	PS/2	<p>Connect the optional, external keyboard to the unit using the supplied cabling. The External Keyboard connection labeled PS/2 is located at the back of the instrument.</p> <p>NOTE: The Keyboard must be installed with Alethia powered off.</p>
USB Port		<p>USB connection is located at the back of the instrument and is identified by the symbol shown.</p>
Handheld Barcode Scanner	PS/2	<p>Connect the optional, handheld barcode scanner to the unit using the supplied cabling. The connection labeled PS/2 is located at the back of the instrument. For further instruction, see Appendix V.</p> <p>NOTE: The Handheld Barcode Scanner must be installed with Alethia powered off.</p>

Alethia Set-up

Instrument set-up is completed under the SYSTEM Menu. The user will be able to set the time format, date format and preferred language. The user will also be able to enable/disable the printer and onboard barcode scanner.

Alethia Performance Verification

Performance verification must be performed after installation and prior to use. Optical verification is completed according to the instructions provided under the SERVICE Menu.

Principles of Operation

Alethia is an automated isothermal amplification and detection system for target nucleic acid sequences found in human specimens. The instrument is used in conjunction with Meridian Bioscience, Inc.'s Alethia LOOP-Mediated Amplification in vitro diagnostic assays.

Alethia is a menu driven laboratory instrument with two independent sample processing blocks, identified as Block A and Block B. Sample heating and optical detection is carried out for up to five two-chambered Alethia test devices per block. Each two-chambered Alethia test device contains a Test Chamber and a Control Chamber. Amplification of target DNA occurs during the heat cycle and results in the formation of precipitate detected by the Alethia optics system. The precipitate generated by the presence of amplified target DNA leads to a turbid Test/Control reaction solution which is then measured by absorbance. Alethia uses the change in turbidity of each Test/Control reaction solution to report assay results as INVALID, POSITIVE, or NEGATIVE.

Alethia operates in four basic modes: ASSAY, RESULTS, SERVICE, and SYSTEM. Assay Selection and Sample Amplification occurs in the ASSAY mode; Test Results are managed in the RESULTS mode; basic instrument set-up is performed in the SYSTEM mode; and optical verification can be completed in the SERVICE mode.

Performance Characteristics and Specifications

Performance Characteristics

Alethia is an automated isothermal amplification and detection system for use with Meridian Bioscience, Inc.'s Alethia Loop-Mediated Amplification products. Alethia was designed with a simple User Interface which includes a keypad, liquid crystal display (LCD), onboard barcode scanner, printer and optional external keyboard. Menu-driven operating instructions are shown on the display and the user inputs commands to the instrument by making selections through the keypad.

Isothermal amplification is carried out by two independently controlled heat blocks capable of operating between 55 C and 65 C, and within 1 C of the temperature set point. The isothermal amplification temperature set point is dictated by the Alethia assay selected. Isothermal amplification time is monitored by the Alethia's internal timer. When in operation, Alethia will display block temperature and incubation time remaining on the main menu screen.

DNA amplification detection is completed by the Alethia optics system. Each Alethia block contains laser diodes illuminating at 650 ± 20 nm and corresponding detectors monitoring light transmission across each Alethia well. Alethia performs an Optics Path Verification prior to run start known as the Empty Well Test. Observed failures in the optics system disable the instrument block until the failure can be resolved. After successful Empty Well Test, Alethia checks for the presence of an Alethia test device in each well. Alethia automatically enters a Sample ID of 'EMPTY WELL' for all wells that a test device cannot be detected in. Alethia measures the absorbance of each Alethia test device at the beginning and end of the isothermal amplification incubation. Sample results are reported as INVALID, POSITIVE, or NEGATIVE based on the observed change in absorbance.

Performance features of Alethia include the following:

- Self-diagnostics are performed at power-on and at Run start.
- Self-diagnostic for Test Device Placement and Optics Path Verification.
- Lid Closure Detection and Lid Lock Feature.
- Thermal Heat Block with Optics Detection at each well with Visible Light Emission/Absorbance (650 ± 20 nm).
- Interval Timer with Real-Time Assay Processing Time Display.
- Simple LCD User Interface for Instrument Set-up, Program Selection and Sample Identification Input.
- Onboard Barcode Scanner for Sample Identification entry.
- Optional Handheld Barcode Scanner for Sample Identification entry.
- Optional External Keyboard available for Sample Identification entry.
- Attached Printer for results reporting.
- Control of amplification product carry-over through use of the self-contained Alethia Test Device.

NOTE: It is recommended that the Sample ID field is not used to record patient name or other identifiable information. The manufacturer recommends that users consider privacy requirements and regulations in their location when deciding their policy for entry of this data. The sample ID field is recorded by the instrument and the manufacturer cannot guarantee that the handling of this data is in line with all privacy related regulations in all locations.

Alethia Specifications

- **Electrical**

Voltage Range:	120 V AC
Operating Range, Supply:	100 – 240 V AC, 50/60 Hz
Voltage and Current Rating:	12 V DC, 4.5 Amp
- **Physical**

Dimensions:	21 cm x 29.2 cm x 9.5 cm
Weight:	2.95 ± 0.05 kg
- **Environmental**

Operating Temperature:	15 – 30 C
Storage Temperature:	10 – 40 C
Relative Humidity, Operating:	10 – 90%, non-condensing
Relative Humidity, Storage:	10 – 95%

Printer Specifications

- **Electrical**

Supply Voltage (SMPS):	Input voltage (S, P) 12 V DC
------------------------	------------------------------
- **Environmental**

Operating Temperature:	0 – 40 C
Storage Temperature:	-10 – 50 C
Relative Humidity, Operating:	30 – 80%
Relative Humidity, Storage:	10 – 90%
- **Paper requirements**

Thermal paper dimensions:	2 ¼" w X 80-85'
---------------------------	-----------------

Operating Instructions

Alethia operates in four basic modes: ASSAY, RESULTS, SERVICE, and SYSTEM. Assay Selection and Sample amplification occurs in the ASSAY mode; Test Results are managed in the RESULTS mode; and basic instrument set-up is performed in the SYSTEM mode. The DIAGNOSTICS option in the SERVICE mode is reserved for trained service professionals and is not accessible by the Laboratory User. General information regarding each mode of operation is provided in this section.

Keypad

The functions of Alethia are navigated through the keypad. The keypad provides a simple user interface and allows for basic menu navigation, input of alphanumeric characters for sample identification, and Assay Run initiation. Keypad functions will be referred to throughout this manual; keypad layout and symbols used are defined below.

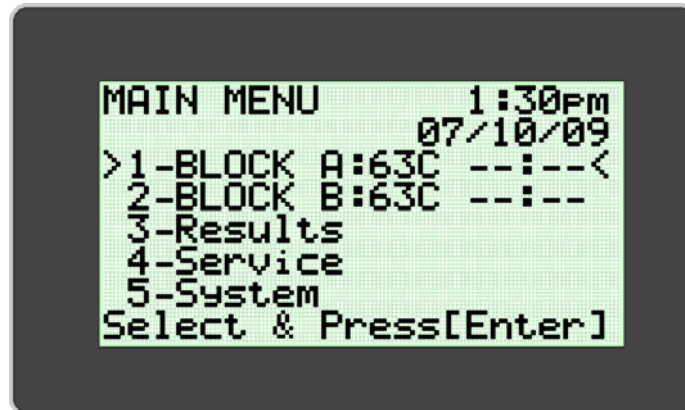
Keypad Button	Character / Function
1	1
2	A, B, C, 2
3	D, E, F, 3
4	G, H, I, 4
5	J, K, L, 5
6	M, N, O, 6
7	P, Q, R, S, 7
8	T, U, V, 8
9	W, X, Y, Z, 9
0	0
-	-
▲	Scroll Up
▼	Scroll Down
◀	Scroll Back, Backspace
↵	Enter
RUN	Run Selected Assay Protocol

NOTE: For keypad buttons with multiple characters, scroll by pressing the keypad button multiple times.

Main Menu

The Main Menu screen allows the user to view Time, Date, Block Status and Block Temperature. The Main Menu is used to access the Results, Service and System Modes.

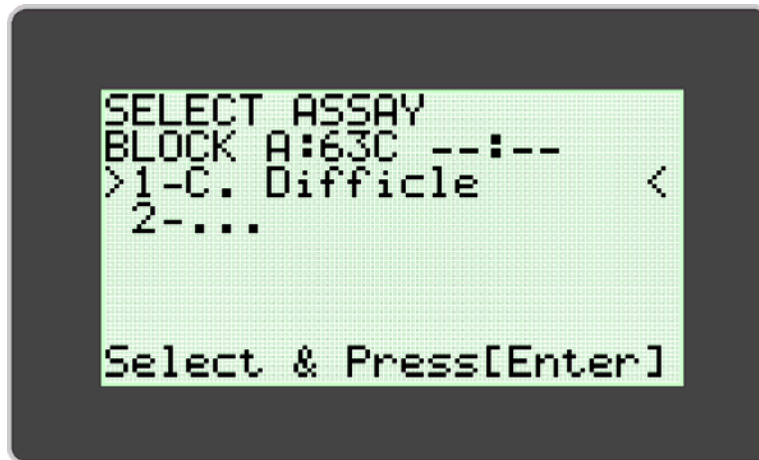
Conventions Used:

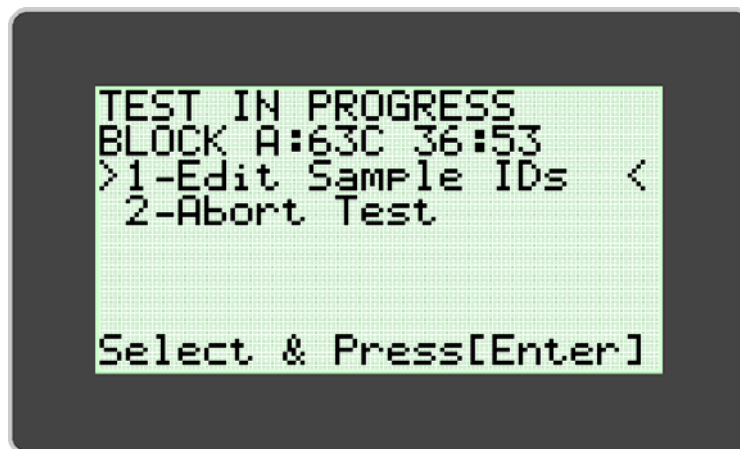
Graphic	Description
.	BLOCK STATUS: Block not at operating temperature, assay run cannot be initiated.
--:--	BLOCK STATUS: Block at operating temperature, assay run can be initiated.
!!!	Warning, Block Requires Optical Verification.
...	Assay Menu Expansion Indicator.
40C **.**	Unit has entered Power Save mode after a period of inactivity. Press any key to return unit to operating temperature.

ASSAY MODE

The ASSAY MODE allows the user to access and run programs on Alethia. The user selects the Block to be used from the Main Menu and then selects the Assay to be run. Each Block can be selected and run independently. Upon selection of the Block to be used, the Assay Mode Menu appears. The User selects the assay to be run and follows the instructions displayed on the Alethia display.

Once the run has been initiated, Alethia will prepopulate sample identification based upon the test well position, date and time that the run was initiated. The user may select the Block with the test run in progress from the main menu to edit sample identification information or to abort the test. Sample Identification information can be entered directly using the keypad, the onboard barcode scanner, optional external keyboard or optional handheld barcode scanner. Additional information regarding the Alethia onboard barcode scanner and the handheld barcode scanner are provided in Appendix IV and Appendix V of this manual respectively.

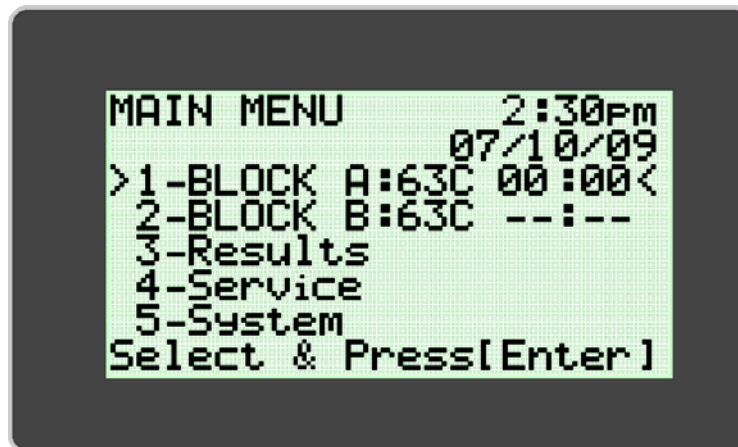
NOTE: Block temperature will appear in the 'TEMPERATURE' field. The 'Block Status' field will display a timer indicating time until assay completion.

NOTE: Sample Identification can only be edited while a test run is in progress.

Once the assay run is complete, the Block Status on the MAIN MENU will display "00:00". Results for the test run can be viewed by selecting the Block displaying completed status and following the instructions shown on the Alethia display. Results will print at this time if the Auto-Print feature is enabled. Results can be also printed manually by selecting the print option at the end of the results display.

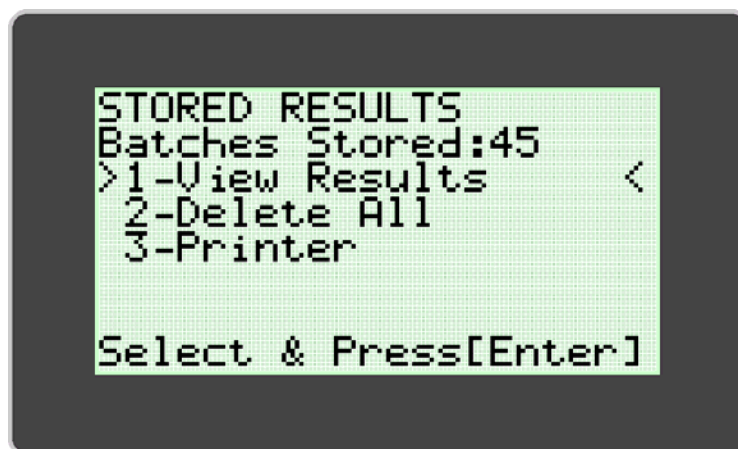
Upon completion of the assay run, the user must remove the Alethia test devices from the unit and discard appropriately.

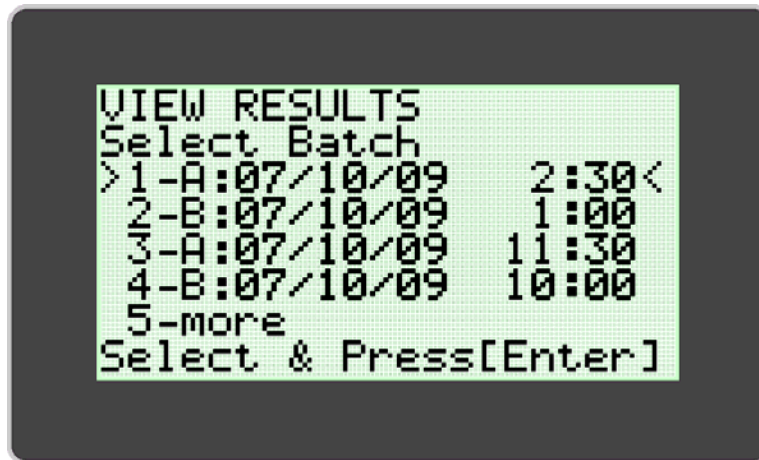
NOTE: Care must be taken to avoid contamination of the equipment and workspace by target and/or amplified nucleic acids. DO NOT open Alethia test devices after assay completion.

RESULTS MODE

The RESULTS MODE allows the user to view stored results, delete stored results and enable the auto-printer. Alethia will store up to 1000 individual test results or 200 batches. The instrument displays a warning to the user when result storage is approaching maximum capacity. Once memory capacity is reached, the instrument will begin to overwrite the oldest stored run with the newest run. RESULTS MODE is accessed by selecting Option 3 from the main menu.

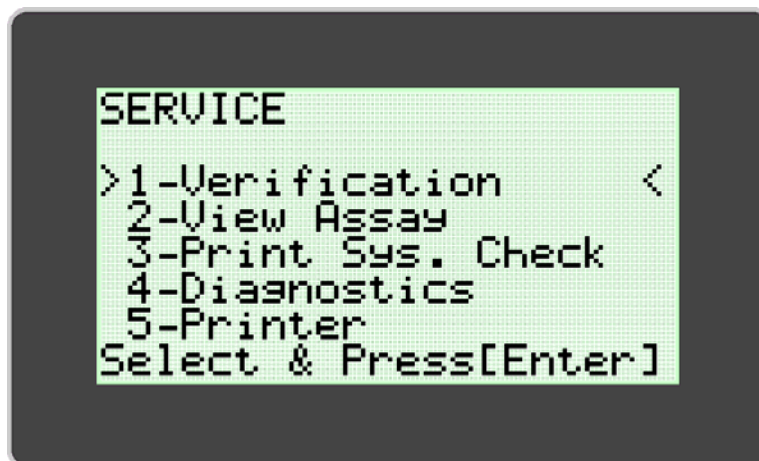
The User can view stored results by selecting "View Results" (Option 1). The 'View Results' menu allows the user to access stored data by Date and Block. Selected Batch Data is displayed and can be printed manually by selecting the print option at the end of the results display.

SERVICE MODE

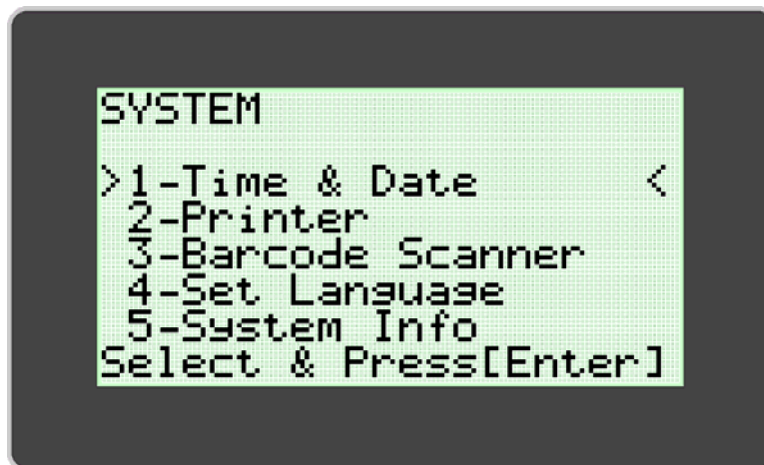
The SERVICE MODE allows the user to perform Optical Verification (Option 1), view assay parameters (Option 2), print system check information (Option 3), and configure the printer (Option 5). **NOTE:** The SERVICE MODE menu includes a 'Diagnostics' Option which can be accessed only by trained service personnel.

OPTICAL VERIFICATION is required to ensure proper function of Alethia. Instructions for completion of the Optical Verification are provided within the **CALIBRATION PROCEDURE** Section of this manual.

SYSTEM MODE

The SYSTEM MODE allows the user to set and format time and date (Option 1), enable/disable the printer (Option 2) and barcode scanner (Option 3), set user language (Option 4), and view system information (Option 5). The user may set time, date, or language (English, Italian, French, Spanish and German) based on local requirements and/or preference. The SYSTEM MODE allows the user to enable or disable automatic printing and the barcode scanner. System configuration is completed by following the instructions shown on the Alethia display.

NOTE: *Time and Date cannot be modified while a test is in process. Language cannot be modified while a test is in process.*

Calibration Procedures

POWER-ON SELF TEST

Alethia automatically performs an internal **Power-On Self Test (POST)** each time the instrument is powered-on. The POST Test confirms that software and hardware components of the system are performing as expected. Successful completion of the POST Test will be followed by an audible tone. POST Test failure will be indicated by an Error Code. Additional information regarding Error Codes is located in Appendix III of this manual.

OPTICAL VERIFICATION

Calibration of Alethia is not required. Verification of the OPTICS SYSTEM for each Block must be performed every 30 days to ensure proper function. User will be prompted by “!!!” on the main menu when optical verification is required. Verification is completed using the Verification Standards included with Alethia.

Optical Verification includes two stages. The first stage, known as the EMPTY WELL TEST, ensures that the optical path is clear and free of obstruction. The second stage confirms that the optics system is transmitting and detecting emitted light particles properly. Transmission and detection verification requires the use of the Alethia Verification Standards. Alethia steps through the Optical Verification protocol and prompts the user for action as necessary.

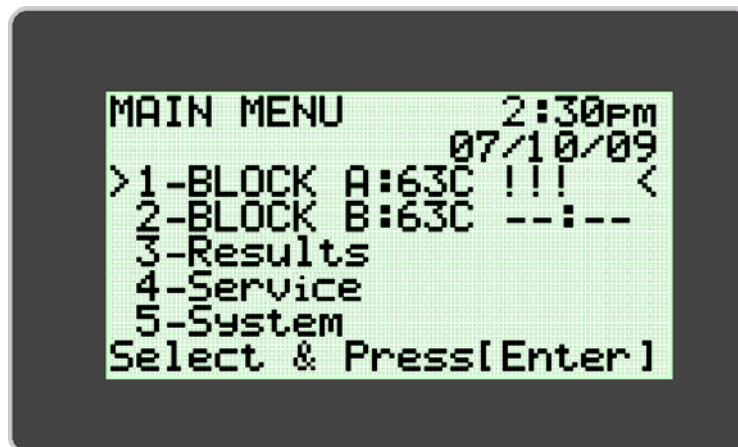
Once the Optical Verification has been initiated, users will be instructed to remove all sample tubes so that the instrument can perform the first stage of optical verification (EMPTY WELL TEST).

After the first stage is complete, the instrument will proceed to the second stage. The user will be instructed to insert Verification Standards. The Verification Standards must be seated firmly in the Heat Block wells. The Verification Standards should be oriented with the serial number tag at the Well 1 position.

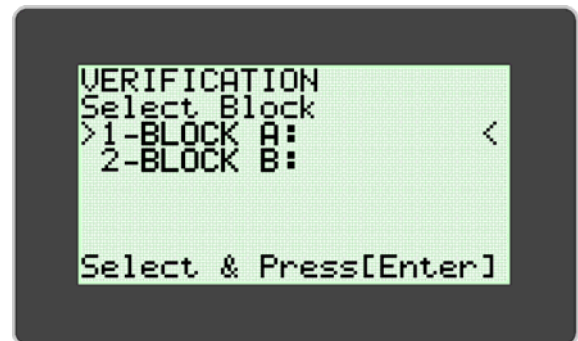
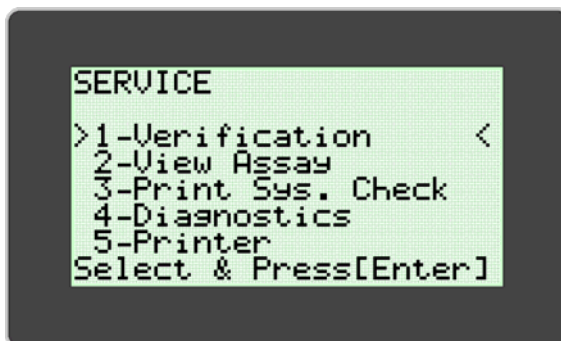
Upon completion of the Optical Verification, Alethia will display verification test results as either 'Pass' or 'Fail'. In the event that Optical Verification testing does not give acceptable results, the user should verify the optical path is free of obstructions, the verification standards are free of visible defects and repeat the verification testing. If repeat testing does not give passing verification results, contact Meridian's Technical Support Services for further assistance.

Performing optical verification:

1. Optical Verification must be performed every 30 days using the provided Verification Standards.
2. When verification is due, the unit will display "!!!" on the Block Status portion on the Main Menu.
3. Press the number on the keypad corresponding to the block requiring verification and follow the on screen prompts to complete verification.

4. To perform Optical Verification when "!!!" is not displayed on the Main Menu, access the SERVICE menu and select the VERIFICATION option. Proceed by following the onscreen prompts.

NOTE: Each Block of the Alethia functions independently. For example, a failed verification test for Block A will not prevent use of Block B when Block B verification testing passes.

NOTE: In the event that the Optical Verification fails, remove Verification Standards from the Alethia instrument. Clean standards with a lint free tissue or lens paper and repeat Optical Verification. Contact Meridian's Technical Support Services at (800)343-3858 if issue persists.

WARNING: *Do NOT leave Verification Standards in Alethia. Verification Standards will get hot and could burn the user. Prolonged exposure to elevated temperatures may impact the performance of the Verification Standards.*

Operational Precautions and Limitations

WARNINGS:

	<p>CAUTION: Risk of Danger. Alethia is an electromechanical device that can cause physical shock or injury to the operator if not used in accordance with the procedures described in this manual.</p>
	<p>LASER RADIATION: Avoid Exposure to Beam. Alethia contains a class 3R laser product. The laser will not function when the lid is in the open position. However, care should be taken in the handling and use of this instrument.</p>
	<p>HOT SURFACE: Keep hands Away from Hot Surfaces. Alethia contains a heat block producing temperatures between 55 – 65 C during operation. Care should be taken to avoid direct contact with the heat block.</p>
	<p>CAUTION: Laser Radiation. Alethia contains a laser product. Service of the unit should be performed only by qualified personnel as optical exposure to the laser beam may cause injury.</p>
<p>IPX-0</p>	<p>CAUTION: Protect from water. Alethia does not have protection from the ingress of water. Do not expose to water or submerge the instrument.</p>

PRECAUTIONS:

- Alethia is an automated instrument that utilizes isothermal Loop-Mediated Amplification Technology. Care must be taken to avoid contamination of the equipment and workspace by target and/or amplified nucleic acids. Only qualified personnel should perform molecular testing.
- Alethia is used in conjunction with Meridian Bioscience, Inc.'s Alethia Loop-Mediated Amplification in vitro diagnostic assays. Samples processed in the Alethia should be handled in accordance with the instructions provided in the specific Alethia assay's instructions for use.
- Selected Language cannot be changed while a run is in process.
- Time and Date cannot be changed while a run is in process
- Verification Standards must be stored in the case provided to protect from light and damage. Verification Standards should not be stored in the Alethia.
- The optional keyboard and handheld barcode scanner must be plugged in while Alethia is powered-off. CAPS Lock light will come on at power-up to indicate keyboard is online and handheld scanner will chime.
- When not in use, Alethia should be stored with the lids closed.

Service and Maintenance

Servicing of Alethia should be performed by qualified professionals only. Contact Meridian Bioscience Technical Support at (800) 343-3858 for technical support or to make arrangements for service. DO NOT ATTEMPT TO SERVICE ALETHIA.

Surface Cleaning

Cleaning of the exterior surfaces of Alethia and the immediate work area should be performed as necessary, no less than daily when in use.

1. Power off unit and disconnect from power source.
2. Moisten a lint-free cloth with 10% bleach solution or DNase/RNase cleaning solution.
3. Wipe outer surfaces of instrument with a moistened lint free cloth.
4. Open the instrument lids and, if available, insert test well covers. Wipe interior surfaces with same moistened cloth. Care should be taken not to allow cleaning solution to enter the test wells.

Note: Avoid cleaning the latch area when cleaning the inside surfaces of the instrument.

WARNING: Surface cleaning should be performed only when the instrument is powered-off AND disconnected from the power source. DO NOT use saturated clothes for cleaning. DO NOT spray the instrument directly. DO NOT clean the test wells with compressed air.

DO NOT spray the Alethia instrument directly with any solution

DO NOT use oversaturated cloths or oversaturated foam swabs for cleaning

DO NOT clean or wipe over latch areas

DO NOT attempt to clean the Alethia instrument using compressed air

Test Well Cleaning

Cleaning of the instrument test wells should be performed by qualified personnel only. Cleaning of the test wells should be performed only when contamination of the instrument is suspected. Contamination of the instrument may be non-DNA Sourced, or DNA Sourced.

Note: Do not clean test wells with fibrous swabs (e.g., Cotton, Rayon, Dacron, Calcium Alginate or Polyester Swabs).

The cleaning protocol followed should be based on the source of the contamination as shown below:

- **non-DNA Contamination Cleaning Protocol**

Note: Non-DNA cleaning should be performed when it is suspected that the optics are compromised by foreign material. This may or may not be evidenced by a failing optical verification.

1. Power off unit and disconnect from power source, allowing unit to cool to room temperature.
 2. Gently wipe heat block chamber with a dry Foam or Flocked Nylon Swab.
 3. Gently wipe heat block chamber with a Foam or Flocked Nylon Swab moistened with 70% Alcohol.
 4. Gently wipe heat block chamber with a dry Foam or Flocked Nylon Swab.
- DO NOT use oversaturated foam swabs for cleaning.**

- **DNA Contamination Cleaning Protocol**

Note: DNA cleaning should be performed when it is suspected that the instrument is contaminated with DNA (e.g., amplicon or patient specimen).

1. Power off unit and disconnect from power source, allowing unit to cool to room temperature.
 2. Gently wipe heat block chamber with a dry Foam or Flocked Nylon Swab.
 3. Gently wipe heat block chamber with a Foam or Flocked Nylon Swab moistened with 10% Bleach Solution.
 4. Gently wipe heat block chamber with a dry Foam or Flocked Nylon Swab.
 5. Gently wipe heat block chamber with a Foam or Flocked Nylon Swab moistened with 70% Alcohol.
 6. Gently wipe heat block chamber with a dry Foam Swab.
- DO NOT use oversaturated Foam Swabs for cleaning.**

WARNING: Test Well cleaning should be performed only when the instrument is powered-off AND disconnected from the power source. DO NOT use oversaturated swabs for cleaning. Instrument should be fully cooled prior to cleaning Test Wells.

WARNING: Do not attempt to clean Alethia using compressed air.

NOTE: Optical Verification must be performed after cleaning Test Wells.

This page intentionally left blank.

Appendix I

Alethia Set-Up and Installation Checklist

Facility Information				
Facility Name:				
Facility Address:		(Street Address)		
		(City, State)		
Instrument Information				
Instrument Serial Number:				
Installation Location:		(Room, Floor, Building, et cetera)		
Installation Performed By:		Installation Date:		
Set-Up and Installation				
1.	The Alethia shipping container and packaging have been inspected for damage; no damage found. Installation of damaged could create a hazard to the end user.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
2.	The Alethia Operator's Manual has been received and reviewed. Operational Precautions and Limitations have been reviewed and are understood.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
3.	The Installation Location meets Electrical and Environmental Specifications described in the Alethia Operator's Manual.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
4.	Installation of the Alethia and its accessories has been completed as described by the Operator's Manual.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
5.	The Alethia has been powered-on and the Power On Self-Test was successful.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
6.	Set-up of Alethia including Time Format, Date Format, and Preferred Language has been completed as described in the Operator's Manual.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
7.	Alethia Performance Verification was completed as described in the Operator's Manual. Acceptable results were obtained for Block A and Block B.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
8.	A copy of the printout for the System Check and Optics System Verification is attached to this record.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	N/A <input type="checkbox"/>
Installation Complete				
(Signature/Date)				

This page intentionally left blank.

Appendix II

Alethia Decontamination Record

Decontamination of laboratory equipment must be completed prior to return of the instrument for servicing. Decontamination must be completed according to the instructions provided. Please complete the information below and attach the completed record to the instrument prior to return.

Instruments returned for service without evidence of decontamination will not be serviced. If the instrument is submitted for service without decontamination documentation, it will be shipped back to the user at the user's expense.

1. Wipe exterior surfaces of the Alethia with a lint-free towel moistened with a 10% Bleach-based Disinfecting Solution. Minimum contact exposure time for kill of Bloodborne Pathogens (Hepatitis-B, Hepatitis-C, HIV, MRSA, et cetera) is 1 minute.
2. Open the Alethia Lids A and B. Wipe interior surfaces with lint-free towel moistened with 10% Bleach-based Disinfecting Solution. Minimum contact exposure time is 1 minute.
3. Close Lids. Affix the completed Alethia Decontamination Record to the top of the instrument. Package the instrument in the protective foam packaging and shipping container provided.

Instrument Decontamination Information	
Instrument Serial Number:	
Facility Name:	
Facility Address:	(Street Address)
	(City, State)
Contact Information:	(Name, Title)
	(Telephone)
	(email)
Performed By:	(Signature/Date)

Appendix III

Alethia Error Codes and Troubleshooting

The Alethia will display Error Codes when system failures are detected. Error Codes, descriptions and the actions recommended for correction are summarized in the table below.

Error Code	Description	Action Required
500	Real time clock not working.	Power-off instrument and unplug for 10 minutes, allowing it to cool. Plug unit back in and turn instrument back on. Allow POST Testing to be completed.
501	Non-Volatile RAM (NVRAM) not working.	
504	Voltage out of range.	
505	Voltage out of range.	
506	Voltage out of range.	
507	Voltage out of range.	
508	Interior temperature is too hot.	Move unit to a cooler environment. Plug unit back in and turn instrument back on. Allow POST Testing to be completed.
509	Block A temperature is too hot.	Turn unit off and wait for it to cool. Plug unit back in and turn instrument back on. Allow POST Testing to be completed.
510	Block B temperature is too hot.	
502	Firmware Error: Heater block is not heating properly.	Power-off instrument and unplug for 10 minutes, allowing it to cool. Plug unit back in and turn instrument back on. Allow POST Testing to be completed.
503	Firmware Error: Non-Volatile RAM (NVRAM) did not initialize properly.	
511	Firmware Error: Test results were not fetched from Non-Volatile RAM (NVRAM) properly.	
512	Firmware Error: Device parameters were not fetched from Non-Volatile RAM (NVRAM) properly.	
515	Invalid block number status check.	
516	Real Time clock read reported error.	
517	Block A did not return a valid temperature reading.	
518	Block B did not return a valid temperature reading.	
519	Initial Optic read on Block A did not report valid status.	
520	Initial Optic read on Block B did not report valid status.	

521	Interval Optic read on Block A did not report valid status.	<p>Power-off instrument and unplug for 10 minutes, allowing it to cool. Plug unit back in and turn instrument back on. Allow POST Testing to be completed.</p>
522	Interval Optic read on Block B did not report valid status.	
523	Final Optic read on Block A did not report valid status.	
524	Final Optic read on Block B did not report valid status.	
170.XXX	Temperature regulation error.	
171.XXX	Temperature regulation error.	

Contact Meridian Bioscience Technical Support Services at (800)343-3858 if issues persist.

Appendix IV

Alethia Onboard Barcode Scanner Information

Alethia allows for entry of Sample identification using barcodes. The onboard barcode scanner has been placed on front of the instrument. The scanner is oriented vertically, next to the keypad.

Use of the onboard barcode scanner:

1. With a run in progress, select the test well position corresponding to specimen to be identified.
2. The barcode scanner will turn on indicating it is ready for ID entry.
3. Ensure the barcode label is held vertically and free from printing defects.
4. Hold barcode perpendicular to bench top and 1-2 inches away from barcode scanner, as depicted below.
5. Scan barcode from left to right in front of scanner.
6. Repeat process for all sample ID's to be entered.

Sample Identification information is limited to 16 characters. Barcodes containing only the following characters will be accepted:

A	B	C	D	E	F	G	H	I	J	K	L	M
N	O	P	Q	R	S	T	U	V	W	X	Y	Z
0	1	2	3	4	5	6	7	8	9	-		

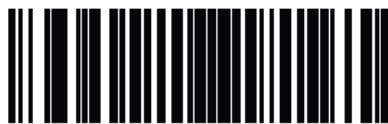
Appendix V

Alethia Handheld Barcode Scanner Intended Use and Function

The Alethia Handheld Barcode Scanner is intended for use with the Alethia automated isothermal amplification and detection system. The Handheld Barcode Scanner is supplied with a preinstalled Y connector cord and an optional stand.

Installation

1. Power off the Alethia instrument. Insert male end of Y connector cord to the port marked "PS/2" on the back of the Alethia instrument. If User would like to install both the Handheld Barcode Scanner and the optional keyboard, optional keyboard can be connected to the female end of the Y connector cord.
2. Power on the instrument.
3. Disable the onboard barcode scanner. From the Main Menu, enter the SYSTEM mode (Option 5) and select "Barcode Scanner" (Option 3). Disable the onboard scanner by pressing "Disable" (Option 2).
4. If the handheld barcode scanner is placed in the optional stand, the scanner will automatically enter Constant Read (Hands-Free (Presentation) Trigger) Mode. This can be toggled off and on by scanning the following barcodes:

***Enable Hands-Free (Presentation) Trigger Mode
(1)**

**Disable Hands-Free (Presentation) Trigger Mode
(0)**

5. The handheld barcode scanner will beep when a barcode is successfully read. The volume of the beep can be adjusted by scanning the following barcodes:

**Low Volume
(2)**

**Medium Volume
(1)**

***High Volume
(0)**

Use of the handheld barcode scanner:

1. With a run in progress, select the test well position using the keypad corresponding to specimen to be identified.
2. Aim the scanner at the barcode
3. Pull trigger if Hands-Free (Presentation) Trigger Mode is disabled.
4. Repeat process for all sample ID's to be entered.

Appendix VI

Alethia Compliance Testing Summary

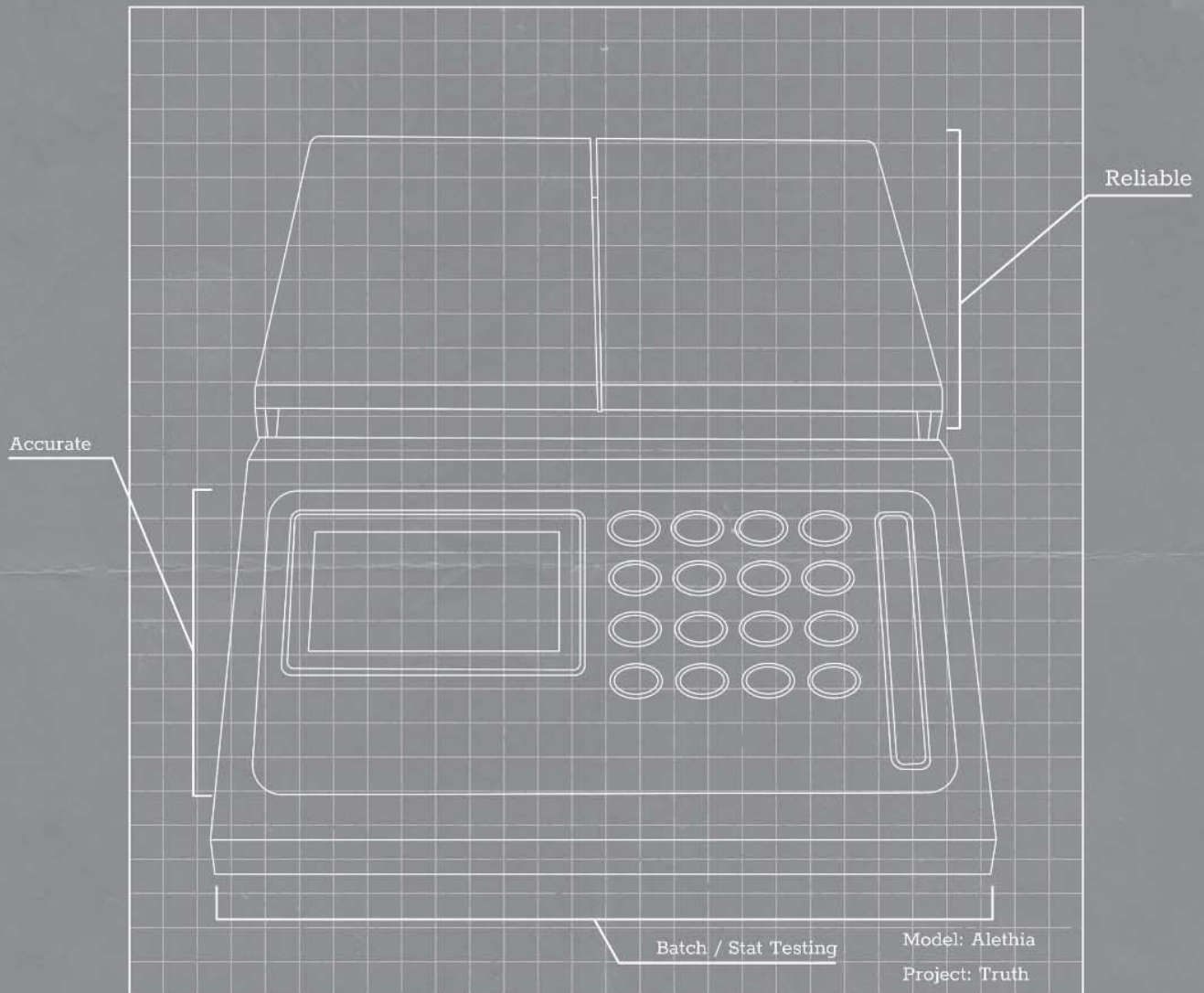
	<p>Alethia has been tested and found to be in compliance with the following requirements and Standards:</p>
---	---

Standard	Description
IEC 61010-1	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 1: General requirements
IEC 61010-2-010	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-010: Particular requirements for laboratory equipment for the heating of materials
IEC 61010-2-081	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-081: Particular requirements for automatic and semi-automatic laboratory equipment for analysis and other purposes
IEC 61010-2-101	Safety requirements for electrical equipment for measurement, control and laboratory use - Part 2-101: Particular requirements for in vitro diagnostic (IVD) medical equipment
UL 61010-1	UL standard for Safety Electrical Equipment for Measurement, Control, and Laboratory Use - Part 1: General Requirements
CSA C22.2#61010-1	UL standard for Safety Electrical Equipment for Measurement, Control, and Laboratory Use - Part 1: General Requirements
CSA C22.2#61010-2-010	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-010: Particular requirements for laboratory equipment for the heating of materials
CSA C22.2#61010-2-081	Safety Requirements for Electrical Equipment for Measurement, Control and Laboratory Use - Part 2-081: Particular Requirements for Automatic and Semi-Automatic Laboratory Equipment for Analysis and Other Purposes
CSA C22.2#61010-2-101	Safety Requirements for Electrical Equipment for Measurement, Control, and Laboratory use - Part 2-101: Particular Requirements for In Vitro Diagnostic (IVD) Medical Equipment

This page intentionally left blank.

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Standard	Description
IEC 61010-1	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 1: General requirements
IEC 61010-2-010	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-010: Particular requirements for laboratory equipment for the heating of materials
IEC 61010-2-081	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-081: Particular requirements for automatic and semi-automatic laboratory equipment for analysis and other purposes
IEC 61010-2-101	Safety requirements for electrical equipment for measurement, control and laboratory use - Part 2-101: Particular requirements for in vitro diagnostic (IVD) medical equipment
UL 61010-1	UL standard for Safety Electrical Equipment for Measurement, Control, and Laboratory Use - Part 1: General Requirements
CSA C22.2# 61010-1	UL standard for Safety Electrical Equipment for Measurement, Control, and Laboratory Use - Part 1: General Requirements
CSA C22.2#61010-2-010	Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-010: Particular requirements for laboratory equipment for the heating of materials
CSA C22.2#61010-2-081	Safety Requirements for Electrical Equipment for Measurement, Control and Laboratory Use - Part 2-081: Particular Requirements for Automatic and Semi-Automatic Laboratory Equipment for Analysis and Other Purposes
CSA C22.2#61010-2-101	Safety Requirements for Electrical Equipment for Measurement, Control, and Laboratory use - Part 2-101: Particular Requirements for In Vitro Diagnostic (IVD) Medical Equipment

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