# SZABO SCANDIC

Part of Europa Biosite

## Produktinformation

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[mail@szabo-scandic.com](mailto:mail@szabo-scandic.com)

[www.szabo-scandic.com](http://www.szabo-scandic.com)

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

# A CLOSED SYSTEM THAT LIMITS EXPOSURE DURING TEST INCUBATION AND DISPOSAL

## EXCEED YOUR CURRENT RSV TESTING PERFORMANCE

- TRU RSV® uses pooled monoclonal antibodies detecting two RSV proteins which may help detect more positive samples

## EASE OF USE

- Simple procedure with easy interpretation of results

## DEFINITIVE ANSWERS, CONFIDENT RESULTS


- TRU RSV® provides definitive answers to ensure patients receive the appropriate therapy as soon as possible

## TEST PROCEDURE

### How to perform the test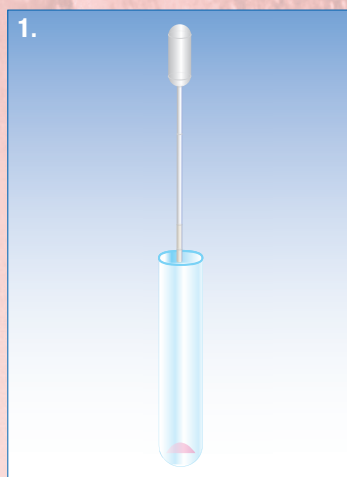


#### Liquid Specimens

Add 100 uL of Sample Diluent (first mark) to the tube. Mix well.


#### Dry Swabs

Add 300uL of Sample Diluent (third mark) to the tube. Mix well.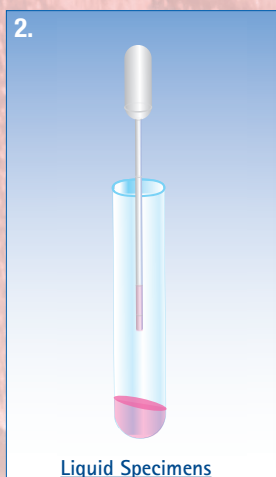

#### Liquid Specimens

Draw Sample to first mark (100 uL) on transfer pipette and add to tube. Mix by squeezing pipette bulb 3 times or vortex for 10 seconds.

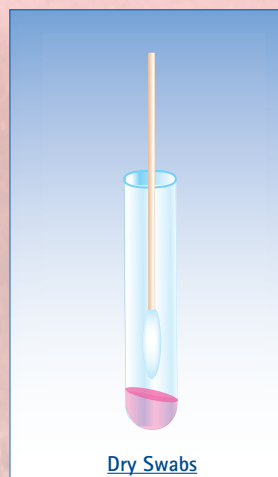
#### Dry Swabs

Dip swab, rotate 3X, press against side and remove.

Add Test Strip to tube and secure in place, incubate at 20-25 C for 15 minutes. Read results within 1 minute.

### RESULTS FOR TRU RSV®

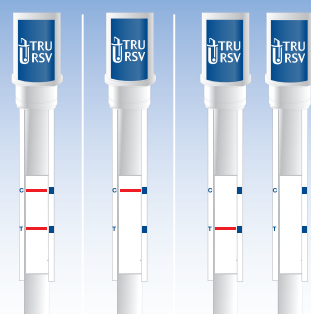Positive  
(+)

Negative  
(-)

Invalid

\*This chart does not contain complete instructions for use. For further information, please refer to the package insert.

### Ordering Information

Product	Quantity	Catalog #
TRU RSV®	32 Tests	751330
FLU/RSV Positive Control	3 Vials of 8 Tests each	751110
Viral Transports	100 Swab Packs	505021

For more information in the US, contact a specialist at 1-888-763-6769.

REFERENCES  
Copyright Dennis Kunkel Microscopy, Inc.

#### WORLD HEADQUARTERS

3471 River Hills Drive  
Cincinnati, Ohio 45244  
Telephone: 1-513-271-3700  
Orders/Customer Service: 1-800-543-1980  
Ordering Fax: 1-513-271-0124  
Email: mbi@meridianbioscience.com

#### Italy

Via dell'Industria, 7  
20020 Villa Cortese, Milano  
Italy  
Telephone: + 39 0331 43 36 36  
Fax: + 39 0331 43 36 16  
Email: info@meridianbioscience.eu

#### Belgium

Rue de l'Industrie 7  
1400 Nivelles  
Belgium  
Telephone: + 32 67 89 59 59  
Fax: + 32 67 89 59 58  
Email: info.bnl@meridianbioscience.eu

#### France

34 Rue de Ponthieu  
Paris 75008  
France  
Telephone: +33 (0)1 42 56 04 40  
Fax: +33 (0)9 70 06 62 10  
Email: info.fr@meridianbioscience.eu

#### The Netherlands

Halderheiweg 6  
5282 SN Boxtel  
The Netherlands  
Telephone: + 31 411 62 11 66  
Fax: + 31 411 62 48 41  
Email: info.bnl@meridianbioscience.eu

#### Australia

Bioline Australia  
Suite G16A, NIC Building  
Australian Technology Park  
Eveleigh NSW 2015  
Telephone: +61 (0)2 9209 4180  
Fax: +61 (0)2 9209 4763  
Email: info.aust@bioline.com

 **Meridian**  
**Bioscience, Inc.**  
Inspired Science. Trusted Solutions.®