SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

High Performance Urine Antigen Testing **TRU LEGIONELLA™**

EXCEED YOUR CURRENT TESTING PERFORMANCE IN THE DETECTION OF *LEGIONELLA* SG 1

- A method of choice as reported by the W.H.O.¹ and complies with CDC recommendations for *Legionella pneumophila* sg 1 detection²
- High performance identification provides a new standard for patient care
- Closed system that limits exposure during test incubation and disposal

SIMPLIFIED WORKFLOW

- Test outcomes in 20 minutes
- Easy procedure with simple result interpretation

DEFINITIVE ANSWERS, CONFIDENT RESULTS

- Ensures optimal patient management for proper antibiotic therapy

References




1. World Health Organization 2007: *Legionella* and Prevention of Legionellosis. Geneva, Switzerland: WHO
2. Top 10 Things Every Clinician Needs to Know About Legionellosis (2011). <http://www.cdc.gov/legionella/top10.htm>

POSITIVE AGREEMENT	NEGATIVE AGREEMENT
95.5%	100%


TEST PROCEDURE

How to perform the test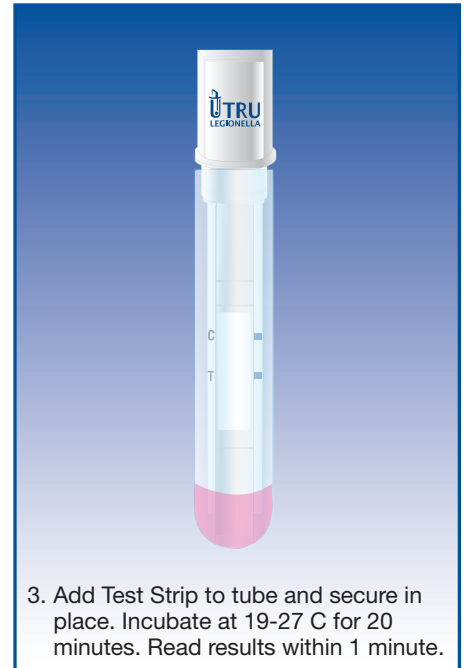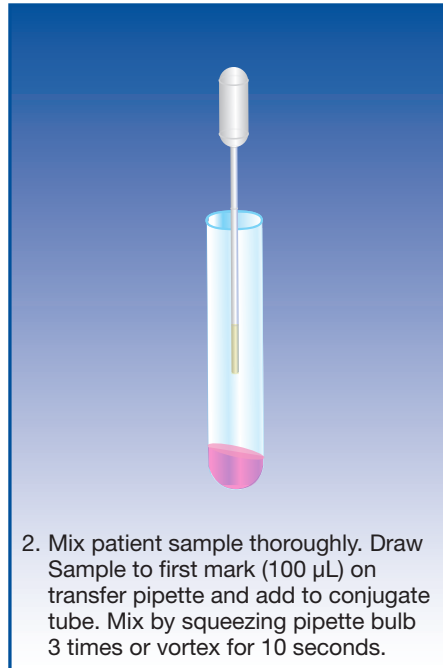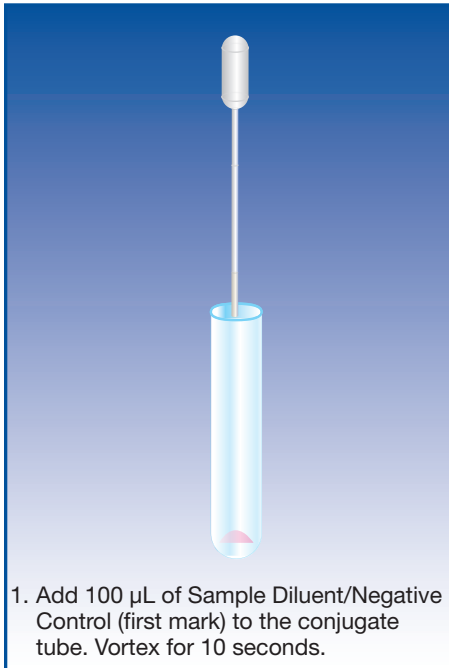

RESULTS

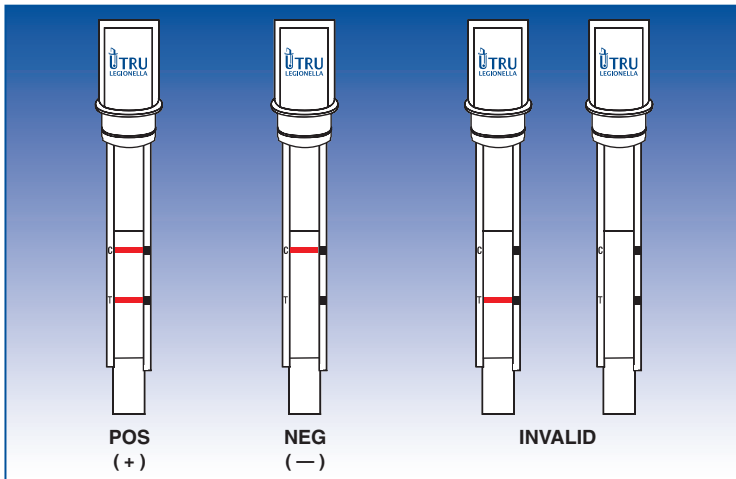EXTERNAL CONTROLS TEST:

1. Positive Control: Add exactly 5 drops of the Positive Control reagent to the Conjugate Tube marked for the Positive Control.
2. Sample Diluent/Negative Control: Add 200 µL (second mark) to the Conjugate Tube marked for the Negative Control.
3. Vortex or swirl the contents of the tubes for 10 seconds.
4. Insert the narrow end of a Test Strip to each Conjugate Tube and firmly press down on the caps to close each tube. Incubate both tubes at 19-27 C for 20 minutes.

NOTE: Please refer to the package insert for complete instructions.

Please refer to the most current version of the package insert for complete instructions.

Ordering Information

Product	Quantity	Catalog #
TRU Legionella™	32 Tests	751930

For more information, contact a specialist at info@meridianbioscience.eu

MERIDIAN BIOSCIENCE EUROPE

Italy – Milan Tel: +39 0331 43 36 36
 France – Paris Tel: +33 1 42 56 04 40
 Belgium – Nivelles Tel: +32 67 89 59 59
 The Netherlands – Boxtel Tel: +31 411 62 11 66
 Web: www.meridianbioscience.eu
 Email: info@meridianbioscience.eu

WORLD HEADQUARTERS

Meridian Bioscience, Inc.
 3471 River Hills Drive
 Cincinnati, Ohio 45244 USA
 Web: www.meridianbioscience.com

AUSTRALIA

Bioline (Aust) Pty Ltd
 Tel: +61 (0)2 9209 4180
 Email: info.aust@bioline.com