**SZABO
SCANDIC**

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

linkedin.com/company/szaboscandic

NovaLisa®

Schistosoma mansoni IgM

ELISA

CE

Only for in-vitro diagnostic use

English2
Deutsch.....	7
Français	12
Italiano	17
Español	22
Português.....	27
Bibliography / Literatur / Bibliographie / Bibliografia / Bibliografía / Bibliografia	34
Abbreviations / Abkürzungen / Abréviations / Abbreviazioni / Abreviaciones / Abreviaturas	34
Symbols Key / Symbolschlüssel / Explication des Symboles / Legenda / Símbolos / Tabela de símbolos.....	35
Summary of Test Procedure / Kurzanleitung Testdurchführung / Résumé de la procedure de test / Schema della procedura / Resumen de la técnica / Resumo do Procedimento de Teste	36

ENGLISH

1. INTRODUCTION

Schistosomes belong to the class of distomas (trematodes). They rank among the most frequent pathogens. Estimations originate in more than 200 million affected people. The mature parasites are 6 – 22 mm long. The most important species are Schistosoma mansoni, S. japonicum and S. haematobium. Schistosoma mansoni is common in Africa, South America and Middle East.

Schistosomiasis (bilharziosis) is – depending on species and location of the parasites – a disease of the intestine, liver and spleen resp. urinary passages. Humans are (re)infected by contact with fresh water which is contaminated by ova containing urine or faeces. If larvae bore into human skin, first a transient skin reaction appears (itch with exanthema or erythema, by repeatedly infection cercarial dermatitis is possible). After 3 - 10 weeks the meanwhile sexually mature worms synthesize cytotoxic and allergic substances which cause feverish reaction in humans (Katayama fever). The infected person is mostly harmed by the eggs, which get into organs via blood excreting proteins and glycoproteins. The person reacts under participation of own antibodies and immune complexes with formation of granuloma and granulomatous proliferation in intestine and urinary bladder mucosa. Not excreted eggs die after 3 weeks and will be dissolved or calcified. The affected tissue gets fibrous.

Species	Disease	Symptoms (e.g.)	Transmission route
Schistosoma mansoni	Schistosomiasis, also known as snail fever	Penetration phase: petechial-like dermatitis (Pruritus, erythema, maculopapular rash). Acute phase (Katayama fever): chills, headache, cough, urticarial; especially with Schistosoma mansoni: anaphylactic shock (Katayama syndrome). Chronic phase: chronic granulomatous inflammation, liver fibrosis, portal hypertension, obstructions in the urogenital area and the portal and pulmonary circulation.	Percutaneous

The presence of pathogen or infection may be identified by:

- Microscopy
- Serology: e.g. ELISA

2. INTENDED USE

The Schistosoma mansoni IgM ELISA is intended for the qualitative determination of IgM antibodies against Schistosoma mansoni in human serum or plasma (citrate or heparin).

3. PRINCIPLE OF THE ASSAY

The qualitative immunoenzymatic determination of specific antibodies is based on the ELISA (Enzyme-linked Immunosorbent Assay) technique.

Microplates are coated with specific antigens to bind corresponding antibodies of the sample. After washing the wells to remove all unbound sample material a horseradish peroxidase (HRP) labelled conjugate is added. This conjugate binds to the captured antibodies. In a second washing step unbound conjugate is removed. The immune complex formed by the bound conjugate is visualized by adding Tetramethylbenzidine (TMB) substrate which gives a blue reaction product.

The intensity of this product is proportional to the amount of specific antibodies in the sample. Sulphuric acid is added to stop the reaction. This produces a yellow endpoint colour. Absorbance at 450/620 nm is read using an ELISA microwell plate reader.

4. MATERIALS

4.1. Reagents supplied

- **Schistosoma mansoni Coated Microplate (IgM):** 12 breakapart 8-well snap-off strips coated with Schistosoma mansoni antigens in resealable aluminium foil.
- **IgM Sample Diluent:** 1 bottle containing 100 ml of phosphate buffer (10 mM) for sample dilution; pH 7.2 ± 0.2; anti-human IgG (RF Absorbent); coloured green; ready to use; white cap.
- **Stop Solution:** 1 bottle containing 15 ml sulphuric acid, 0.2 mol/l; ready to use; red cap.
- **Washing Buffer (20x conc.):** 1 bottle containing 50 ml of a 20-fold concentrated phosphate buffer (0.2 M); pH 7.2 ± 0.2; for washing the wells; white cap.
- **Schistosoma mansoni anti-IgM Conjugate:** 1 bottle containing 20 ml of peroxidase labelled antibody to human IgM; in phosphate buffer (10 mM); coloured red, ready to use; black cap.
- **TMB Substrate Solution:** 1 bottle containing 15 ml 3,3',5,5'-tetramethylbenzidine (TMB), < 0.1%; ready to use; yellow cap; <5% NMP.
- **Schistosoma mansoni IgM Positive Control:** 1 vial containing 2 ml control (human serum or plasma); coloured yellow; ready to use; red cap.
- **Schistosoma mansoni IgM Cut-off Control:** 1 vial containing 3 ml control (human serum or plasma); coloured yellow; ready to use; green cap.
- **Schistosoma mansoni IgM Negative Control:** 1 vial containing 2 ml control (human serum or plasma); coloured yellow; ready to use; blue cap.

For potential hazardous substances please check the safety data sheet.

4.2. Materials supplied

- 1 Cover foil
- 1 Instruction for use (IFU)
- 1 Plate layout

4.3. Materials and Equipment needed

- ELISA microwell plate reader, equipped for the measurement of absorbance at 450/620 nm
- Incubator 37 °C
- Manual or automatic equipment for rinsing wells
- Pipettes to deliver volumes between 10 and 1000 µl
- Vortex tube mixer
- Distilled water
- Disposable tubes

5. STABILITY AND STORAGE

Store the kit at 2...8 °C. The opened reagents are stable up to the expiry date stated on the label when stored at 2...8 °C.

6. REAGENT PREPARATION

It is very important to bring all reagents and samples to room temperature (20...25 °C) and mix them before starting the test run!

6.1. Coated Microplate

The break-apart snap-off strips are coated with Schistosoma mansoni antigens. Immediately after removal of the strips, the remaining strips should be resealed in the aluminium foil along with the desiccant supplied and stored at 2...8 °C.

6.2. Washing Buffer (20x conc.)

Dilute Washing Buffer 1 + 19; e. g. 10 ml Washing Buffer + 190 ml distilled water. The diluted buffer is stable for 5 days at room temperature (20...25 °C). In case crystals appear in the concentrate, warm up the solution to 37°C e.g. in a water bath. Mix well before dilution.

6.3. TMB Substrate Solution

The reagent is ready to use and has to be stored at 2...8 °C, away from the light. The solution should be colourless or could have a slight blue tinge. If the substrate turns into blue, it may have become contaminated and should be thrown away.

7. SAMPLE COLLECTION AND PREPARATION

Use human serum or plasma (citrate or heparin) samples with this assay. For CSF please use the instruction for use ABVL0001. If the assay is performed within 5 days after sample collection, the samples should be kept at 2...8 °C; otherwise they should be aliquoted and stored deep-frozen (-70...-20 °C). If samples are stored frozen, mix thawed samples well before testing. Avoid repeated freezing and thawing.

Heat inactivation of samples is not recommended.

7.1. Sample Dilution

Before assaying, all samples should be diluted 1+100 with IgM Sample Diluent. Dispense 10 µl sample and 1 ml Sample Diluent into tubes to obtain a 1+100 dilution and thoroughly mix with a Vortex.

8. ASSAY PROCEDURE

8.1. Test Preparation

Please read the instruction for use carefully before performing the assay. Result reliability depends on strict adherence to the instruction for use as described. The following test procedure is only validated for manual procedure. If performing the test on ELISA automatic systems we recommend increasing the washing steps from three to five and the volume of Washing Buffer from 300 µl to 350 µl to avoid washing effects. Pay attention to chapter 12. Prior to commencing the assay, the distribution and identification plan for all samples and standards/controls (duplicates recommended) should be carefully established on the plate layout supplied in the kit. Select the required number of microtiter strips or wells and insert them into the holder.

Perform all assay steps in the order given and without any delays.

A clean, disposable tip should be used for dispensing each standard/control and sample.

Adjust the incubator to 37 ± 1 °C.

1. Dispense 100 µl standards/controls and diluted samples into their respective wells. Leave well A1 for the Substrate Blank.
2. Cover wells with the foil supplied in the kit.
3. **Incubate for 1 hour ± 5 min at 37 ± 1 °C.**
4. When incubation has been completed, remove the foil, aspirate the content of the wells and wash each well three times with 300 µl of Washing Buffer. Avoid overflows from the reaction wells. The interval between washing and aspiration should be > 5 sec. At the end carefully remove remaining fluid by tapping strips on tissue paper prior to the next step!
Note: Washing is important! Insufficient washing results in poor precision and false results.
5. Dispense 100 µl Conjugate into all wells except for the Substrate Blank well A1.
6. **Incubate for 30 min at room temperature (20...25 °C).** Do not expose to direct sunlight.
7. Repeat step 4.
8. Dispense 100 µl TMB Substrate Solution into all wells.
9. **Incubate for exactly 15 min at room temperature (20...25 °C) in the dark.** A blue colour occurs due to an enzymatic reaction.
10. Dispense 100 µl Stop Solution into all wells in the same order and at the same rate as for the TMB Substrate Solution, thereby a colour change from blue to yellow occurs.
11. Measure the absorbance at 450/620 nm within 30 min after addition of the Stop Solution.

8.2. Measurement

Adjust the ELISA microwell plate reader **to zero** using the **Substrate Blank**.

If - due to technical reasons - the ELISA microwell plate reader cannot be adjusted to zero using the Substrate Blank, subtract its absorbance value from all other absorbance values measured in order to obtain reliable results!

Measure the absorbance of all wells at **450 nm** and record the absorbance values for each standard/control and sample in the plate layout.

Bichromatic measurement using a reference wavelength of 620 nm is recommended.

Where applicable calculate the **mean absorbance values** of all duplicates.

9. RESULTS

9.1. Run Validation Criteria

In order for an assay to be considered valid, the following criteria must be met:

- **Substrate Blank:** Absorbance value **< 0.100**
- **Negative Control:** Absorbance value **< 0.200 and < Cut-off**
- **Cut-off Control:** Absorbance value **0.150 – 1.300**
- **Positive Control:** Absorbance value **> Cut-off**

If these criteria are not met, the test is not valid and must be repeated.

9.2. Calculation of Results

The Cut-off is the mean absorbance value of the Cut-off Control determinations.

Example: Absorbance value Cut-off Control 0.44 + absorbance value Cut-off control 0.42 = 0.86 / 2 = 0.43
 Cut-off = 0.43

9.2.1. Results in Units [NTU]

Sample (mean) absorbance value x 10 = [NovaTec Units = NTU]

Cut-off

Example: $\frac{1.591 \times 10}{0.43} = 37 \text{ NTU (Units)}$

9.3. Interpretation of Results

Cut-off	10 NTU	-
Positive	> 11 NTU	Antibodies against the pathogen are present. There has been a contact with the antigen (pathogen resp. vaccine).
Equivocal	9 – 11 NTU	Antibodies against the pathogen could not be detected clearly. It is recommended to repeat the test with a fresh sample in 2 to 4 weeks. If the result is equivocal again the sample is judged as negative .
Negative	< 9 NTU	The sample contains no antibodies against the pathogen. A previous contact with the antigen (pathogen resp. vaccine) is unlikely.
Diagnosis of an infectious disease should not be established on the basis of a single test result. A precise diagnosis should take into consideration clinical history, symptomatology as well as serological data. In immunocompromised patients and newborns serological data only have restricted value.		

9.3.1. Antibody Isotypes and State of Infection

Serology	Significance
IgM	Characteristic of the primary antibody response High IgM titer with low IgG titer: → suggests a current or very recent infection Rare: → persisting IgM
IgG	Characteristic of the secondary antibody response May persist for several years High IgG titer with low IgM titer: → may indicate a past infection

10. SPECIFIC PERFORMANCE CHARACTERISTICS

The results refer to the groups of samples investigated; these are not guaranteed specifications.

For further information about the specific performance characteristics please contact NovaTec Immundiagnostica GmbH.

10.1. Precision

Intraassay	n	Mean (E)	CV (%)
#1	24	2.183	3.4
#2	24	0.835	11.2

Interassay	n	Mean (NTU)	CV (%)
#1	19	29.8	3.3
#2	24	10.2	4.3

10.2. Diagnostic Specificity

The diagnostic specificity is defined as the probability of the assay of scoring negative in the absence of the specific analyte. It is 96.21% (95% confidence interval: 91.38% - 98.76%).

10.3. Diagnostic Sensitivity

The diagnostic sensitivity is defined as the probability of the assay of scoring positive in the presence of the specific analyte. It is 92.59% (95% confidence interval: 82.11% - 97.94%).

10.4. Interferences

Interferences with hemolytic, lipemic or icteric samples are not observed up to a concentration of 10 mg/ml hemoglobin, 5 mg/ml triglycerides and 0.5 mg/ml bilirubin.

10.5. Cross Reactivity

Cross reaction of the antigens with antibodies against Ascaris, Malaria and, Fasciola cannot be excluded.

11. LIMITATIONS OF THE PROCEDURE

Bacterial contamination or repeated freeze-thaw cycles of the sample may affect the absorbance values.

12. PRECAUTIONS AND WARNINGS

- In compliance with article 1 paragraph 2b European directive 98/79/EC the use of the in vitro diagnostic medical devices is intended by the manufacturer to secure suitability, performances and safety of the product. Therefore the test procedure, the information, the precautions and warnings in the instructions for use have to be strictly followed. The test procedure, the information, the precautions and warnings in the instructions for use have to be strictly followed. The use of the testkits with analyzers and similar equipment has to be validated. Any change in design, composition and test procedure as well as for any use in combination with other products not approved by the manufacturer is not authorized; the user himself is responsible for such changes. The manufacturer is not liable for false results and incidents for these reasons. The manufacturer is not liable for any results by visual analysis of the patient samples.
- Only for in-vitro diagnostic use.
- All materials of human or animal origin should be regarded and handled as potentially infectious.
- All components of human origin used for the production of these reagents have been tested for anti-HIV antibodies, anti-HCV antibodies and HBsAg and have been found to be non-reactive.
- Do not interchange reagents or strips of different production lots.
- No reagents of other manufacturers should be used along with reagents of this test kit.
- Do not use reagents after expiry date stated on the label.
- Use only clean pipette tips, dispensers, and lab ware.
- Do not interchange screw caps of reagent vials to avoid cross-contamination.
- Close reagent vials tightly immediately after use to avoid evaporation and microbial contamination.
- After first opening and subsequent storage check conjugate and standard/control vials for microbial contamination prior to further use.
- To avoid cross-contamination and falsely elevated results pipette patient samples and dispense reagents without splashing accurately into the wells.
- The ELISA is only designed for qualified personnel who are familiar with good laboratory practice.

12.1. Disposal Considerations

Residues of chemicals and preparations are generally considered as hazardous waste. The disposal of this kind of waste is regulated through national and regional laws and regulations. Contact your local authorities or waste management companies which will give advice on how to dispose hazardous waste.

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BIBLIOGRAPHY / LITERATUR / BIBLIOGRAPHIE / BIBLIOGRAFÍA / BIBLIOGRAFIA

- Schistosomiasis (Bilharziose) (2011). In Robert Koch Institut (RKI): Steckbriefe seltener und importierter Infektionskrankheiten. Berlin, pp. 144–146.
- Attallah, Abdelfattah M.; Ismail, Hisham; El Masry, Samir A.; Rizk, Hassan; Handousa, Aia; El Bendary, Mahmoud et al. (1999): Rapid detection of a Schistosoma mansoni circulating antigen excreted in urine of infected individuals by using a monoclonal antibody. In *Journal of Clinical Microbiology* 37 (2), pp. 354–357.
- Cesari, Italo M.; Ballen, Diana E.; Mendoza, Leydi; Matos, Cesar (2005): Detection of Schistosoma mansoni membrane antigens by immunoblot analysis of sera of patients from low-transmission areas. In *Clinical and diagnostic laboratory immunology* 12 (2), pp. 280–286. DOI: 10.1128/CDLI.12.2.280-286.2005.
- Jonge, Niels de; Fillié, Yvonne E.; Deelder, André M. (1987): A simple and rapid treatment (trichloroacetic acid precipitation) of serum samples to prevent non-specific reactions in the immunoassay of a proteoglycan. In *Journal of immunological methods* 99 (2), pp. 195–197.
- Jonge, Niels de; Fillié, Yvonne E.; Hilberath, Gerard W.; Krijger, Frederik W.; Lengeler, Christian; Savigny, Donald H. de et al. (1989): Presence of the schistosome circulating anodic antigen (CAA) in urine of patients with Schistosoma mansoni or S. haematobium infections. In *The American journal of tropical medicine and hygiene* 41 (5), pp. 563–569.
- King, Charles H. (2006): Schistosomiasis. In Richard L. Guerrant, David H. Walker, Peter F. Weller (Eds.): Tropical infectious diseases. Principles, pathogens & practice. 2nd ed. Philadelphia: Churchill Livingstone, pp. 1341–1348.
- Robert Koch-Institut (RKI) (2006): Reiseassoziierte Infektionskrankheiten im Jahr 2005. In *Epidemiologisches Bulletin* 37.
- van Dam, Govert J.; Dood, Claudia J. de; Lewis, Melanie; Deelder, Andre M.; van Lieshout, Lisette; Tanke, Hans J. et al. (2013): A robust dry reagent lateral flow assay for diagnosis of active schistosomiasis by detection of Schistosoma circulating anodic antigen. In *Experimental parasitology* 135 (2), pp. 274–282. DOI: 10.1016/j.exppara.2013.06.017.
- van Dam, Govert J.; Qian, Z. L.; Fillie, Y. E.; Rotmans, J. P.; Deelder, A. M. (1993): Detection of IgM antibodies directed against the gut-associated circulating cathodic antigen in sera from Schistosoma mansoni infected patients. Development and comparison of three enzyme-linked immunoassays. In *Tropical and geographical medicine* 45 (2), pp. 59–65.

ABBREVIATIONS / ABKÜRZUNGEN / ABRÉVIATIONS / ABBREVIAZIONI / ABREVIACIONES / ABREVIATURAS

NMP	N-Methyl-2-pyrrolidone
-----	------------------------

SYMBOLS KEY / SYMBOLSCHLÜSSEL / EXPLICATION DES SYMBOLES / LEGENDA / SÍMBOLOS / TABELA DE SÍMBOLOS

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Schistosoma mansoni IgM ELISA

Test Preparation

Prepare reagents and samples as described.

Establish the distribution and identification plan for all samples and standards/controls on the plate layout supplied in the kit.

Select the required number of microtiter strips or wells and insert them into the holder.

Assay Procedure

	Substrate Blank (A1)	Negative Control	Cut-off Control	Positive Control	Sample (diluted 1+100)
Negative Control	-	100 µl	-	-	-
Cut-off Control	-	-	100 µl	-	-
Positive Control	-	-	-	100 µl	-
Sample (diluted 1+100)	-	-	-	-	100 µl
Cover wells with foil supplied in the kit					
Incubate for 1 h at 37 °C					
Wash each well three times with 300 µl of Washing Buffer					
Conjugate	-	100 µl	100 µl	100 µl	100 µl
Incubate for 30 min at room temperature (20...25 °C)					
Do not expose to direct sunlight					
Wash each well three times with 300 µl of Washing Buffer					
TMB Substrate solution	100 µl	100 µl	100 µl	100 µl	100 µl
Incubate for exactly 15 min at room temperature (20...25 °C) in the dark					
Stop Solution	100 µl	100 µl	100 µl	100 µl	100 µl
Photometric measurement at 450 nm (reference wavelength: 620 nm)					

NovaTec Immundiagnostica GmbH

Waldstraße 23 A6
63128 Dietzenbach, Germany

Tel.: +49 (0) 6074-48760 Fax: +49 (0) 6074-487629
Email: info@NovaTec-ID.com
Internet: www.NovaTec-ID.com

SCHM0410engl,dt,es,port-05072017-Ka-gr.Lot004-1