



SZABO SCANDIC

Part of Europa Biosite

Produktinformation



Forschungsprodukte & Biochemikalien



Zellkultur & Verbrauchsmaterial



Diagnostik & molekulare Diagnostik



Laborgeräte & Service

Weitere Information auf den folgenden Seiten!
See the following pages for more information!



Lieferung & Zahlungsart

siehe unsere [Liefer- und Versandbedingungen](#)

Zuschläge

- Mindermengenzuschlag
- Trockeneiszuschlag
- Gefahrgutzuschlag
- Expressversand

SZABO-SCANDIC HandelsgmbH

Quellenstraße 110, A-1100 Wien

T. +43(0)1 489 3961-0

F. +43(0)1 489 3961-7

mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

Eosin Y Solution (Alcoholic)

Description: Eosin Y Solution is intended for use in the histological demonstration of cytoplasm and is commonly used as a counterstain for Hematoxylin. When used correctly, various shades of pink can be obtained to aid in visualization of tissue components. Erythrocytes, collagen, and the cytoplasm of muscle or epithelial cells will stain with different shades of pink.


Collagen, Muscle, Erythrocytes: Various shades of Pink
Nuclei after Eosin: Blue to Violet

Uses/Limitations: Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use if reagent become cloudy.
Do not use past expiration date.
Use caution when handling reagent.
Non-Sterile.

Control Tissue: Any well-fixed tissue.

Availability/Contents:

<u>Item #</u>	<u>Volume</u>	<u>Storage</u>
EYA500	500 ml	18-25°C
EYA999	1000 ml	18-25°C
EYA3800	1 Gallon	18-25°C

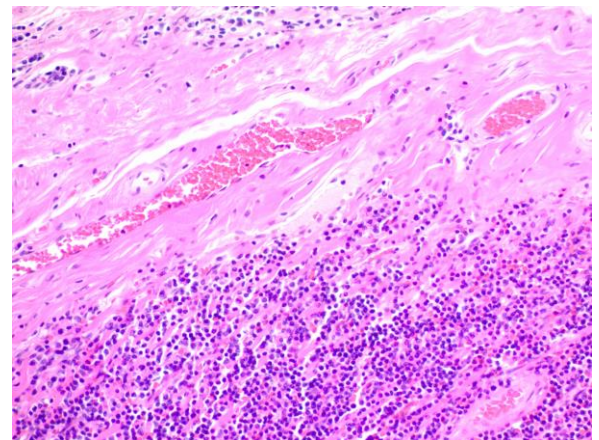



Also available in bulk. Please contact for pricing and availability.

Precautions: Avoid contact with skin and eyes.
Solution is alcoholic, avoid sparks and flame and tighten lid securely.
Follow all Federal, State, and local regulations regarding disposal.

Procedure:

1. Stain for 1-5 minutes in preferred Hematoxylin **Note: Usually longer incubation times provide a darker stain.** (ScyTek Cat# HMM)
2. Rinse slide in tap or deionized water.
3. If desired, blue in bluing reagent for 15-30 seconds (ScyTek Cat# BRT).

Storage: 2° C  8° C

 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

CE IVD

EC REP
Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands

4. Rinse slide in tap or deionized water.
5. Dip or rinse slide quickly several times in 90-100% ethanol to remove water and prep tissue for Eosin Y (Alcoholic).
6. Continue with incubation in Eosin Y Solution (EYA) for 1-5 minutes based on desired intensity. **Note:** *If pipetting small amounts of Eosin onto a horizontal slide, expect substantial evaporation over time which can concentrate the solution and lead to darker staining. Allowing complete drying of reagent on slide is not suggested.*
7. Rinse quickly in 90-100% alcohol followed by further dehydration in absolute alcohol.
8. Clear, and mount in synthetic resin.

References:

1. Niu, Shuiqin, Ling-Kun Zhang, Li Zhang, Siyi Zhuang, Xiuyu Zhan, Wu-Ya Chen, Shiwei Du, et al. "Inhibition by Multifunctional Magnetic Nanoparticles Loaded with Alpha-Synuclein RNAi Plasmid in a Parkinson's Disease Model." *Theranostics* 7, no. 2 (January 1, 2017): 344–56. <https://doi.org/10.7150/thno.16562>.

Storage: 2° C



8° C

ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands