SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

Hematoxylin, for Automation


Description: Hematoxylin, for Automation is a diluted format of Mayer's hematoxylin that has been specifically formulated for Autostainers that may require a longer incubation time and a slightly lighter blue nucleus.

Uses/Limitations: Not to be taken internally.
 For In-Vitro Diagnostic use only.
 Histological applications.
 Do not use if reagent becomes cloudy.
 Do not use past expiration date.
 Use caution when handling reagent.
 Non-Sterile.

Control Tissue: Any well-fixed tissue.

Availability:

<u>Item #</u>	<u>Volume</u>
HAQ500	500 ml
HAQ999	1000 ml
HAQ010	10 Liters

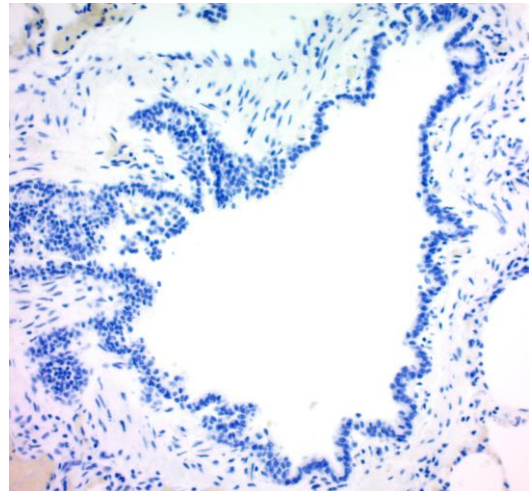Storage: Room Temperature.
 Keep container tightly closed.

Precautions: Avoid contact with skin and eyes.
 Wash after use.
 Observe all federal, state and local environmental regulations regarding disposal

Manual Procedure:

1. Re-hydrate tissue as usual.
2. Apply enough Hematoxylin to completely cover specimen and incubate for 2-10 minutes with hematoxylin according to desired intensity.
3. Apply Bluing Reagent (ScyTek catalog# BRT500) for 5-30 seconds.
4. Rinse in DI water.
5. Coverslip using media of choice.

Storage: 2° C  8° C

 ScyTek Laboratories, Inc.
 205 South 600 West
 Logan, UT 84321
 435-755-9848
 U.S.A.

CE IVD

EC REP
 Emergo Europe
 Prinsessegracht 20
 2514 AP The Hague, The Netherlands