



SZABO SCANDIC

Part of Europa Biosite

Produktinformation



Forschungsprodukte & Biochemikalien



Zellkultur & Verbrauchsmaterial



Diagnostik & molekulare Diagnostik



Laborgeräte & Service

Weitere Information auf den folgenden Seiten!
See the following pages for more information!



Lieferung & Zahlungsart

siehe unsere [Liefer- und Versandbedingungen](#)

Zuschläge

- Mindermengenzuschlag
- Trockeneiszuschlag
- Gefahrgutzuschlag
- Expressversand

SZABO-SCANDIC HandelsgmbH

Quellenstraße 110, A-1100 Wien

T. +43(0)1 489 3961-0

F. +43(0)1 489 3961-7

mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

Oil Red O Solution (For Fat)

Description: Oil Red O Solution is a component of the Oil Red O Stain Kit (For Fat) (Catalog# ORK-1) intended for use in the histological visualization of fat cells and neutral fat. This kit may be used **ONLY** on frozen tissue sections, fresh smears, or touch preps.

Fat Cells:	Red
Neutral Fat:	Red
Nuclei:	Blue

Uses/Limitations: For In-Vitro Diagnostic use only. Histological applications. Do not use past expiration date.

Control Tissue: Any frozen section containing fat.

Availability/Contents:

<u>Item #</u>	<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
ORG125	Oil Red O Solution	125 ml	Room Temperature
ORG500	Oil Red O Solution	500 ml	Room Temperature
ORG999	Oil Red O Solution	1000 ml	Room Temperature

Required But Not Included:


<u>Item #</u>	<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
PRG500	Propylene Glycol	500 ml	Room Temperature
HMM125	Hematoxylin, Mayer's (Lillie's Mod.)	125 ml	Room Temperature

Precautions: Keep away from open flame. Avoid contact with skin and eyes. Harmful if swallowed. Follow all Federal, State, and local regulations regarding disposal. Use in chemical fume hood whenever possible.

Procedure (Standard):

1. Prepare fresh or frozen tissue section as usual.
 2. Place slide in Propylene Glycol for 2 minutes.
 3. Incubate slide in Oil Red O Solution for 6 minutes.
- Note: Prepare mixture of 85% Propylene Glycol in distilled water.
4. Differentiate tissue section in 85% Propylene Glycol for 1 minute.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.

CE IVD
EC REP (Regulatory affairs only)
 EmergoEurope (31)(0) 70 345-8570
 Molsnstraat 15
 2513 BH Hague, The Netherlands

Instructions For Use
ORG-IFU

Rev. Date: July 14, 2009

Revision: 1


Page 2 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

5. Rinse slide in 2 changes of distilled water.
6. Incubate slide in Hematoxylin, Mayer's (Lillie's Modification) for 1-2 minutes.
7. Rinse slide thoroughly in tap water.
8. Rinse slide in 2 changes of distilled water.
9. Coverslip using an aqueous mounting medium (cat# AML060).

References:

1. Sheehan, DC., Hrapchak, BB. Theory and Practice of Histotechnology; 1980, page 225.

Storage: 18° C  25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.

CE



IVD

(Regulatory affairs only)
EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands