# SZABO SCANDIC

Part of Europa Biosite

## Produktinformation

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[mail@szabo-scandic.com](mailto:mail@szabo-scandic.com)

[www.szabo-scandic.com](http://www.szabo-scandic.com)

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

## PT Mount - Mounting Medium


**Description:** An organic solvent (Toluene) based mount for permanent mounting of tissue specimens. This mount is compatible with all biological dyes and immunohistochemical stains requiring a solvent based mount. PT mount designed to be optically clear with a refractive index close to that of fixed protein. Slides will not fade over time when stored in the dark.

<b>Availability/Contents:</b>	<u><b>Item #</b></u>	<u><b>Volume</b></u>
	PTM100	100 ml
	PTM500	500 ml

**Uses/Limitations:**

- Not to be taken internally.
- For In-Vitro Diagnostic use.
- Histological applications.
- Do not use if reagents become cloudy.
- Do not use past expiration date.
- Use caution when handling reagents.
- Non-Sterile.

**Ordering Information and Current Pricing at [www.scytek.com](http://www.scytek.com)**

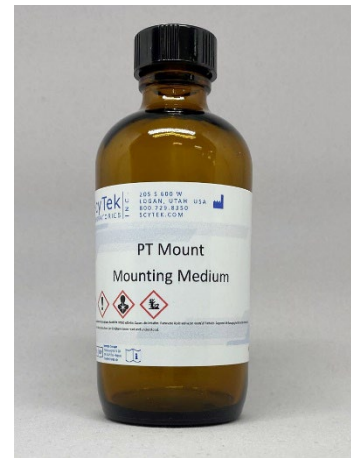**Precautions:**

- Flammable liquid or vapor.
- Avoid contact with skin and eyes.
- Harmful or fatal if swallowed or inhaled.
- Follow all Federal, State, and local regulations regarding disposal.

**Procedure:**

1. Dehydrate tissue specimen through graded alcohols or by air drying.
2. Clear slide in xylene or xylene substitute.
3. Carefully apply a small amount of PT Mount on specimen and mount with coverslip and allow to dry.
4. Store slides in the dark to prevent fading over time.

**Note:** Keep in the original or other tightly sealed container. Add more xylene or toluene if solution becomes too viscous from drying.

**Removal:** This reagent can be removed by soaking slides in xylene or substitute.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.  
205 South 600 West  
Logan, UT 84321  
U.S.A.

Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague, The Netherlands