SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com


www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

An Innovative Rapid Test Designed by WANTAI

Rapid Test for IgM Antibody to Hepatitis E Virus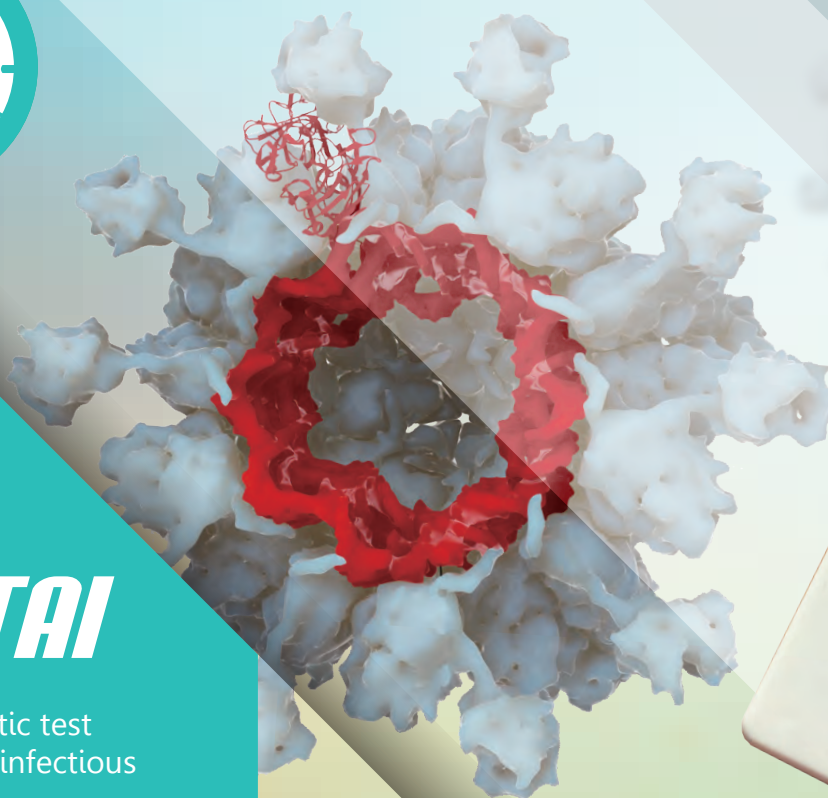

WANTAI

Leading diagnostic test
manufacturer of infectious
diseases

Your Global Partner in IVD

*Developing Scientifically
Focusing on the Health*

Innovative

High Performance

Easy-to-use


Rapid

Test Process


1. Prepare Test

Allow the test cassette to reach room temperature, use it within 20 minutes after opening the pouch.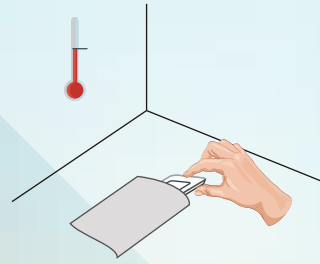


2. Add Specimen

1) Pipette 15µl of serum or plasma into the Sample Dilution Tube and mix sufficiently.

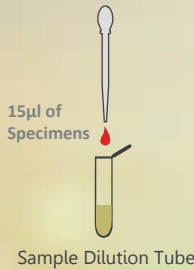
2) Pipette 80µl of diluted specimen into the specimen well on the cassette.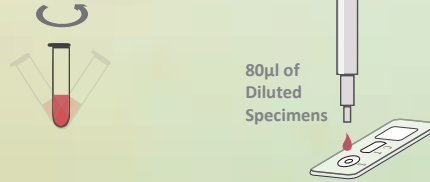

3. Read Result


Read the results in 10 minutes.

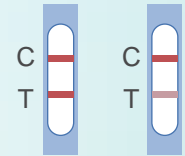Positive

Negative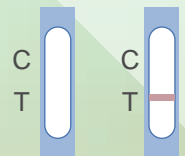

Invalid

* Please refer to product insert for detailed instruction.

Advanced Diagnostic Technology

- 01 Lateral Flow Technology:** makes it possible to perform rapid on-site testing
- 02 Superior Performance:** 97.7% sensitivity and 99.28% specificity reported by European Centre for Environment & Human Health, UK
- 03 Easy-to-use:** No special equipment. Works with serum and plasma samples
- 04 Fast:** clear answer within 10 minutes only
- 05 All in one:** all accessories in the kit allows to do point-of-care testing
- 06 Room Temperature Storage:** 18 months shelf-life

Order Information

Cat.	Product	Detection	Specimen	Pack size
WJ-1510	HEV-IgM Antibody rapid test	IgM Antibody	Serum/Plasma	10T/kit