SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

Instructions For Use

INTRODUCTION AND PURPOSE OF USE

The quany WNV system is a quantitative test that allows the RNA amplification, by means of Real Time PCR, of 3'UTR region of West Nile Virus (WNV), discriminating it from Usutu Virus, also belonging to the Flavivirus family and closely related to WNV. The Procedure allows the detection and quantification of the RNA target by means a One step retro-amplification reaction. The analysis of the results is made using a Real Time PCR analyzer (thermal cycler integrated with a system for fluorescence detection and a dedicated software).

CONTENT

The kit contains reagents enough to perform 48 amplification tests:

	Quantity	Description
R1	3 x 10 µl	Enzyme mMix Reverse Transcriptase e Taq Polymerase enzymes (Pink Cap)
R2	3 x 270 µl	WNV probe Mix Upstream primer, downstream primer, Target probe WNV (FAM), Target probe Usutu Virus (VIC), Internal Control (IC) probe (CY5), Nuclease-free water, ROX dNTPs, Tris-HCl, KCl, MgCl ₂ (White Cap)
R3	3 x 35 µl	synthetic RNA corresponding to 3'UTR region of WNV 100.000 cps/µl (Blu Cap)
R4	3 x 35 µl	synthetic RNA corresponding to 3'UTR region of WNV 10.000 cps/µl (Green Cap)
R5	3 x 35 µl	synthetic RNA corresponding to 3'UTR region of WNV 1.000 cps/µl (Yellow Cap)
R6	3 x 35 µl	synthetic RNA corresponding to 3'UTR region of WNV 100 cps/µl (White Cap)
R7	3 x 160 µl	Internal control (phage MS2) (Red Cap)
R8	1 x 50 µl	Negative Control
R9	3 x 35 µl	Usutu Virus Positive Control

MATERIALS AND STRUMENTATION REQUIRED BUT NOT SUPPLIED

Disposable latex powder-free gloves or similar material;
Bench microcentrifuge (12,000 - 14,000 rpm);
Micropipettes and Sterile tips with aerosol filter;
Vortex;
Plastic materials (microplate and optical adhesive cover);
Heat block (only for extraction)
EZ1 Advanced XL DSP Virus Card. - Ref. 9018703 - QIAGEN.
ATL Buffer - Ref. 939016 - QIAGEN.
2 ml Microtube PP Skirted - Ref. 72.694 - QIAGEN
QIASymphony plastic materials (Filter Tips, sample prep Cartridges, 8 Rod covers, etc...)

Reagents

The **quany WNV** kit was developed and validated to be used with the following extraction method:

Manual Extraction

Ref. 52906. *QIAmp Viral RNA Mini Kit*

The kit allows the manual RNA extraction from tested samples. The kit contains reagents for 250 samples. (QIAGEN)

Automatic extraction

Ref. 62724. *EZ1 XL DSP Virus Kit*.

The kit allows the automatic viral RNA from Human samples.

The kit contains reagents for 48 samples. (QIAGEN)

Ref.937055. *QIASymphony® DSP Virus/Pathogen midi kit*.

The kit allows the automatic viral RNA from Human samples. The kit contains reagents for 96 samples. (QIAGEN)

Instruments

The **quany WNV** kit was evaluated to be used with the following instruments:

Extraction System

Ref. 9001492. *EZ1 Advanced XL*.

Robotic Workstation for the automatic purification of the nucleic acids until 14 samples simultaneously (QIAGEN)

Ref. 9001297 *QIASymphony SP*

Robotic Workstation for the automatic purification of the nucleic acids (QIAGEN).

Real Time PCR

The **quanty WNV** kit was evaluated to be used with the following real time PCR instruments:

- *7500 Fast* from Lifetechnologies
- *Rotor-Gene Q MDx* from QIAGEN
- *Light Cycler 480* from Roche
- *CFX96 Real Time PCR System* from Bio-Rad

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturer's instruction and recommendations.

SAMPLES

The **quanty WNV** kit must be used with extracted RNA from biological samples: Serum and Urine. Collected samples must be shipped and stored at +2 - +8°C and used within 3 days from the collected data. Store the sample at -20°C if it is used after 3 days.

PRECAUTIONS FOR USE

- This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.
- Carefully read this Instruction For Use before using the kit.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: RNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- Periodically wipe the working area with 0,5% hypochlorite.
- Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.
- Use powder-free gloves. Do not leave fingerprints on optical caps. Do not write on caps as this may cause an interference with fluorescent detection.
- Avoid any contact between hands and eyes or nose during specimen collection and testing.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood. Waste should be discarded according to local law.
- Never pipette solutions by mouth.
- Use calibrated and regularly checked pipettes and instrumentation only.
- Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.
- Wash hands carefully after handling samples and reagents.
- Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
- Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

LIMITS OF THE METHOD

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and/or controls. Therefore, you should:

- physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products
- use tips with filters to prevent cross-contamination between samples
- use disposable gloves and change them frequently
- carefully open test tubes to prevent aerosol formation
- close every test tube before opening another one

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

Patient Drug treatment may interfere with the final result of the molecular biology analysis.

The proper functioning of the amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

WARNINGS

- Read carefully the instructions for use before using this test.
 - Use only extracted RNA from Serum and/or Urine
 - Do not mix reagents from different lots.
 - Thaw and carefully mix the reagents of the kit before use.
 - The PCR mix has to be freshly prepared every time.
- After reconstitution, the amplification master mix must be used in one time (16 reactions). Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.
- Do not use beyond the expiration date which appears on the package label.
 - Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
 - In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
 - In case of damaged package, contact the technical support before using the kit.
 - In case of any serious incident that has occurred in relation to the device, a notice shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

STORAGE AND STABILITY

Store the product **quanty WNV** at -20°C.

The **quanty WNV** kit is shipped on dry ice. The kit components should be frozen.

If one or more components are not frozen upon receipt or if the tubes have been compromised during transport, contact Clonit srl for assistance.

An intact and well stored product has a stability of 12 months from the date of production. Do not use beyond the expiration date which appears on the package label.

Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.

ANALYTICAL PROCEDURE

Manual Extraction

Ref. 52906. QIAmp Viral RNA Mini Kit

Procedure to Serum and Urine

Follow the instructions inside the kit QIAmp Viral RNA Mini Kit.

After the incubation period for 10 min at room temperature, add 5 µl of *Internal Control*

Follow the instructions inside the kit

Elute the sample in 50 µl of buffer AVE.

Samples are now ready for amplification or storage at -80°C

Automatic extraction

Ref. 62724. EZ1 XL DSP Virus Kit

Procedure to Serum

Follow the instructions inside the kit EZ1 XL DSP Virus Kit.

Volume of sample to be used:

Serum [µl]
200

Preparation of the Carrier and the Internal Control

Solve completely the lyophilize RNA carrier in elution buffer (AVE), from 310 µl, split in aliquots and store to -20 ± 5°C. Not thawing and freezing the aliquots more than 2 times.

For each analyzed sample, dilute 3,6 µl of a original solution include the RNA Carrier and 10 µl of *Internal Control* in total volume of 60 µl using elution buffer (AVE)

Select the protocol starting from 200 µl of samples with the elution of 60 µl.

Follow the instructions inside the kit EZ1 XL DSP Virus Kit.

Samples are now ready for amplification or storage at -80°C

Procedure to Urine

Follow the instructions inside the kit EZ1 XL DSP Virus Kit.

Volume of sample to be used:

Urine [µl]	ATL [µl]	Final volume samples [µl]
300	100	400

Preparation of the Carrier and the Internal Control

Solve completely the lyophilize RNA carrier in elution buffer (AVE), from 310 µl, split in aliquots and store to -20 ± 5°C. Not thawing and freezing the aliquots more than 2 times.

For each analyzed sample, dilute 3,6 µl of a original solution include the RNA Carrier and 10 µl of *Internal Control* in total volume of 60 µl using elution buffer (AVE)

Select the protocol starting from 400 µl of samples with the elution of 60 µl.

Follow the instructions inside the kit EZ1 XL DSP Virus Kit.

Samples are now ready for amplification or storage at -80°C

Automatic Extraction

Ref.937055. QIASymphony® DSP Virus/Pathogen midi kit.

Procedure to Serum and Urine

Follow the instructions inside the kit QIASymphony DSP Virus/Pathogen Midi. Use this volume of sample with 2 ml Microtube PP Skirted:

Serum [µl] (including excess volume)
1200

Select the protocol **Cellfree1000_V7_DSP_default_IC**.

Urine [µl] (including excess volume)
950

Select the protocol **Complex800_V6_DSP_default_IC**.

Preselected elution volume

Preselected elution volume (µl)	Initial elution volume (µl)
60	90

Preparation of the Carrier

To prepare a carrier RNA (CARRIER) stock solution, add 1350 µl Buffer AVE (AVE) (provided in 2µl vials) to the tube containing 1350 µg lyophilized carrier RNA (CARRIER) to obtain a solution of 1 µg/µl. Dissolve the carrier RNA (CARRIER) divide it into conveniently sized aliquots, and store at 2-8°C for up to 2 weeks.

Addition of the internal control to the samples

For each Serum samples, 10µl internal controls must be added with carrier RNA (CARRIER)-Buffer AVE (AVE) mixture, and the total volume of the internal control-carrier RNA (CARRIER)-Buffer AVE (AVE) mixture remains 120 µl.

elution volume (µl)	RNA CARRIER (µl)	internal control (µl)	Buffer AVE (µl)	Final volume per sample (µl)
60	4.5	10	105,5	120

For each Urine samples, 10µl internal controls must be added with carrier RNA (CARRIER)–Buffer AVE (AVE) mixture, and the total volume of the internal control–carrier RNA (CARRIER)–Buffer AVE (AVE) mixture remains 120 µl.

elution volume (µl)	RNA CARRIER (µl)	internal control (µl)	Buffer AVE (µl)	Final volume per sample (µl)
60	2.5	10	107,5	120

The internal control will be added to the sample automatically by the instrument.

SOFTWARE SETTING

Lifetechnologies 7500 fast

Turn the instrument and the computer on and open the control software. Click on "**Advance Setup**": by default the software will show the page "**experiment properties**". Write in the "**experiment name**" the file name, choose the type of instrument (**7500 o 7500fast**), the type of reaction (**quantitation standard curve**), the type of used reagent (**Taqman®Reagents**) and the reaction time of analysis (**Standard ≈ 2 hours to complete a run**).

Open the page named "**page setup**" (sheet **Define Target and Samples**).

In the window **Define Targets** set:

Target	Reporter	Quencher
WNV probe:	FAM	MGBNFQ
Usutu probe:	VIC	MGBNFQ
Internal Control (IC) probe:	CY5	NONE

Set the samples' name in the window "**Define Samples**".

In the same page "**plate setup**" select the sheet "**Assign Target and Samples**". On the screen you will see the microplate draft.

Select an area of the plate where the controls will be placed: select wells of the plate and set targets (WNV, Usutu and IC):

- Select "**Assign target to selected wells**" in the blank, the "**task Standard (S)**" for WNV target and set the controls' concentration.
- Choose an area in the plate where Usutu Positive control will be placed: select "**Assign target to selected wells**" in the blank, the "**task Standard (S)**" for the targets.
- Choose an area in the plate where negative control will be placed: select "**Assign target to selected wells**" in the blank, the "**task Negative (N)**" for the targets.

Select an area of the plate where samples will be placed: select the wells and set targets (WNV, Usutu and IC). Link every well to a sample, through the window "**Assign samples to selected wells**".

For each sample, select in the blank "**Assign targets to selected wells**" the "**task UnKnown (U)**" for the targets.

Set ROX as passive reference, using it as normalizer of detecting fluorescence.

Open "**Run Method**" (sheet **Graphic View**) and set the right thermal cycling with the "collect data" in annealing phase.

cycles	denaturation	annealing	extension
1	50° C 30 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

In the window "**Reaction volume plate per well**" set a volume of 30 µl.

After preparing the plate, and correctly inserting it in the instrument, press the button "**Start Run**".

Rotor Gene Q MDx

The experiments can be **set using the Quick Start Wizard or the Advanced Wizard**, which appears when the software is started.

Select the wizard "**Advanced**". As a first step, select the model "**Two Step Reaction**" with a double click in the "**New Run**".

In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "**Locking Ring Attached**", check the checkbox and then click "**Next**".

Enter the name of the operator and the reaction volume of 30 µl, and then click "**Next**".

In the next window click on "**edit profile**". Set the following thermal cycle:

cycles	denaturation	annealing	extension
1	50° C 30 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

Select the annealing from the thermal profile and click on "**Acquiring A to cycling**".

In the next window, select yellow and red from the available channels and add it to acquiring channel along with the green channel and click "**OK**". In the next window click on "**OK**" and then click "**Next**".

Click on "Edit Gain" button and set the following values for each channel:

Reporter	Gain
Green	5
Yellow	5
Red	8

To begin the course, click on the button "**Start Run**". You can save the model before you begin your run by clicking on "**Save Template**".

After clicking on the button "**Start Run**" window appears "**Save As**". The stroke can be saved in the desired position by the user.

Once the run started, the window "**Edit Samples**" allows you to set the name of samples and controls in the positions in which they were loaded on the instrument.

Select the locations where they were positioned the controls of known concentration and designate them as WNV Standard. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Standards**". Enter the concentrations of the controls.

Select the location where you placed the **Usutu Positive Control** and name it as Positive Control. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Positive Controls" Select the location where you placed the Negative Control and name it as Negative Control. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Negative Controls" Select the location of each sample and enter the name or code of the patient. Clicking on the box next to "Type" correspondent, in the dropdown menu "Samples" you can select the type of sample being analyzed. Select "Unknown" At the end of the operation click "OK" in the "edit samples" and wait until the end of the race for the analysis (see "Interpretation of Results").

LightCycler 480

Turn the instrument and the computer on and start the control software. In the principal screen, on "Experiment Creation" select the "Plate type" and push the "New experiment" button. The window "experiment" appears. On the "Run protocol" sheet set: Thermal protocol, Reaction volume (30µl) and Detection format (**3 colour hydrolysis probe**).

Customize the Detection format in Dynamic mode, with the following reading channels:

Filter Combination	
FAM (465-510)	
VIC / HEX / Yellow555 (533-580)	
Cy 5 /Cy 5.5 (618-660)	

Set the following thermal profile, with signal acquisition in the annealing phase

cycles	denaturation	annealing	extension
1	50° C 30 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

Push the "Subset editor" button and in this window, select an area of the plate where controls and samples will be placed. Push the "Sample editor" button. Select the correct workflow (Step1: **Abs quantification**), choose the samples Subset created in the step before and insert the name for each well. Choose the correct type for each well: Positive CTR, Negative CTR or Unknown sample. Push again the "experiment button", insert the plate in the instrument and push "start run".

CFX96 Real Time PCR

Turn the instrument and the computer on and start the control software. In the principal screen will appear the window "Startup wizard": select "CFX96" and press "ok". In the next window push "create new" and set the thermal protocol, with the "collect data" in annealing phase, and set the reaction volume (30µl):

cycles	denaturation	annealing	extension
1	50° C 30 min		
1	95°C 2 min		
45	95° C 15 sec	58° C 45 sec	72° C 15 sec

Save the protocol and click the next button. The software will open by default the sheet "plate". Click "create new" and select the "Fluorophores" button to choose fluorophores (FAM, VIC and Cy5). Select the locations where they were positioned the controls of known concentration and choose the "Sample Type" **Standards**. Click "Load" check boxes to load fluorophores and Type or select Target Name. In the box "load concentration" set the concentrations of the 4 calibrators, following the instructions indicated in the paragraph Analysis of the results. Select the location where you placed the Negative Control. Choose the "Sample Type" **NTC**. Click "Load" check boxes to load fluorophores and Type or select Target Name Select the location of each sample and enter the name or code of the patient. Choose the "Sample Type" **Unknown**. Click "Load" check boxes to load fluorophores and Type or select Target Name. Save the plate clicking the next button and start the experiment.

PREPARATION OF THE REACTIONS:

Thaw a tube of **Enzyme mMix**;

Thaw a tube of **WNV probes Mix**;

For a total number of tests other than 16, prepare a mixture for n + 1 tests, following the volumes shown in the table:

Reagent	Description	Volume for single test (µl)
R1 -	Enzyme mMix	0,5
R2	WNV probes Mix	14,5

Mix carefully by vortex **9µl of Enzyme mMix** and **260µl of WNV probes Mix** (the mix as produced is enough to prepare **16 reactions** of amplification: **5 positive controls, 1 negative control and 10 samples**).

Distribute, in the amplification plate, **15µl of just reconstituted mix** in chosen positions as already set on the instrument software.

Distribute, in the negative control position, **15 µl** of solution taken by the negative control vial.

Distribute, in chosen position for each sample, **15 µl** of corresponding sample.

Distribute, in chosen positions for the positive controls, **15µl of 10² copies/µl, 10³ copies/µl, 10⁴ copies/µl and 10⁵ copies/µl** and the Usutu positive control.

Seal up accurately the plate using an optical adhesive film and verify that there aren't air bubbles in the mix, to avoid the amplification interferences.

For the Rotor-Gene Q MDx, seal each tube with the appropriate cap. The air bubbles presence is not influent; the rotor centrifugal force will allow automatic deletion.

Transfer the plate in the instrument and push the button "Start Run".

QUANTITATIVE ANALYSIS

Lifetechnologies 7500 Fast

At the end of the PCR run, the software automatically opens the "Analysis" window in the "Amplification plot" sheet on the menu on the left.

Select the wells corresponding to the positive control, negative control and samples for analysis. Select in the "Option" window inside the "Target" pop-up menu the **WNV target**. Check the correct setting of the threshold. Select in the "Option" window inside the "Target" pop-up menu the **Usutu target**. Check the correct setting of the threshold. Select in the "Option" window inside the "Target" pop-up menu the **IC Control target**. Check the correct setting of the threshold.

The analysis of the results is made selecting from the menu in the left the page "Analysis". From the page "Standard Curve", maintaining open the sheet "view well plate" in the right side of the software select the wells containing the points of the curve and verify the parameters described in the paragraph "Interpretation of the Results" (coefficient of correlation, slope ecc...). From the page "Amplification Plot" verify the amplification plot for every single sample. Opening the sheet "view well table" in the right side of the software it is possible to verify the data obtained from experiments: Threshold Cycles, emitted fluorescence and the target quantification expressed in copies/reaction or copies/ml, depending on the settings of the calibration curve. Clicking from the menu file and selecting the box export, the window "export properties" will open. Indicate the file name, select the position to save it (Browse) and click on button "Start export". In this way the software will permit to save a excel file with all the data corresponding to selected experiment.

Rotor Gene-Q MDx

At the end of the PCR run open the "Analysis" window. Select the "Quantification" sheet and click on "cycling A (green)". Eliminate the first 10 amplification cycles from the analysis by selecting the "Ignore First" button and setting the value 10. Check the correct setting of the threshold in the space provided "CT calculation – Threshold" and delete the first 10 cycles from the interpolation of the Threshold by entering the value 10 in the "Eliminate Cycles Before" box. Select from the menu "Dynamic Tube" and subsequently "Slope correct". If necessary, reduce the background noise of non-specific signals by selecting the "Outlier Removal" option and entering a value between 5% and 30%. Repeat the steps above for each of the fluorescence channels "cycling A (yellow)" and "cycling A (red)". You can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification cycling A (green), then the file cycling A (yellow) and then the file cycling A (red).

LightCycler 480

When the run is completed select analysis and choose the correct kind of analysis you want: "Abs Quant/Fit Points". Choose the samples subset you want to analyze. Select the "NoiseBand" sheet, under the plot you can choose "NoiseBand (Fluoresc.)"; and move the line of the NoiseBand on the plot with the mouse of your PC. Repeat this action for each fluorophore using the "Filter comb" button.

Clicking the sheet "Analysis" you can set the threshold choosing the option "Threshold(manual)".

After setting parameters push the "Calculate" button. Repeat this action for each fluorophore.

CFX96 Real Time PCR System

At the end of the PCR, select the "quantitation" sheet. On the top of the screen, select "settings" from the menu and choose "Baseline Threshold...".

You can export the report pushing the paper block figure on the top of the screen.

INTERPRETATION OF RESULTS

Through the Real Time PCR reaction, it is possible to obtain the RNA quantification of WNV RNA, by setting the values of the positive controls of the calibration curve. To calculate these values, all the dilutions steps that the sample has undergone during the extraction and amplification stages must be considered.

The system is able to detect from 1,500,000 to about 15 RNA copies per reaction.

The Ct values obtained from the amplification of 4 controls of known titre are used by the software for the calculation of the calibration curve from which the unknown samples are interpolated.

A proper functioning of the amplification mix can be verified analyzing these parameters:

Parameters	Reference
RTS conc. 10 ⁵ copie/μl (FAM)	Ct ≤ 25
Correlation Coefficients	0.990 ≤ r ² ≤ 1
Slope	-3,6 ≤ Slope ≤ -3,1
PCR Efficiency	90 ≤ Efficiency ≤ 110

If the RTS amplification reaction at a concentration of 10⁵ copies produces a Ct > 25 or undetermined the session can't be considered valid and must be repeated.

Verify that the correlation coefficient value (r²), the slope or the reaction efficiency fit to the limited indicated in the above table or do not deviate much from them, which represent the ideal range for a proper PCR reaction.

By correctly setting the standards concentration as a function of the extraction system you can get the quantization of the sample directly in copies / ml:

	Manual Extraction Ref. 52906. QIAmp Viral RNA Mini Kit	Automatic Extraction Ref. 6272 EZ1 XL DSP Virus Kit		Automatic Extraction Ref. 937055 QIAasymphony® DSP Virus/Pathogen midi kit		Alternative Extraction
	copies/ml	Serum copies/ml	Urine copies/ml	Serum copies/ml	Urine copies/ml	copies/reaz
RTS 1	35.700.000	30.000.000	20.000.000	9.000.000	11.250.000	1.500.000
RTS 2	3.570.000	3.000.000	20.000.000	900.000	1.125.000	150.000
RTS 3	357.000	300.000	20.000.000	90.000	112.500	15.000
RTS 4	35.700	30.000	20.000.000	9.000	11.250	1.500

When alternative systems are used sample concentration expressed in copies/ml will be obtained using the formula:

$$copie/ml = \frac{1000}{V_e} \times \frac{E_v}{E_a} \times C_{reaz}$$

where:

- **V_e**: extracted sample Volume expressed in μl
- **E_v**: eluted sample Volume during extraction stage expressed in μl
- **E_a**: extracted sample volume used for amplification expressed in μl
- **C_{reaz}**: copies provided by the instrument.

As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data

and the other laboratory tests done on the patient.

Given the close phylogenetic correlation between Usutu Virus and WNV, Usutu virus strongly positive patients could show cross reactivity with West Nile Virus. For this reason, a probe was inserted in the reaction to allow the correct identification of both viral targets WNV positive patients will show a single amplification signal in the FAM channel, dedicated to WNV. Strongly positive patients for Usutu Virus will present a double amplification signal: one in the FAM channel and one specific in the VIC channel, dedicated to Usutu Virus.

FAM Fluorescence	VIC Fluorescence	Result
Present	Absent	West Nile Virus Positive
Present	Present	Usutu Virus Positive
Absent	Absent	Negative

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results. The use of positive and negative controls in each amplification session allow to verify the correct functioning of the amplification mix and the absence of any contamination.

Thanks to the presence of Usutu Positive Control it is also possible to monitor the correct signal amplification for Usutu Virus.

In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from extraction process until detection step.

In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from reverse transcription process until detection step. Be sure that emitted fluorescence from internal control amplification has not a Ct > 35 or undetermined. If a sample shows an undetermined WNV RNA and an internal control Ct >35, this means that there have been problems in the extraction stage or in the retro-amplification stage; therefore the sample could be a false negative.

Repeat the sample.

It can be considered valid the samples with a Ct > 35 for the internal control, and a high concentration of WNV RNA. In this case, the competitive nature of PCR reaction can hide or disadvantage the internal control amplification.

Detector FAM	Detector CY5	PCR Run	Sample
Ct undetermined	Ct > 35 or undetermined	Not Valid	repeat
Ct undetermined	Ct < 35	Valid	Negative
High Ct	Ct < 35	Valid	Positive
Low Ct	Ct > 35 or undetermined	Valid	HighPositive

PERFORMANCES

Analytical sensitivity:

It is considered as analytical sensitivity the highest dilution (title) to which a positive sample can be diluted without the system losing the ability to detect with a positivity rate of ≥ 95%. The analytical sensitivity of the system was assessed by analyzing synthetic RNA, quantified by spectrophotometric analysis, containing the regions of interest (3'UTR) of the virus in serial dilutions.

Analytical sensitivity of **quany WNV** determined by Probit analysis

Probit Analysis		Manual Extraction cps/ml	EZ1 Automatic Extraction cps/ml	QIASymphony Automatic Extraction cps/ml
cps/μl	1,87 cps/μl	668 cps/ml	561 cps/ml	168 cps/ml
95% confidence interval	Lower 0,80 cps/μl Upper 22,16 cps/μl	Lower 286 cps/ml Upper 7913 cps/ml	Lower 240 cps/ml Upper 664 cps/ml	Lower 72 cps/ml Upper 1994 cps/ml

Probit Analysis		Manual Extraction cps/ml	EZ1 Automatic Extraction cps/ml	QIASymphony Automatic Extraction cps/ml
cps/μl	1,87 cps/μl	668 cps/ml	374 cps/ml	210 cps/ml
95% confidence interval	Lower 0,80 cps/μl Upper 22,16 cps/μl	Lower 286 cps/ml Upper 7913 cps/ml	Lower 160 cps/ml Upper 4431 cps/ml	Lower 90 cps/ml Upper 2492 cps/ml

Clinical sensitivity:

It is considered as clinical sensitivity the ability to detect true positive samples in the totality of the samples screened as positive. The analysis was made on WNV positive samples and the test was performed following the method recommendations. Positive samples were confirmed with an other CE approved method available on the market.

Obtained results show a clinical sensitivity of 100%: for RNA samples extracted from the following matrices: Serum and Urine.

Linearity/Proportionality

The system linearity was valued analyzing synthetic RNA (WNV-RNA), quantified by spectrophotometric analysis, containing the regions of interest 3'UTR of the virus in serial dilutions (1:10) from 1.500.000.000 copies/reactions to 15 copies/reaction of RNA in 15μl of extracted material added in the retro-amplification reaction.

The evaluation was performed analyzing 10 calibration curves, that showed these parameters:

RTS conc. 10 ⁵ copies (FAM)	Ct ≤ 25	Medium Ct 21.64
Correlation Coefficient	0.990 ≤ r ² ≤ 1	Medium r2 0,996
Slope	-3,6 ≤ Slope ≤ -3.1	Medium slope -3,291
PCR efficiency	90 ≤ Efficiency ≤ 110	Medium Eff 101%

Reproducibility and Repeatability

The reproducibility and repeatability of the system were valued analyzing 3 dilutions of synthetic RNA containing the 3'UTR region of interest for WNV and quantified by spectrophotometric analysis, plus a negative control (negative RNA).

For each session, 5 replicates were made in 3 different sessions, performed by different technicians, using 3 different lots of product.

Hypothetical value	Lot	N° repetitions	Med. Reveal. Conc.	Inaccuracy %
100.000	L1	15	106728	7%
100.000	L2	15	109441	9%
100.000	L3	15	105359	5%
1.000	L1	15	1026	3%
1.000	L2	15	1039	4%
1.000	L3	15	885	11%
10	L1	15	9,4	6%
10	L2	15	9,2	8%
10	L3	15	9,2	8%

The average Inaccuracy percentage of WNV method is 7%.

Diagnostic Specificity:

It is considered as diagnostic specificity the ability of the method to detect true negative samples. The diagnostic specificity of the system was evaluated analyzing human samples tested and confirmed as WNV negative with another CE marked method available on the market.

Diagnostic specificity is 100% for RNA samples extracted from the following matrices: Serum and Urine.

Analytical Specificity:

Test's specificity was guaranteed by the use of specific primers for WNV.

The alignment of the chosen regions for specific primers' hybridization for WNV with available sequences of the 3'UTR region present in database, demonstrated: their conservation, the absence of significant mutations for the analysed target.

The analysis "in Silico" showed how the alignment of primers and probes allows the correct identification of the Lineage 1-7.

Cross-Reactivity:

The exam of alignment of the region chosen for hybridization of primers specific for WNV with the sequences available in the bank data of the 3'UTR region showed their conservation, the absence of significant mutations and the complete specificity for the targets analyzed.

To check the cross-reactivity of the assay, samples tested as positive for other viruses were analysed following the method instructions.

Sample	Positive sample	Obtained Results cps/ml
1	ZIKA	< 100
2	Dengue	< 100
3	Yellow Fever Virus	< 100
4	Chikungunya Virus	< 100

Patients with Usutu virus strongly positive, could show a cross reactivity with West Nile Virus.

INTERFERENCES:

Verify that in the RNA extracted from the sample there is no contamination from mucoproteins and haemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorption of Nucleic Acids) and 280 nm (maximum absorption of Proteins). A pure RNA should have a ratio of approximately 2.

QUALITY CONTROL

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

In accordance with the Clonit srl ISO EN 13485 Certified quality Management System, each lot of **quany WNV** is tested against predetermined specification to ensure consistent product quality.

BIBLIOGRAPHY

Ana Vázquez, Laura Herrero, Anabel Negrodo, Lourdes Hernández, María Paz Sánchez-Seco, Antonio Tenorio. **Real time PCR assay for detection of all known lineages of West Nile virus.** Journal of Virological Methods 236 (2016) 266-270.

Thomas Briese, William G Glass, W Ian Lipkin. **Detection of West Nile virus sequences in cerebrospinal fluid.** THE LANCET • Vol 355 • May 6, 2000 p. 1614.

Department of Health and Human Services Centers for Disease Control and Prevention National Center for Emerging and Infectious Diseases Division of Vector-Borne Diseases. **West Nile Virus in the United States: Guidelines for Surveillance, Prevention, and Control**

June 14, 2013 4th Revision

Mattia Calzolari, Chiara Chiapponi, Paolo Bonilauri, Davide Lelli, Laura Baioni, Ilaria Barbieri, Antonio Lavazza, Stefano Pongolini, Michele Dottori, Ana Moreno. **Co-circulation of two Usutu virus strains in Northern Italy between 2009 and 2014.** Infection, Genetics and Evolution 51 (2017) 255-262

TECHNICAL ASSISTANCE

For any question and support please contact our Technical support:

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BIBLIOGRAFIA

Ana Vázquez, Laura Herrero, Anabel Negredo, Lourdes Hernández, María Paz Sánchez-Seco, Antonio Tenorio. **Real time PCR assay for detection of all known lineages of West Nile virus.** Journal of Virological Methods 236 (2016) 266-270.

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Department of Health and Human Services Centers for Disease Control and Prevention National Center for Emerging and Infectious Diseases Division of Vector-Borne Diseases. **West Nile Virus in the United States: Guidelines for Surveillance, Prevention, and Control** June 14, 2013 4th Revision

Mattia Calzolari, Chiara Chiapponi, Paolo Bonlauro, Davide Lelli, Laura Baioni, Ilaria Barbieri, Antonio Lavazza, Stefano Pongolini, Michele Dottori, Ana Moreno. **Co-circulation of two Usutu virus strains in Northern Italy between 2009 and 2014.** Infection, Genetics and Evolution 51 (2017) 255-262

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CLONIT S.r.l.