SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

quany Measles
Detection and quantification of the Measles Virus with Real Time PCR

REV ST.RT-06-ENITA.4

REF RT-06 - 48 tests

Instructions For Use

INTRODUCTION AND PURPOSE OF USE

The quany Measles system is a quantitative test that allows the RNA amplification, by means of Real Time PCR, of N3 gene of Measles Virus.

The Procedure allows the detection and quantification of the RNA target by means a One step retro-amplification reaction.

The analysis of the results is made using a Real Time PCR analyzer (thermal cycler integrated with a system for fluorescence detection and a dedicated software).

CONTENT

The kit contains reagents enough to perform 48 amplification tests:

	Quantity	Description
R1	3 x 10 µl	Enzyme mMix Reverse Transcriptase e Taq Polymerase enzymes (pink cap)
R2	3 x 270 µl	MeV probe Mix Upstream primer, downstream primer, Target probe MeV (VIC), Internal Control (IC) probe (CY5), Nuclease-free water, ROX dNTPs, Tris-HCl, KCl, MgCl2 (White cap)
R3	3 x 35 µl	synthetic RNA corresponding to N3 region of Measles 100.000 cps/µl (Blue cap)
R4	3 x 35 µl	synthetic RNA corresponding to N3 region of Measles 10.000 cps/µl (Green cap)
R5	3 x 35 µl	synthetic RNA corresponding to N3 region of Measles 1.000 cps/µl (Yellow cap)
R6	3 x 35 µl	synthetic RNA corresponding to N3 region of Measles 100 cps/µl (White cap)
R7	3 x 160 µl	Internal control (phage MS2) (Red cap)
R8	1 x 50 µl	Negative Control

MATERIALS AND STRUMENTATION REQUIRED BUT NOT SUPPLIED

Disposable latex powder-free gloves or similar material;
Bench microcentrifuge (12,000 - 14,000 rpm);
Micropipettes and Sterile tips with aerosol filter;
Vortex;
Plastic materials (microplate and optical adhesive cover);
Heat block (only for extraction)
EZ1 Advanced XL DSP Virus Card. - Ref. 9018703 - QIAGEN.

Reagents

The **quany Measles** kit was developed and validated to be used with the following extraction method:

Manual Extraction

Ref. 52906. *QIAmp Viral RNA Mini Kit*

The kit allows the manual RNA extraction from tested samples. The kit contains reagents for 250 samples. (QIAGEN)

Automatic extraction

Ref. 62724. *EZ1 XL DSP Virus Kit*.

The kit allows the automatic viral RNA from Human samples. The kit contains reagents for 48 samples. (QIAGEN)

Instruments

The **quany Measles** kit was evaluated to be used with the following instruments:

Extraction System

Ref. 9001492. *EZ1 Advanced XL*.

Robotic Workstation for the automatic purification of the nucleic acids until 14 samples simultaneously (QIAGEN)

Real Time PCR

The **quany Measles** kit was evaluated to be used to be used with the following real time PCR instruments:

- 7500 Fast from Lifetechnologies
- Rotor-Gene Q MDx from QIAGEN
- Light Cycler 480 II from Roche

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturer's instruction and recommendations.

SAMPLES

The **quany Measles** kit must be used with extracted RNA from biological samples: **Serum and Urine**. Collected samples must be shipped and stored at +2 - +8°C and used within 3 days from the collected data.

Store the sample at -20°C if it is used after 3 days.

PRECAUTIONS FOR USE

- This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.
- Carefully read this Instruction For Use before using the kit.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: RNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- Periodically wipe the working area with 0,5% hypochlorite.
- Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.
- Use powder-free gloves. Do not leave fingerprints on optical caps. Do not write on caps as this may cause an interference with fluorescent detection.
- Avoid any contact between hands and eyes or nose during specimen collection and testing.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood. Waste should be discarded according to local law.
- Never pipette solutions by mouth.
- Use calibrated and regularly checked pipettes and instrumentation only.
- Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.
- Wash hands carefully after handling samples and reagents.
- Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
- Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

LIMITS OF THE METHOD

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and/or controls. Therefore, you should:

-physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products

- use tips with filters to prevent cross-contamination between samples
- use disposable gloves and change them frequently
- carefully open test tubes to prevent aerosol formation
- close every test tube before opening another one

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

Patient Drug treatment may interfere with the final result of the molecular biology analysis.

The proper functioning of the amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

WARNINGS

- Read carefully the instructions for use before using this test.
- Use only extracted RNA from Serum and/or Urine
- Do not mix reagents from different lots.
- Thaw and carefully mix the reagents of the kit before use.
- The PCR mix has to be freshly prepared every time.

After reconstitution, the amplification master mix must be used in one time (16 reactions). Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.

- Do not use beyond the expiration date which appears on the package label.
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.

- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).

- In case of damaged package, contact the technical support before using the kit.

In case of any serious incident that has occurred in relation to the device, a notice shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

STORAGE AND STABILITY

Store the product **quany Measles** at **-20°C**.

The **quany Measles** kit is shipped on dry ice. The kit components should be frozen.

If one or more components are not frozen upon receipt or if the tubes have been compromised during transport, contact Clonit srl for assistance.

An intact and well stored product has a stability of 12 months from the date of production. Do not use beyond the expiration date which appears on the package label.

Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.

ANALYTICAL PROCEDURE

Manual Extraction

Ref. 52906. *QIAmp Viral RNA Mini Kit*

Procedure to Serum and Urine

Follow the instructions inside the kit *QIAmp Viral RNA Mini Kit*.

After the incubation period for 10 min at room temperature, add 5 µl of Internal Control

Follow the instructions inside the kit

Elute the sample in 50 µl of buffer AVE

Samples are now ready for amplification or storage at -20°C

Automatic extraction

Ref. 62724. EZ1 XL DSP Virus Kit

Procedure to Serum

Follow the instructions inside the kit *EZ1 XL DSP Virus Kit*.

Volume of sample to be used:

Serum [μ l]
200

Preparation of the Carrier and the Internal Control

Solve completely the lyophilize RNA carrier in elution buffer (AVE), from 310 μ l, split in aliquots and store to $-20 \pm 5^\circ\text{C}$. Not thawing and freezing the aliquots more than 2 times.

For each analyzed sample, dilute 3,6 μ l of a original solution include the RNA Carrier and 10 μ l of *Internal Control* in total volume of 60 μ l using elution buffer (AVE)

Select the protocol starting from 200 μ l of samples with the elution of 60 μ l.

Follow the instructions inside the kit *EZ1 XL DSP Virus Kit*.

Samples are now ready for amplification or storage at -20°C

Procedure to Urine

Before the reaction on EZ1 Advanced XL instrument mix.

Urine [μ l]	ATL [μ l]	final sample volume [μ l]
300	100	400

Preparation of the Carrier and the Internal Control

Solve completely the lyophilize RNA carrier in elution buffer (AVE), from 310 μ l split in aliquots and store to $-20 \pm 5^\circ\text{C}$. Not thawing and freezing the aliquots more than 2 times.

For each analyzed sample, dilute 3,6 μ l of a original solution include the RNA Carrier and 10 μ l of *Internal Control* in total volume of 60 μ l using elution buffer (AVE)

Follow the instructions inside the kit *EZ1 XL DSP Virus Kit*.

Select the protocol starting from 400 μ l of samples with the elution of 60 μ l.

Samples are now ready for amplification or storage at -20°C .

SOFTWARE SETTING

Lifetechnologies 7500 fast

Turn the instrument and the computer on and open the control software. Click on "**Advance Setup**": by default the software will shows the page "**experiment properties**". Write in the "**experiment name**" the file name, choose the type of instrument (**7500 o 7500fast**), the type of reaction (**quantitation standard curve**), the type of used reagent (**Taqman[®] Reagents**) and the reaction time of analysis (**Standard \approx 2 hours to complete a run**).

Open the page named "**page setup**" (sheet **Define Target and Samples**).

In the window **Define Targets** set:

Target	Reporter	Quencher
Measles probe:	VIC	NONE
Internal Control (IC) probe:	CY5	NONE

Set the samples' name in the window "**Define Samples**".

In the same page "**plate setup**" select the sheet "**Assign Target and Samples**". On the screen you will see the microplate draft. Select an area of the plate where the controls will be placed: select wells of the plate and set both targets (Measles and IC). Select "**Assign target to selected wells**" in the blank, the "**task Standard (S)**" for Measles target and set the controls' concentration.

Choose an area in the plate where negative control will be placed: select "**Assign target to selected wells**" in the blank, the "**task Negative (N)**" for the Measles target.

Select an area of the plate where samples will be placed: select the wells and set both targets (Measles and IC). Link every well to a sample, through the window "**Assign samples to selected wells**".

For each sample, select in the blank "**Assign targets to selected wells**" the "**task UnKnown (U)**" for the Measles target.

Set ROX as passive reference, using it as normalizer of detecting fluorescence.

Open "**Run Method**" (sheet **Graphic View**) and set the right thermal cycling with the "collect data" in annealing phase at 58°C .

cycles	Retrotranscription	annealing	extension
1	50°C 40 min		
1	95°C 2 min		
45	95°C 15 sec	58°C 45 sec	72°C 15 sec

In the window "**Reaction volume plate per well**" set a volume of 30 μ l.

Select the annealing phase from the thermal profile to collect the data.

After preparing the plate, and correctly inserting it in the instrument, press the button "**Start Run**".

Rotor Gene Q MDx

The experiments can be **set using the Quick Start Wizard or the Advanced Wizard**, which appears when the software is started.

Select the wizard "**Advanced**". As a first step, select the model "**Two Step Reaction**" with a double click in the "**New Run**".

In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "**Locking Ring Attached**", check the checkbox and then click "**Next**".

Enter the name of the operator and the reaction volume of 30 μ l, and then click "**Next**".

In the next window click on "**edit profile**". Set the following thermal cycle:

cycles	Retrotranscription	annealing	extension
1	50° C 40 min		
1		95°C 2 min	
45		95° C 15 sec	58° C 45 sec 72° C 15 sec

Select the annealing phase from the thermal profile and click on "**Acquiring A to cycling.**"

In the next window, select red and orange from the available channels and add it to acquiring channel along with the yellow channel and click "**OK**". In the next window click on "**OK**" and then click "**Next**".

Click on "Edit Gain" button and set the following values for each channel:

Reporter	Gain
Yellow	5
Red	5
Orange	5

To begin the course, click on the button "**Start Run**". You can save the model before you begin your run by clicking on "**Save Template**".

After clicking on the button "**Start Run**" window appears "**Save As**". The stroke can be saved in the desired position by the user. Once the run started, the window "**Edit Samples**" allows you to set the name of samples and controls in the positions in which they were loaded on the instrument.

Select the locations where they were positioned the controls of known concentration and designate them as Measles Standard. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Standards**". Enter the concentrations of the controls.

Select the location where you placed the Negative Control and name it as Negative Control. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Negative Controls**".

Select the location of each sample and enter the name or code of the patient. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Unknown**".

At the end of the operation click "**OK**" in the "**edit samples**" and wait until the end of the race for the analysis (see "**Interpretation of Results**").

LightCycler 480

Turn the instrument and the computer on and start the control software. In the principal screen, select "**Open Tools**" (wrench image), click on "**Detection formats**" and set a NEW filter combination selection: flag the wavelength dedicated 553-580 (VIC) and 618-660 (CY5) and give the Name "**VIC CY5**".

Target	Reporter
Measles probe:	VIC
Internal Control (IC) probe:	CY5

On "**Experiment Creation**" push the "**New experiment**" button. The window "experiment" appears. On the "Run protocol" sheet set:

1- Detection format: VIC-CY5

2- Reaction volume: 30 µl

3- thermal profile, with signal acquisition in the annealing phase:

Cycles	Retrotranscription	Denaturation	Annealing	Extension
1	50° C 40 min			
1		95°C 2 min		
45		95° C 15 sec	58° C 45 sec	72° C 15 sec

Push the "**Subset editor**" button and in this window. Create a New Subset clicking in (+) button and rename it as Measles. Select an area of the plate where controls and samples will be placed and click **Apply**.

Push the "**Sample editor**" button. Select the correct workflow (Step1: **Abs quantification**), choose the samples Subset created in the step before and insert the name for each well. Choose the correct type for each well: Standard, Negative CTR or Unknown sample. Push again the "**experiment button**", insert the plate in the instrument and push "**start run**".

PREPARATION OF THE REACTIONS:

Thaw a tube of **Enzyme mMix**;

Thaw a tube of **MeV probes Mix**;

Mix carefully by vortex **9µl** of **Enzyme mMix** and **260µl** of **MeV probes Mix** (the mix as produced is enough to prepare **16 reactions** of amplification: **4 positive controls, 1 negative control and 11 samples**).

For a total number of tests other than 16, prepare a mixture for n + 1 tests, following the volumes shown in the table:

Reagent	Description	Volume for single test (µl)
R1	Enzyme mMix	0,5
R2	MeV probes Mix	14,5

Distribute, in the amplification plate, **15µl of just reconstituted mix** in chosen positions as already set on the instrument software.

Distribute, in the negative control position, **15 µl** of solution taken by the negative control vial.

Distribute, in chosen position for each sample, **15 µl** of corresponding sample.

Distribute, in chosen positions for the positive controls, **15µl** of **10² copies/µl, 10³ copies/µl, 10⁴ copies/µl and 10⁵ copies/µl**.

Seal up accurately the plate using an optical adhesive film and verify that there aren't air bubbles in the mix, to avoid the amplification interferences.

For the Rotor-Gene Q MDx, seal each tube with the appropriate cap. The air bubbles presence is not influent; the rotor centrifugal force will allow automatic deletion.

Transfer the plate in the instrument and push the button "**Start Run**".

QUANTITATIVE ANALYSIS
Lifetechnologies 7500 Fast

At the end of the PCR run, the software automatically opens the "Analysis" window in the "Amplification plot" sheet on the menu on the left.

Select the wells corresponding to the positive control, negative control and samples for analysis.

Select in the "Option" window inside the "Target" pop-up menu the **Measles target**. Check the correct setting of the threshold.

Select in the "Option" window inside the "Target" pop-up menu the **IC Control target**. Check the correct setting of the threshold.

The analysis of the results is made selecting from the menu in the left the page "Analysis". From the page "Standard Curve", maintaining open the sheet "view well plate" in the right side of the software select the wells containing the points of the curve and verify the parameters described in the paragraph "Interpretation of the Results" (coefficient of correlation, slope ecc...).

From the page "Amplification Plot" verify the amplification plot for every single sample.

Opening the sheet "view well table" in the right side of the software it is possible to verify the data obtained from experiments: Threshold Cycles, emitted fluorescence and the target quantification expressed in copies/reaction or copies/ml, depending on the settings of the calibration curve. Clicking from the menu file and selecting the box export, the window "export properties" will open.

Indicate the file name, select the position to save it (Browse) and click on button

"Start export". In this way the software will permit to save a excel file with all the data corresponding to selected experiment.

Rotor Gene-Q MDx

At the end of the PCR run click "Option" button under the amplification plot on yellow channel. Select "normalize to Cycling A. orange". At the end of the PCR run open the "Analysis" window. Select the "Quantification" sheet and click on "cycling A.Yellow/cycling A.orange". Check the correct setting of the threshold in the space provided "CT calculation - Threshold" and delete the first 10 cycles from the interpolation of the Threshold by entering the value 10 in the "Eliminate Cycles Before" box.

Select from the menu "Dynamic Tube" and subsequently "Slope correct". If necessary, reduce the background noise of non-specific signals by selecting the "Outlier Removal" option and entering a value between 5% and 30%.

Repeat the steps above for each of the fluorescence channels "cycling A (red)".

You can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification cycling A (yellow) and then the file cycling A (red).

LightCycler 480

When the run is completed select analysis and choose the correct kind of analysis you want: "Abs Quant/Fit Points". Choose the samples subset you want to analyze. Select the "NoiseBand" sheet, under the plot you can choose "NoiseBand (Fluoresc.)"; and move the line of the NoiseBand on the plot with the mouse of your PC. Repeat this action for each fluorophore using the "Filter comb" button.

Clicking the sheet "Analysis" you can set the threshold choosing the option "Threshold (manual)".

After setting parameters push the "Calculate" button. Repeat this action for each fluorophore.

INTERPRETATION OF RESULTS

Through the Real Time PCR reaction, it is possible to obtain the RNA quantification of Measles RNA, by setting the values of the positive controls of the calibration curve. To calculate these values, all the dilutions steps that the sample has undergone during the extraction and amplification stages must be considered.

The Ct values obtained from the amplification of 4 controls of known titre are used by the software for the calculation of the calibration curve from which the unknown samples are interpolated.

A proper functioning of the amplification mix can be verified analyzing these parameters:

Parameters	Reference
RTS conc. 10 ⁵ copie/μl (VIC)	Ct ≤ 22
Correlation Coefficients	0.990 ≤ r ² ≤ 1
Slope	-3.6 ≤ Slope ≤ -3.1
PCR Efficiency	90 ≤ Efficiency ≤ 110

If the RTS amplification reaction at a concentration of 10⁵ copies/μl produces a Ct > 22 or undetermined the session can't be considered valid and must be repeated.

Verify that the correlation coefficient value (r²), the slope or the reaction efficiency fit to the limited indicated in the above table or do not deviate much from them, which represent the ideal range for a proper PCR reaction.

By correctly setting the standards concentration as a function of the extraction system you can get the quantization of the sample directly in copies / ml:

	Manual Extraction	Automatic Extraction	Alternative Extraction
	QIAmp Viral RNA mini kit	EZ1 Advanced XL EZ1 XL DSP Virus Kit	
	copies/ml		Copie/react
RTS 1	35.700.000	30.000.000	1.500.000
RTS 2	3.570.000	3.000.000	150.000
RTS 3	357.000	300.000	15.000
RTS 4	35.700	30.000	1.500

When alternative systems are used sample concentration expressed in copies/ml will be obtained using the formula:

$$copie/ml = \frac{1000}{V_e} \times \frac{E_v}{E_a} \times C_{reaz}$$

where:

- **Ve**: extracted sample Volume expressed in μl
- **Ev**: eluted sample Volume during extraction stage expressed in μl
- **Ea**: extracted sample volume used for amplification expressed in μl
- **Creaz**: copies provided by the instrument.

As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data and the other laboratory tests done on the patient.

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

The use of positive and negative controls in each amplification session allow to verify the correct functioning of the amplification mix and the absence of any contamination.

In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from extraction process until detection step.

In the amplification reaction of each sample, the Ct values for the internal control specific probe are used to validate the analysis session, from reverse transcription process until detection step.

Be sure that emitted fluorescence from internal control amplification has not a Ct > 35 or undetermined. If a sample shows an undetermined Measles RNA and an internal control Ct >35, this means that there have been problems in the extraction stage or in the retro-amplification stage; therefore the sample could be a false negative.

Repeat the sample.

It can be considered valid the samples with a Ct > 35 for the internal control, and a high concentration of Measles RNA. In this case, the competitive nature of PCR reaction can hide or disadvantage the internal control amplification.

Detector VIC	Detector CY5	PCR Run	Sample
Ct undetermined	Ct > 35 or undetermined	Not Valid	repeat
Ct undetermined	Ct < 35	Valid	Negative
High Ct	Ct < 35	Valid	Positive
Low Ct	Ct > 35 or undetermined	Valid	High Positive

PERFORMANCES

Analytical sensitivity:

It is considered as analytical sensitivity the highest dilution (title) to which a positive sample can be diluted without the system losing the ability to detect with a positivity rate of $\geq 95\%$. The analytical sensitivity of the system was assessed by analyzing synthetic RNA, quantified by spectrophotometric analysis, containing the regions of interest (N3) of the virus in serial dilutions.

Analytical sensitivity of **quanty Measles** determined by Probit analysis

Probit Analysis		Manual Extraction	EZ1 Automatic Extraction	
		cps/ml	cps/ml	
		Serum/Urine	Serum	Urine
cps/ μ l	0,27 cps/ μ l	96,42 cps/ml	81 cps/ml	40,5 cps/ml
95% confidence interval	Lower 0,09 cps/ μ l Upper 4,03 cps/ μ l	Lower 32,14cps/ml Upper 1439 cps/ml	Lower 27 cps/ml Upper 1209 cps/ml	Lower 13,5 cps/ml Upper 604 cps/ml

Traceability versus controls material

Amplirun Measles RNA control

To validate the traceability of the **quanty Measles** versus Amplirun Measles RNA control "Vircell" reference material.

The used control Amplirun Measles RNA control (rev. 06/2014), 1 vial with lyophilized RNA of Measles Virus (Edmonston strain), the concentration will be more than 12.500-20.000 copies/ μ l once reconstituted.

Linearity/Proportionality

The system linearity was valued analyzing synthetic RNA (MV-RNA), quantified by spectrophotometric analysis, containing the regions of interest (N3) of the virus in serial dilutions (1:10) from 15.000.000 copies/reactions to 150 copies/reaction of RNA in 15 μ l of extracted material added in the retro-amplification reaction.

The evaluation was performed analyzing 10 calibration curves, that showed these parameters:

RTS conc. 10^5 copies (VIC)	Ct ≤ 22	medium Ct 19,336
Correlation Coefficient	$0,990 \leq r^2 \leq 1$	medium r^2 0,999
Slope	$-3,6 \leq \text{Slope} \leq -3,1$	medium slope. -3,361
PCR efficiency	$90 \leq \text{Efficiency} \leq 110$	medium Eff. 97%

Reproducibility and Repeatability

The reproducibility and repeatability of the system were valued analyzing 3 dilutions of synthetic RNA containing the N3 region of interest for Measles and quantified by spectrophotometric analysis, plus a negative control (negative RNA).

For each session, 5 replicates were made in 3 different sessions, performed by different technicians, using 3 different lots of product.

Hypothetical value	Lot	N° repetitions	Med. Reveal. Conc.	Inaccuracy %
100.000 copie	Lot 1	30	93.823	6%
100.000 copie	Lot 2	30	100.495	0%
100.000 copie	Lot 3	30	105.020	5%
100.000 copie	Lot 19	15	94.914	5%
100.000 copie	Lot 20	15	105.531	6%
100.000 copie	Lot 21	15	111.256	11%
1.000 copie	Lot 1	30	1.010	7%
1.000 copie	Lot 2	30	1.073	7%
1.000 copie	Lot 3	30	1.028	3%
1.000 copie	Lot 19	15	1.082	8%
1.000 copie	Lot 20	15	1.073	7%
1.000 copie	Lot 21	15	1.028	3%
10 copie	Lot 1	30	10	4%
10 copie	Lot 2	30	11	7%
10 copie	Lot 3	30	10	2%
10 copie	Lot 19	15	11	10%
10 copie	Lot 20	15	9	8%
10 copie	Lot 21	15	9	6%

The average Inaccuracy percentage of Measles method is 6%.

Diagnostic Specificity:

It is considered as diagnostic specificity the ability of the method to detect true negative samples. The diagnostic specificity of the system was evaluated analyzing human samples tested and confirmed as negative with another CE marked method available on the market.

Diagnostic specificity is 100% for RNA sample extracted

Analytical Specificity:

Test's specificity was guaranteed by the use of specific primers for Measles.

The alignment of the chosen regions for specific primers' hybridization for Measles with available sequences of the N3 gene present in database, demonstrated: their conservation, the absence of significant mutations and the complete specificity for the analysed target.

Analytical Specificity:

Test's specificity was guaranteed by the use of specific primers for Measles.

The alignment of the chosen regions for specific primers' hybridization for Measles with available sequences of the N3 gene present in database, demonstrated: their conservation, the absence of significant mutations and the complete specificity for the analysed target.

The analysis "in Silico" showed how the alignment of primers and probes allows the correct identification of the following subtypes :

- Genotype B2
- Genotype B3
- Genotype D4
- Genotype D8
- Genotype D9
- Genotype G3
- Genotype H1

Cross-Reactivity:

The exam of alignment of the region chosen for hybridization of primers specific for Measles Virus with the sequences available in the bank data of the N3 region showed their conservation, the absence of significant mutations and the complete specificity for the targets analyzed.

To check the cross-reactivity of the assay, samples tested as positive for other viruses were analysed following the method instructions.

Sample	Sample Positive	Result cps/ml
1	Rubella Virus	< 100
2	Respiratory Syncytial Virus A	< 100
3	Respiratory Syncytial Virus B	< 100

The analysis "in Silico" of the chosen regions for specific primers' hybridization for Measles with available sequences of the N3 gene present in database, didn't show annealing that justifies the cross reactivity

Sample	Sample Positive	Result cps/ml
1	Mump Virus	< 100
2	Parainfluenza Virus	< 100
3	Metapneumovirus	< 100

INTERFERENCES:

Verify that in the RNA extracted from the sample there is no contamination from mucoproteins and haemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorption of Nucleic Acids) and 280 nm (maximum absorption of Proteins). A pure RNA should have a ratio of approximately 2.

QUALITY CONTROL

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

Each batch of quantity Measles was produced according to predetermined specifications in order to ensure compliance with the ISO EN 13485 certified quality management system

BIBLIOGRAPHY

Kimberly B. Hummel, Luis Lowe, William J. Bellini, Paul A. Rota. **Development of quantitative gene-specific real-time RT-PCR assays for the detection of measles virus in clinical specimens.** *Journal of Virological Methods* 132 (2006) 166-173.

Felicia R., Lillian M., Joanne H., Rebecca J.M., Bettina B., Sarah C., Amy L., Nicole F., Annette M., Paul A. R., Alberto S.

Rapid Identification of Measles Virus Vaccine Genotype by Real-Time PCR *Journal of Clinical Microbiology*, March 2017 Volume 55 Issue 3

World Health Organization. **Global measles and rubella laboratory network---update.** *Wkly Epidemiol Rec* 2005;80:384--8.

World Health Organization. **Manual for the laboratory diagnosis of measles and rubella infection**, 2nd ed. Geneva, Switzerland: World Health Organization; 2007. WHO/IVB/07.01.

TECHNICAL ASSISTANCE

For any question and support please contact our Technical support:

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LightCycler 480

When the run is completed select analysis and choose the correct kind of analysis you want: **"Abs Quant/Fit Points"**. Choose the samples subset you want to analyze. Select the **"NoiseBand"** sheet, under the plot you can choose **"NoiseBand (Fluoresc.)"**; and move the line of the NoisBand on the plot with the mouse of your PC. Repeat this action for each fluorophore using the **"Filter comb"** button.

Clicking the sheet **"Analysis"** you can set the threshold choosing the option "Threshold (manual)".

After setting parameters push the **"Calculate"** button. Repeat this action for each fluorophore.

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BIBLIOGRAFIA

Kimberly B. Hummel, Luis Lowe, William J. Bellini, Paul A. Rota. **Development of quantitative gene-specific real-time RT-PCR assays for the detection of measles virus in clinical specimens.** *Journal of Virological Methods* 132 (2006) 166–173.

Felicia R., Lillian M., Joanne H., Rebecca J.M., Bettina B., Sarah C., Amy L., Nicole F., Annette M., Paul A. R., Alberto S. **Rapid Identification of Measles Virus Vaccine Genotype by Real-Time PCR** *Journal of Clinical Microbiology*, March 2017 Volume 55 Issue 3

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World Health Organization. **Manual for the laboratory diagnosis of measles and rubella infection**, 2nd ed. Geneva, Switzerland: World Health Organization; 2007. WHO/IVB/07.01

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CLONIT S.r.l.