SZABO SCANDIC

Part of Europa Biosite

Produktinformation

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mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

quany Enterovirus (5'UTR region)
Detection and quantification of the Enterovirus with Real Time PCR

REV ST.RT-33-ENITA.3

REF RT-33 - 48 tests

Instructions For Use

INTRODUCTION AND PURPOSE OF USE

The **quany Enterovirus** system is a quantitative test that allows the RNA amplification and quantification, by means of *Real Time PCR*, of 5'UTR region of Enterovirus RNA.

The Procedure allows the detection of the RNA target by means a retro-amplification reaction.

The analysis of the results is made using a Real Time PCR analyzer (thermal cycler integrated with a system for fluorescence detection and a dedicated software).

CONTENT

The kit contains reagents enough to perform 48 amplification tests:

| | Quantity | Description |
|----|------------|---|
| R1 | 3 x 10 µl | Enzyme mMix Reverse Transcriptase e Taq Polymerase enzymes (Pink cap) |
| R2 | 3 x 270 µl | Enterovirus probe Mix Upstream primer, downstream primer, Target probe (FAM), Internal Control (IC) probe (VIC), Nuclease-free water, ROX dNTPs, Tris-HCl, KCl, MgCl ₂ (White Cap) |
| R3 | 3 x 45 µl | synthetic RNA corresponding to 5'UTR gene 100.000 cps/µl (Blue cap) |
| R4 | 3 x 45 µl | synthetic RNA corresponding to 5'UTR gene 10.000 cps/µl (Green cap) |
| R5 | 3 x 45 µl | synthetic RNA corresponding to 5'UTR gene 1.000 cps/µl (Yellow Cap) |
| R6 | 3 x 45 µl | synthetic RNA corresponding to 5'UTR gene 100 cps/µl (White cap) |
| R7 | 3 x 160 µl | Internal control phage MS2 (Red Cap) |
| R8 | 1 x 50 µl | Negative Control |

MATERIALS AND STRUMENTATION REQUIRED BUT NOT SUPPLIED

Disposable latex powder-free gloves or similar material;
 Bench microcentrifuge (12,000 - 14,000 rpm);
 Micropipettes and Sterile tips with aerosol filter;
 Vortex;
 Plastic materials (microplate and optical adhesive cover);
 Heat block (only for extraction)
 EZ1 Advanced XL DSP Virus Card. - Ref. 9018703 - QIAGEN.

Reagents

The **quany Enterovirus** kit was developed and validated to be used with the following extraction method:

Manual Extraction

Ref. 52906. *QIAmp Viral RNA Mini Kit*

The kit allows the manual RNA extraction from tested samples. The kit contains reagents for 250 samples. (QIAGEN)

Automatic extraction

Ref. 62724. *EZ1 XL DSP Virus Kit*.

The kit allows the automatic viral RNA from Human samples.

The kit contains reagents for 48 samples. (QIAGEN)

Instruments

The **quany Enterovirus** kit was evaluated to be used with the following instruments:

Extraction System

Ref. 9001492. *EZ1 Advanced XL*.

Robotic Workstation for the automatic purification of the nucleic acids until 14 samples simultaneously (QIAGEN)

Real Time PCR

The **quany Enterovirus** kit was evaluated to be used with the following real time PCR instruments:

- 7500 *Fast* from Lifetechnologies
- *Versant kPCR AD* from Siemens o *Stratagene MX3005P/MX3000P*
- *Rotor-Gene Q MDx* from QIAGEN
- *CFX96 Real Time PCR System* from Bio-Rad
- *LightCycler 480* from Roche

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturer's instruction and recommendations.

SAMPLES

The quantity Enterovirus kit must be used with extracted RNA from biological samples: Plasma and CSF. Collected samples must be shipped and stored at +2 - +8°C and used within 3 days from the collected data.

PRECAUTIONS FOR USE

- This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.
- Carefully read this Instruction For Use before using the kit.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: RNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- Periodically wipe the working area with 0,5% hypochlorite.
- Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.
- Use powder-free gloves. Do not leave fingerprints on optical caps. Do not write on caps as this may cause an interference with fluorescent detection.
- Avoid any contact between hands and eyes or nose during specimen collection and testing.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood. Waste should be discarded according to local law.
- Never pipette solutions by mouth.
- Use calibrated and regularly checked pipettes and instrumentation only.
- Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.
- Wash hands carefully after handling samples and reagents.
- Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
- Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

LIMITS OF THE METHOD

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and/or controls. Therefore, you should:

- physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products
- use tips with filters to prevent cross-contamination between samples
- use disposable gloves and change them frequently
- carefully open test tubes to prevent aerosol formation
- close every test tube before opening another one

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results. Patient Drug treatment may interfere with the final result of the molecular biology analysis.

The proper functioning of the amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

WARNINGS

- Read carefully the instructions for use before using this test.
- Use only extracted RNA from CSF and/or Plasma
- Do not mix reagents from different lots.
- Thaw and carefully mix the reagents of the kit before use.
- The PCR mix has to be freshly prepared every time.

After reconstitution, the amplification master mix must be used in one time (16 reactions). Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently.

- Do not use beyond the expiration date which appears on the package label.
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- In case of damaged package, contact the technical support before using the kit.
- In case of any serious incident that has occurred in relation to the device, a notice shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

STORAGE AND STABILITY

Store the product quantity Enterovirus at -20°C.

The quantity Enterovirus kit is shipped on dry ice. The kit components should be frozen.

If one or more components are not frozen upon receipt or if the tubes have been compromised during transport, contact Clonit srl for assistance. An intact and well stored product has a stability of 12 months from the date of production. Do not use beyond the expiration date which appears on the package label. Repeat thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The individual reagents should be frozen in aliquots, if they are to be used intermittently

ANALYTICAL PROCEDURE

Manual Extraction

Ref. 52906. *QIAmp Viral RNA Mini Kit*

Procedure to CSF and Plasma

Follow the instructions inside the kit *QIAmp Viral RNA Mini Kit*.

After the incubation period for 10 min at room temperature, add 5 µl of Internal Control

Follow the instructions inside the kit

Elute the sample in 50 µl of buffer AVE.

Samples are now ready for amplification or storage at -80°C

Automatic extraction

Ref. 62724. *EZ1 XL DSP Virus Kit*

Procedure to CSF and Plasma

Follow the instructions inside the kit *EZ1 XL DSP Virus Kit*.

Volume of sample to be used:

| CSF [μ l] | Plasma [μ l] |
|----------------|-------------------|
| 200 | 200 |

Preparation of the Carrier and the Internal Control

Solve completely the lyophilize RNA carrier in elution buffer (AVE), from 310 μ l, split in aliquots and store to $-20 \pm 5^\circ\text{C}$. Not thawing and freezing the aliquots more than 2 times.

For each analyzed sample, dilute 3,6 μ l of a original solution include the RNA Carrier and 10 μ l of *Internal Control* in total volume of 60 μ l using elution buffer (AVE)

Select the protocol starting from 200 μ l of samples with the elution of 60 μ l.

Samples are now ready for amplification or storage at -80°C

SOFTWARE SETTING

Lifetechnologies 7500 fast

Turn the instrument and the computer on and open the control software. Click on "**Advance Setup**": by default the software will shows the page "**experiment properties**". Write in the "**experiment name**" the file name, choose the type of instrument (**7500 o 7500fast**), the type of reaction (**quantitation standard curve**), the type of used reagent (**Taqman®Reagents**) and the reaction time of analysis (**Standard \approx 2 hours to complete a run**).

Open the page named "**page setup**" (sheet **Define Target and Samples**). In the window **Define Targets** set:

| Target | Reporter | Quencher |
|-------------------------------------|----------|----------|
| Enterovirus probe: | FAM | TAMRA |
| Internal Control (IC) probe: | VIC | TAMRA |

Set the samples' name in the window "**Define Samples**".

In the same page "**plate setup**" select the sheet "**Assign Target and Samples**". On the screen you will see the microplate draft.

Select an area of the plate where the controls will be placed: select wells of the plate and set both targets (Enterovirus and IC). Select

"**Assign target to selected wells**" in the blank, the "**task Standard (S)**" for Enterovirus target and set the controls' concentration.

Choose an area in the plate where negative control will be placed: select "**Assign target to selected wells**" in the blank, the "**task**

Negative (N)" for the Enterovirus target. Select an area of the plate where samples will be placed: select the wells and set both

targets (Enterovirus and IC). Link every well to a sample, through the window "**Assign samples to selected wells**".

For each sample, select in the blank "**Assign targets to selected wells**" the "**task UnKnown (U)**" for the Enterovirus target.

Set ROX as passive reference, using it as normalizer of detecting fluorescence.

Open "**Run Method**" (sheet **Graphic View**) and set the right thermal cycling, selecting "collect data" in the annealing phase.

| cycles | denaturation | annealing | extension |
|--------|--------------|--------------|--------------|
| 1 | 50° C 20 min | | |
| 1 | 95° C 2 min | | |
| 45 | 95° C 15 sec | 58° C 45 sec | 72° C 15 sec |

In the window "**Reaction volume plate per well**" set a volume of 30 μ l. After preparing the plate, and correctly inserting it in the instrument, press the button "**Start Run**".

Versant kPCR AD or Stratagene MX3005P/MX3000P

Turn the instrument on and wait until both green lamps have fixed light, turn on the computer and start the control software. In the main screen will appear the window "**New Experiment Options**": select "**Experiment type**": **quantitative PCR (Multiple Standard)**. Turn the lamp on 20 minutes before doing a new experiment. For turning the lamp on, click on the icon of the lamp in

the tool bar or select "**Lamp On**" from the menu "**Instruments**". Verify the right setting of the gain of the fluorescent reporters: in

the menu of settings, choose: "**Instrument**" and then "**Filter set gain setting**".

| Reporter | Gain |
|----------|------|
| FAM | 4 |
| HEX | 4 |
| ROX | 1 |

Click on button "**setup**" in the tool bar and choose "**Plate Setup**".

Sign the wells corresponding to calibrators. Define the calibrator's positions in right menu, setting:

| Well type: | Collect Fluorescent Data: | Reference Dye: | Replicate Symbol: |
|------------|---------------------------|----------------|-------------------|
| Standard | FAM/HEX/ROX | ROX | None |

Clicking on every single well, the window "**well information**" will appear: choose the name of the calibrator.

In the window "**Select Quantity**", set the concentrations of the 4 calibrators, following the instructions indicated in the paragraph

Interpretation of the results. Sign the wells correspondent to Negative control. Define the NTC positions in right menu, setting:

| Well type: | Collect Fluorescent Data: | Reference Dye: | Replicate Symbol: |
|------------|---------------------------|----------------|-------------------|
| NTC | FAM/HEX/ROX | ROX | None |

Clicking on every single well, the window "**well information**" will appear: set NTC as the name. Sign the wells correspondent to the Samples. Define the samples positions in right menu, setting:

| Well type: | Collect Fluorescent Data: | Reference Dye: | Replicate Symbol: |
|------------|---------------------------|----------------|-------------------|
| UnKnown | FAM/HEX/ROX | ROX | None |

Clicking on every single well, the window "**well information**" will appear: set the name or the code of the sample.

It's possible, indeed, set near the name of fluorescent reporter the name of analyzed targets:

| FAM | HEX |
|-------------|------------------|
| Enterovirus | Internal Control |

In the tool bar choose the sheet "**Thermal Profile Setup**", set the correct thermal cycle and reading the fluorescence in the annealing step:

| cycles | denaturation | annealing | extension |
|--------|--------------|--------------|--------------|
| 1 | 50° C 20 min | | |
| 1 | 95° C 2 min | | |
| 45 | 95° C 15 sec | 58° C 45 sec | 72° C 15 sec |

After preparing the plate and inserting it in the instrument, press the button "Run", selecting the sheet Thermal profile status and check the correctness of thermal profile.

Select the box **Turn Lamp Off** at the end of execution. Push the button Start: the software will ask you to indicate the name of saved file. The analysis will start.

Rotor Gene Q MDx

The experiments can be **set using the Quick Start Wizard or the Advanced Wizard**, which appears when the software is started.

Select the wizard "**Advanced**". As a first step, select the model "**Two Step Reaction**" with a double click in the "**New Run**".

In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "**Locking Ring Attached**", check the checkbox and then click "**Next**".

Enter the name of the operator and the reaction volume of 30 µl, and then click "**Next**". In the next window click on "**edit profile**".

Set the following thermal cycle:

| cycles | denaturation | annealing | extension |
|---------------|---------------------|------------------|------------------|
| 1 | 50° C 20 min | | |
| 1 | 95° C 2 min | | |
| 45 | 95° C 15 sec | 58° C 45 sec | 72° C 15 sec |

Select the annealing phase from the thermal profile and click on "**Acquiring A to cycling**."

In the next window, select yellow and orange from the available channels and add it to acquiring channel along with the green channel and click "**OK**". In the next window click on "**OK**" and then click "**Next**". Click on "Edit Gain" button and set the following values for each channel:

| Reporter | Gain |
|-----------------|-------------|
| Green | 5 |
| Yellow | 5 |
| Orange | 5 |

To begin the course, click on the button "**Start Run**". You can save the model before you begin your run by clicking on "**Save Template**".

After clicking on the button "**Start Run**" window appears "**Save As**". The stroke can be saved in the desired position by the user.

Once the run started, the window "**Edit Samples**" allows you to set the name of samples and controls in the positions in which they were loaded on the instrument.

Select the locations where they were positioned the controls of known concentration and designate them as Enterovirus Standard. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Standards**". Enter the concentrations of the controls.

Select the location where you placed the Negative Control and name it as Negative Control. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**Negative Controls**".

Select the location of each sample and enter the name or code of the patient. Clicking on the box next to "**Type**" correspondent, in the dropdown menu "**Samples**" you can select the type of sample being analyzed. Select "**UnKnown**". At the end of the operation click "**OK**" in the "**edit samples**" and wait until the end of the race for the analysis (see "**Interpretation of Results**").

CFX96 Real Time PCR

Turn the instrument and the computer on and start the control software. In the principal screen will appear the window "**Startup wizard**": select "**CFX96**" and press "**ok**". In the next window push "**create new**" and set the thermal protocol and the reaction volume (30µl), with the "collect data" in annealing phase:

| cycles | denaturation | annealing | extension |
|---------------|---------------------|------------------|------------------|
| 1 | 50° C 20 min | | |
| 1 | 95° C 2 min | | |
| 45 | 95° C 15 sec | 58° C 45 sec | 72° C 15 sec |

Save the protocol and click the next button. The software will open in default the sheet "**plate**". Click "**create new**", select "**Fluorophores button**" to choose fluorophores (FAM and VIC). Select the locations where they were positioned the controls of known concentration and choose the "**Sample Type Standards**".

Click "**Load**" check boxes to load fluorophores and Type or select Target Name. In the box "**load concentration**" set the concentrations of the 4 calibrators, following the instructions indicated in the paragraph **Analysis of the results**.

Select the location where you placed the Negative Control. Choose the "**Sample Type NTC**". Click "**Load**" check boxes to load fluorophores and Type or select Target Name. Select the location of each sample and enter the name or code of the patient. Choose the "**Sample Type Unknown**". Click "**Load**" check boxes to load fluorophores and Type or select Target Name. Save the plate clicking the next button and start the experiment

LightCycler 480

Turn the instrument and the computer on and start the control software. In the principal screen, on "**Experiment Creation**" select the "**Plate type**" and push the "**New experiment**" button. The window "experiment" appears. On the "Run protocol" sheet set: Thermal protocol, Reaction volume (30µl) and Detection format (**dual colour hydrolysis probe**), with the "collect data" in annealing phase:

| cycles | denaturation | annealing | extension |
|---------------|---------------------|------------------|------------------|
| 1 | 50° C 20 min | | |
| 1 | 95° C 2 min | | |
| 45 | 95° C 15 sec | 58° C 45 sec | 72° C 15 sec |

Push the "**Subset editor**" button and in this window, select an area of the plate where controls and samples will be placed.

Push the "**Sample editor**" button. Select the correct workflow (Step1: **Abs quantification**), choose the samples Subset created in the step before and insert the name for each well. Choose the correct type for each well: Positive CTR, Negative CTR or Unknown sample. Push again the "**experiment button**", insert the plate in the instrument and push "**start run**".

PREPARATION OF THE REACTIONS:

Thaw a tube of **Enzyme mMix**;

Thaw a tube of **Enterovirus probes Mix**;

Mix carefully by vortex **9µl of Enzyme mMix** and **260µl of Enterovirus probes Mix** (the mix as produced is enough to prepare **16 reactions** of amplification: **4 positive controls, 1 negative control and 11 samples**).

For a total number of tests other than 16, prepare a mixture for n + 1 tests, following the volumes shown in the table:

| Reagent | Description | Volume for single test (µl) |
|---------|------------------------|-----------------------------|
| R1 - | Enzyme mMix | 0,5 |
| R2 | Enterovirus probes Mix | 14,5 |

Distribute, in the amplification plate, **15µl of just reconstituted mix** in chosen positions as already set on the instrument software. Distribute, in the negative control position, **15 µl** of solution taken by the negative control vial. Distribute, in chosen position for each sample, **15 µl** of corresponding sample. Distribute, in chosen positions for the positive controls, **15µl of 10² copies/µl, 10³ copies/µl, 10⁴ copies/µl and 10⁵ copies/µl**. Seal up accurately the plate using an optical adhesive film and verify that there aren't air bubbles in the mix, to avoid the amplification interferences. For the Rotor-Gene Q MDx, seal each tube with the appropriate cap. The air bubbles presence is not influent; the rotor centrifugal force will allow automatic deletion. Transfer the plate in the instrument and push the button "**Start Run**".

QUANTITATIVE ANALYSIS Lifetechnologies 7500 Fast

At the end of the PCR run, the software automatically opens the "**Analysis**" window in the "**Amplification plot**" sheet on the menu on the left.

Select the wells corresponding to the positive control, negative control and samples for analysis. Select in the "**Option**" window inside the "**Target**" pop-up menu the **Enterovirus target**. Check the correct setting of the threshold. Select in the "**Option**" window inside the "**Target**" pop-up menu the **IC Control target**. Check the correct setting of the threshold. The analysis of the results is made selecting from the menu in the left the page "**Analysis**". From the page "**Standard Curve**", maintaining open the sheet "**view well plate**" in the right side of the software select the wells containing the points of the curve and verify the parameters described in the paragraph "**Interpretation of the Results**" (coefficient of correlation, slope ecc...). From the page "**Amplification Plot**" verify the amplification plot for every single sample. Opening the sheet "**view well table**" in the right side of the software it is possible to verify the data obtained from experiments: Threshold Cycles, emitted fluorescence and the target quantification expressed in copies/reaction or copies/ml, depending on the settings of the calibration curve. Clicking from the menu file and selecting the box export, the window "**export properties**" will open. Indicate the file name, select the position to save it (**Browse**) and click on button "**Start export**". In this way the software will permit to save a excel file with all the data corresponding to selected experiment.

Versant kPCR AD or Stratagene MX3005P/MX3000P

Click on button "**Analysis**" in the toolbar. The software will open in default the sheet "**Analysis Term Setting**". Activate the buttons FAM and HEX in the lower part of the screen and select testing samples. Click on sheet "**results**"; the software will open in default the page "**Amplification plot**". Check the correct setting of the threshold in the specific window "**Threshold fluorescence**", in the menu on the right of the screen. Selecting the box "**Standard Curve**" from menu "**Area to Analyze**" it's possible to visualize the data related to the calibration curve and verify the parameters described in paragraph "**Interpretation of Results**" (coefficient of correlation, slope ecc...). Selecting the box **Text report**" from menu "**Area to Analyze**" in the right side of the screen it's possible to verify the data obtained from the experiments: Threshold Cycles, emitted Fluorescences and the target quantification expressed in copies/reaction o copies/ml depending on the settings of the calibration curve. From the window **Text Report** it's possible to export the results obtained clicking **file, export** on main menu.

Rotor Gene-Q MDx

At the end of the PCR run click "**Option**" button under the amplification plot on green channel. Select "**normalize to Cycling A. orange**". Open the "**Analysis**" window. Select the "**Quantification**" sheet and click on "**cycling A.green/cycling A.orange**". Select from the menu "**Dynamic Tube**" and subsequently "**Slope correct**". If necessary, reduce the background noise of non-specific signals be selecting the "**Outlier Removal**" option and entering a value between 5% and 30%. Check the correct setting of the threshold in the space provided "**CT calculation – Threshold**". Repeat the steps above for each of the fluorescence channels "**cycling A (yellow)**". You can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification **cycling A.green/cycling A.orange** and then the file **cycling A.yellow/cycling A.orange**

CFX96 Real Time PCR System

At the end of the PCR, select the "**quantitation**" sheet. On the top of the screen, select "**settings**" from the menu and choose "**Baseline Threshold...**". You can export the report pushing the paper block figure on the top of the screen.

LightCycler 480

When the run is completed select analysis and choose the correct kind of analysis you want: "**Abs Quant/Fit Points**". Choose the samples subset you want to analyze. Select the "**NoiseBand**" sheet, under the plot you can choose "**NoiseBand (Fluoresc.)**"; and move the line of the NoiseBand on the plot with the mouse of your PC. Repeat this action for each fluorophore using the "**Filter comb**" button. Clicking the sheet "**Analysis**" you can set the threshold choosing the option "Threshold(manual)". After setting parameters push the "**Calculate**" button. Repeat this action for each fluorophore.

INTERPRETATION OF RESULTS

Through the Real Time PCR reaction it is possible to obtain the RNA quantification of Enterovirus RNA, by setting the values of the positive controls of the calibration curve. To calculate these values, all the dilutions steps that the sample has undergone during the extraction and amplification stages must be considered. The system can detect from 1.500.000 to about 15 copies of RNA per reaction. The Ct values obtained from the amplification of 4 controls of known titre are used by the software for the calculation of the calibration curve from which the unknown samples are interpolated.

A proper functioning of the amplification mix can be verified analyzing these parameters:

| Parameters | Reference |
|--|----------------------------|
| RTS conc. 10 ⁵ copie/µl (FAM) | Ct ≤ 22 |
| Correlation Coefficients | 0.990 ≤ r ² ≤ 1 |
| Slope | -3,6 ≤ Slope ≤ -3.1 |
| PCR Efficiency | 90 ≤ Efficiency ≤ 110 |

If the RTS amplification reaction at a concentration of 10^5 copies produces a $C_t > 22$ or undetermined the session can't be considered valid and must be repeated.

Verify that the correlation coefficient value (r^2), the slope or the reaction efficiency fit to the limited indicated in the above table or do not deviate much from them, which represent the ideal range for a proper PCR reaction.

By correctly setting the standards concentration as a function of the extraction system you can get the quantization of the sample directly in copies / ml:

| | Manual Extr. Ref. 52906 (QIAGEN) | Automatic Extr. Ref. 62724 (QIAGEN) | Alternative extract. |
|-------|-------------------------------------|---|----------------------|
| RTS 1 | 35.700.000 copies | 30.000.000 copies | 1.500.000 copies |
| RTS 2 | 3.570.000 copies | 3.000.000 copies | 150.000 copies |
| RTS 3 | 357.000 copies | 300.000 copies | 15.000 copies |
| RTS 4 | 35.700 copies | 30.000 copies | 1.500 copies |

When alternative systems are used sample concentration expressed in copies/ml will be obtained using the formula:

$$copie/ml = \frac{1000}{V_e} \times \frac{E_v}{E_a} \times C_{reaz}$$

where:

- **Ve**: extracted sample Volume expressed in μ l
- **Ev**: eluted sample Volume during extraction stage expressed in μ l
- **Ea**: extracted sample volume used for amplification expressed in μ l
- **Creaz**: copies provided by the instrument.

As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data and the other laboratory tests done on the patient.

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results. The use of positive and negative controls in each amplification session allow to verify the correct functioning of the amplification mix and the absence of any contamination.

In the amplification reaction of each sample, the C_t values for the internal control specific probe are used to validate the analysis session, from extraction process until detection step.

In the amplification reaction of each sample, the C_t values for the internal control specific probe are used to validate the analysis session, from reversetranscription process until detection step.

Be sure that emitted fluorescence from internal control amplification has not a $C_t > 35$ or undetermined. If a sample shows an undetermined Enterovirus RNA and an internal control $C_t > 35$, this means that there have been problems in the extraction stage or in the retro-amplification stage; therefore the sample could be a false negative.

Repeat the sample.

It can be considered valid the samples with a $C_t > 35$ for the internal control, and a high concentration of Enterovirus RNA. In this case, the competitive nature of PCR reaction can hide or disadvantage the internal control amplification.

| Detector FAM | Detector VIC/Hex | PCR Run | Sample |
|-----------------|-------------------------|-----------|--------------|
| Ct undetermined | Ct > 35 or undetermined | Not Valid | repeat |
| Ct undetermined | Ct < 35 | Valid | Negative |
| High Ct | Ct < 35 | Valid | Positive |
| Low Ct | Ct > 35 or undetermined | Valid | HighPositive |

PERFORMANCES

Analytical sensitivity:

It is considered as analytical sensitivity the highest dilution (title) to which a positive sample can be diluted without the system losing the ability to detect with a positivity rate of $\geq 95\%$. The analytical sensitivity of the system was assessed by analyzing synthetic RNA, quantified by spectrophotometric analysis, containing the regions of interest (5'UTR) of the virus in serial dilutions.

Analytical sensitivity of **quany Enterovirus** determined by Probit analysis

| | | Average cps/ μ l | Manual Extraction | Automatic Extraction |
|-----------------|-------------------------|--------------------------|-------------------|----------------------|
| Probit Analysis | Copies/ μ l | 0,37 cps/ μ l | 132,14 cps/ml | 111 cps/ml |
| | 95% confidence interval | Lower 0,11 cps/ μ l | 40,71 cps/ml | 34,20 cps/ml |
| | | Upper. 4,90 cps/ μ l | 1748,93 cps/ml | 1469,10 cps/ml |

Clinical sensitivity:

It is considered as clinical sensitivity the ability to detect true positive samples in the totality of the samples screened as positive. The analysis was made on Enterovirus positive samples and the test was performed following the method recommendations. Positive samples were confirmed with an other CE approved method available on the market.

Obtained results show a clinical sensitivity of 100%.

Traceability versus controls material

Enterovirus 71 RNA Control "Vircell"

To validate the traceability of the **quany Enterovirus** versus Enterovirus 71 RNA Control "Vircell" reference material.

The used control (Enterovirus 71 RNA Control "Vircell, date revised 05/2012), 1 vial with lyophilized RNA of Enterovirus 71, (BrCr strain ATCC VR-784), the concentration will be more than 12500 copie/ μ l once reconstituted.

| Founded Average Log (cps/ μ l) | Awaited Log (cps/ μ l) |
|---------------------------------------|-------------------------------|
| Log 3.42 | Log 4.09 |

Traceability versus controls material

quany Enterovirus was checked versus *QCMD RNA Programme EQA*.

Linearity/Proportionality

The system linearity was valued analyzing syntetic RNA (pGEM-ENT-RNA), quantified by spectrophotometric analysis, containing the regions of interest 5'UTR of the virus in serial dilutions (1:10) from 1.500.000 copie/reactions to 15 copies/reaction of RNA in 15 μ l of extracted material added in the retro-amplification reaction.

The evaluation was performed analyzing 10 calibration curves, that showed these parameters:

| | | |
|--|----------------------------|---------------------|
| RTS conc. 10 ⁵ copies (FAM) | Ct ≤ 22 | Medium Ct 18,49 |
| Correlation Coefficient | 0.990 ≤ r ² ≤ 1 | Medium r2 0,997 |
| Slope | -3,6 ≤ Slope ≤ -3.1 | Medium slope -3,309 |
| PCR efficiency | 90 ≤ Efficiency ≤ 110 | Medium Eff 101% |

Reproducibility and Repeatability

The reproducibility and repeatability of the system were valued analyzing 3 dilutions of synthetic RNA containing the 5'UTR region of interest for Enterovirus and quantified by spectrophotometric analysis, plus a negative control (negative RNA).

For each session, 5 replicates were made in 3 different sessions, performed by different technicians, using 3 different lots of product.

| Hypothetical value | Lot | N° repetitions | Med. Reveal. Conc. | Inaccuracy % |
|--------------------|-----------|----------------|--------------------|--------------|
| 1.500.000 copie | L1/L4 | 45 | 1535894 | 8% |
| 1.500.000 copie | L2/L5 | 45 | 1430681 | 5% |
| 1.500.000 copie | L3/L6 | 30 | 1680742 | 12% |
| 1.500.000 copie | L8/L9/L10 | 45 | 1441482 | 4% |
| 15.000 copie | L1/L4 | 45 | 15564 | 8% |
| 15.000 copie | L2/L5 | 45 | 16075 | 7% |
| 15.000 copie | L3/L6 | 30 | 15456 | 4% |
| 15.000 copie | L8/L9/L10 | 45 | 14877 | 6% |
| 150 copie | L1/L4 | 45 | 142 | 13% |
| 150 copie | L2/L5 | 45 | 153 | 3% |
| 150 copie | L3/L6 | 30 | 129 | 14% |
| 150 copie | L8/L9/L10 | 45 | 166 | 10% |

The average Inaccuracy percentage of Enterovirus method is 8%.

Diagnostic Specificity:

It is considered as diagnostic specificity the ability of the method to detect trues negative samples. The diagnostic specificity of the system was evaluated analyzing human samples tested and confirmed as Enterovirus negative with an other CE marked method available on the market.

Diagnostic specificity is 100% for material extracted from plasma.

Analytical Specificity:

Test's specificity was guaranteed by the use of specific primers for Enterovirus.

The alignment of the choose regions for specific primers' hybridization for Enterovirus with available sequences of the 5'UTR region present in database, demonstrated: their conservation, the absence of significative mutations and the complete specificity for the analysed target.

The analysis "in Silico " showed how the alignment of primers and probes allows the correct identification of the following subtypes :

| Poliovirus | Coxsackievirus A | Coxsackievirus B | Echovirus | Enterovirus |
|------------|------------------|------------------|-----------|-------------|
| 1 | 1 | 1 | 1 | 68 |
| 2 | 2 | 2 | 2 | 69 |
| 3 | 3 | 3 | 3 | 70 |
| | 4 | 4 | 4 | 71 |
| | 5 | 5 | 5 | 73 |
| | 6 | 6 | 6 | 74 |
| | 7 | 7 | 7 | 75 |
| | 8 | 8 | 8 | 76 |
| | 9 | 9 | 9 | 77 |
| | 10 | 10 | 10 | 79 |
| | 11 | 11 | 11 | 80 |
| | 12 | 12 | 12 | 81 |
| | 13 | 13 | 13 | 82 |
| | 14 | 14 | 14 | 83 |
| | 15 | 15 | 15 | 84 |
| | 16 | 16 | 16 | 85 |
| | 17 | 17 | 17 | 86 |
| | 18 | 18 | 18 | 87 |
| | 19 | 19 | 19 | 88 |
| | 20 | 20 | 20 | 89 |
| | 21 | 21 | 21 | 90 |
| | 22 | 24 | 24 | 91 |
| | 24 | 25 | 25 | 92 |
| | | 26 | 26 | 93 |
| | | 27 | 27 | 94 |
| | | 28 | 28 | 96 |
| | | 29 | 29 | 97 |
| | | 20 | 20 | 98 |
| | | 31 | 31 | 99 |
| | | 32 | 32 | 100 |
| | | 33 | 33 | 101 |
| | | | | 102 |
| | | | | 106 |
| | | | | 107 |
| | | | | 113 |
| | | | | 116 |

Highlight the subtypes included panel QCMD 2015 Enterovirus RNA EQA Programme Challenge 1 and Challenge 2 successfully detected by the kit "**quanty Enterovirus**".

Cross-Reactivity:

The exam of alignment of the region chosen for hybridization of primers specific for enteroviruses with the sequences available in the bank data of the 5'UTR region showed the their conservation, the absence of significant mutations and the complete specificity for the target analyzed.

To check the cross-reactivity of the assay, samples tested as positive for other viruses were analysed following the method instructions.

| Sample | Positive sample | Obtained Results cps/ml |
|--------|-----------------|----------------------------|
| 1 | HIV | < 100 |
| 2 | HCV | < 100 |

INTERFERENCES:

Verify that in the RNA extracted from the sample there is no contamination from mucoproteins and haemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorbance of Nucleic Acids) and 280 nm (maximum absorbance of Proteins). A pure RNA should have a ratio of approximately 2.

QUALITY CONTROL

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

In accordance with the Clonit srl ISO EN 13485 Certified quality Management System, each lot of **quany Enterovirus** is tested against predetermined specification to ensure consistent product quality.

BIBLIOGRAPHY

Dubot-Pères A, Tan CY, de Chesse R, Sibounheuang B, Vongsouvath M, Phommasone K, Bessaud M, Gazin C, Thirion L, Phetsouvanh R, Newton PN, de Lamballerie X. **SYBR green real-time PCR for the detection of all enterovirus-A71 genogroups** References: PLoS One. 2014 Mar 20;9(3):e89963

Osterback R, Tevaluoto T, Ylinen T, Peltola V, Susi P, Hyypiä T, Waris M.: **Simultaneous detection and differentiation of human rhino- and enteroviruses in clinical specimens by real-time PCR with locked nucleic acid probes**. References: J Clin Microbiol. 2013 Dec;51(12):3960-7. doi: 10.1128/JCM.01646-13. Epub 2013 Sep 18.

Qian Chena, Zheng Hua, Qihua Zhanga, Minghui Yu. **Development and evaluation of a real-time method of simultaneous amplification and testing of enterovirus 71 incorporating a RNA internal control system**. References: Journal of Virological Methods 196 (2014) 139-144

Bustin SA. (2000). **Absolute quantification of mRNA using real-time reverse transcription polymerase chain reaction assays**. J Mol Endocrinol 25 (2):169-193

Bustin SA. (2000). **Quantification of mRNA using real-time reverse transcription PCR (RT-PCR): trends and problems**. J Mol Endocrinol 29 (1):23-39

Freeman WM1, Walker SJ, Vrana KE. (1999). **Quantitative RT-PCR: pitfalls and potential**. Biotechniques. 1999 26(1):112-22, 124-5.

Larionov A, Krause A, Miller W. (2005). **A standard curve based method for relative real time PCR data processing**. BMC Bioinformatics. 6:62

Singh S, Chow VT, Phoon MC, Chan KP, Poh CL (2002). **Direct Detection of Enterovirus 71 (EV71) in Clinical Specimens From a Hand, Foot, and Mouth Disease Outbreak in Singapore by Reverse Transcription-PCR with Universal Enterovirus and EV71-Specific Primers**. J Clin Microbiol 40(8):2823-7

TECHNICAL ASSISTANCE

For any question and support please contact our Technical support:

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CLONIT S.r.l.