



SZABO SCANDIC

Part of Europa Biosite

Produktinformation



Forschungsprodukte & Biochemikalien



Zellkultur & Verbrauchsmaterial



Diagnostik & molekulare Diagnostik



Laborgeräte & Service

Weitere Information auf den folgenden Seiten!
See the following pages for more information!



Lieferung & Zahlungsart

siehe unsere [Liefer- und Versandbedingungen](#)

Zuschläge

- Mindermengenzuschlag
- Trockeneiszuschlag
- Gefahrgutzuschlag
- Expressversand

SZABO-SCANDIC HandelsgmbH

Quellenstraße 110, A-1100 Wien

T. +43(0)1 489 3961-0

F. +43(0)1 489 3961-7

mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

Instruction for Use

【Product Name】: Sampling Swab

【Product Description】: sampling swab is composed of swab rod and swab head. This product is a sterile, single-use device.

【Product Model】: TYPE-A01, TYPE-A02, TYPE-A03, TYPE-A04, TYPE-A5, TYPE-B, TYPE-C, TYPE-D, TYPE-E, TYPE-F

【Intended Use】: Intended for the collection of respiratory specimens by swabbing the oropharynx, the nasal cavity, and it is also applicable for sampling on skin surface.


【Shelf Life】: Three years from end of manufacturing.

【Sterilization】: EO sterilization

【The combination of the device with product】: N/A

【Instruction for Use】:

- Make sure the product is within the valid period and packaged well. Then open the package bag and take out the swab;
- Sterile gloves and protective clothing and eyewear should be worn when collecting and handling microbiology specimens.

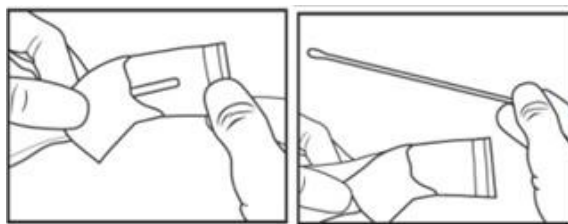

Instruction for Use

【Cautions】

- Please read the IFU carefully before use.
- Single use only.
- Don't use if the package is destructed or damaged.
- The samples used and the culture wastes shall be considered as potentially infectious substances and shall be disposed in accordance with the operating regulation specification of the infectious diseases laboratory before being discarded.
- Samples should be collected by medical professionals and sampling process shall performed strictly following up rules for the sampling procedures stipulated by the state; The samples shall be tested in a laboratory which shall comply with the safety standard.
- Do not re-sterilize unused swabs.
- Do not use if the swab is visibly damaged (i.e., if the swab tip is broken).
- Do not bend disposable sampling swabs prior to specimen collection.
- Beware of swab congestion risk when swabbing is taken.
- Do not use for more than one patient.
- Do not use the swab that have expired.
- Samples collected must be sent for inspection as soon as possible to ensure the accuracy of the results.









【Storage and transportation】:

- This product should be stored indoor at a non-corrosive gas, dry, avoid the sun, good ventilation and clean environment, away from open fire.
- When transported, they should be avoid direct sunlight and rain. Do not use after expiration date, which is clearly printed on the

Instruction for Use

outer box label and on each individual pouch unit.

【Symbol】 :

	Manufacturer		CE mark and Identification number of Notified Body
	Authorized Representative in the European community		Batch code
	Consult instructions for use		Date of manufacture
	Caution		Use by

Instruction for Use



Do not reuse



Keep dry



Sterilized
using irradiation



Keep away from
sunlight



Do not re-sterilize



Use no hooks




Do not use
if
package is
damaged



Keep upright
during
transport



Name: Taizhou Rich
Medical Products Co., Ltd.
ADD: Industrial Area,
Suchen
Town, Taizhou 225300,
Jiangsu, China
Tel: +086-0523-
82771055 **Fax:** +086-0523-
82771050 **E-mail:**
info@richmed2003.com

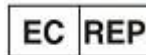

Name: Caretechion GmbH
DIMDI Code
DE/0000048026 CE0197
STERILER 20210416
Address: Niederrheinstr 71,
40474,
Duesseldorf, Germany