



SZABO SCANDIC

Part of Europa Biosite

Produktinformation



Forschungsprodukte & Biochemikalien



Zellkultur & Verbrauchsmaterial



Diagnostik & molekulare Diagnostik



Laborgeräte & Service

Weitere Information auf den folgenden Seiten!
See the following pages for more information!



Lieferung & Zahlungsart

siehe unsere [Liefer- und Versandbedingungen](#)

Zuschläge

- Mindermengenzuschlag
- Trockeneiszuschlag
- Gefahrgutzuschlag
- Expressversand

SZABO-SCANDIC HandelsgmbH

Quellenstraße 110, A-1100 Wien

T. +43(0)1 489 3961-0

F. +43(0)1 489 3961-7

mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

Saliva Collection Kit

Instruction For Use

English VER 0.0

REF Refer to Box Label.

PRODUCT NAME

Saliva Collection Kit

KIT SIZE

1 set/kit, 10 set/kit, 20 set/kit

INTENDED USE

Used for saliva collection.

PRINCIPLE

The ergonomically designed collector is easy for saliva collection. The preservation solution is Hank's Balanced Salt Solution, which can help to stabilize the virus. Phenol red was added to the preservation solution to indicate the preservation state. Antibiotics were added to prevent the growth of bacteria and fungus.

STORAGE CONDITION AND EXPIRATION

Expiration: 3 years.

Storage condition: room temperature (2-30°C).

KIT CONTENT


1. Saliva Collector
2. 5 mL/10 mL Tube pre-filled with 1 mL/2 mL buffer
3. Label Sticker
4. Biosafety Bag
5. Instruction For Use

USAGE METHOD

1. Do not eat, drink, smoke, or use oral hygiene products for at least 10 minutes before you start the collection process.
2. Take off the cap of the preservation solution tube, screw the saliva collector onto the tube.
3. Rinse your mouth with water, wait for at least 5 minutes before sampling.
4. Relax your cheeks, spit total 2 mL saliva into the saliva collector, make sure the amount do not include bubbles.
Note: For detection pathogen related to Respiratory infections, *e.g.*, SARS-CoV-2, please make a deep cough before spitting.
5. Remove the saliva collector and screw up the cap.
6. Invert tube for about 10 times to mix.
7. Fulfill the necessary information on the label provided and stick it to the tube.
8. Put the tube into Biosafety Bag for safe transportation.

* For the kit which contains 1mL preservation solution, 1 mL saliva is sufficient.

QUICK OPERATION GUIDE

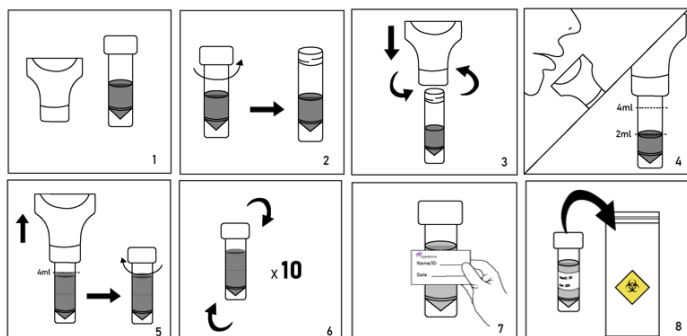

PRECAUTIONS

1. Don't touch the inside of the saliva collector with hands or other items when using the saliva collector.
2. The product is disposal, don't reuse it.
3. Don't use the product when its package is broken.
4. Don't use the product when it expires.
5. The product should be stored in a dry, ventilated, radiation-free, clean room.
6. Don't drink, touch the preservation solution. The solution can be harmful if ingested and may cause irritation if exposed to the skin and eyes.
7. If possible, please wear disposable gloves during whole operation.

Manufacturing date and expiration date: view on label

INDEX OF SYMBOLS

	Consult Instructions for Use		Tests per kit		Authorized Representative
	In Vitro Diagnostic		Use by		Do not reuse
	Do not use if package is damaged		Batch Code		Temperature limit 2-30°C
	Catalog #		Manufacturer		Warning and Precautions
	CE Symbol		Manufacture Date		



Jiangsu Mole Bioscience Co., Ltd
6-7th Floor, G116 Building, No.805, Jiankang Avenue,
Medical New& Hi-Tech District,
Taizhou, Jiangsu Province, China
info@molechina.com
www.molechina.com

EC REP

SUNGO Europe B.V.
Olympisch Stadion 24,
1076DE Amsterdam,
Netherlands
+31(0) 202111106
ec.rep@sungogroup.com



Number: BMA5490202c
Effective Date: 2021-08-26